## DESCRIPTION

VPS-Hydro Bite registration is a Vinyl Polysiloxane derivative impression material. It has a non-slumping consistency and provides no resistance to a bite registration.

## DIRECTIONS

1. Insert the cartridge into the cartridge dispensing gun. Be sure to seat the cartridge firmly so the plungers of the gun will line up properly. Remove cap and discard.
2. Seat mixing tip onto cartridge, secure with a twisting motion of 1/4 turn.  
**Extrude 1/4 to 1/2 inch (0,5 cm to 1 cm) of material to check the mix.**
3. Simply extrude bite paste directly onto the occlusal table. You can vary the ribbon thickness by simply overlaying the material. You will need to fill in any bridge or edentulous area with the material to achieve a continuous bite. Check the impression with a blunt instrument and remove when set. The registration can be rinsed with tap water, dried and poured at your convenience, or sent to the dental lab.
4. It is best to store the multiple use cartridge between applications with the used mixing tip in place. For a new patient, simply replace the mixing tip on the cartridge.

Prevent contact of the material with latex gloves, this can influence the setting.

For complete protection of personnel, the impression can be sterilized by soaking in e.g. Eurosept MAX-Impressions (900-1296) solution as per manufacturer's instructions.

## SPECIFICATIONS

Working Time:	30 seconds
Set Time:	90 seconds (includes workingtime)
Durometer:	95
Shelf Life:	3 years

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Henry Schein U.K. Holdings Ltd.