Capsule Mixer Cap II (GB)

Instructions:

The Capsule Mixer Cap II can be used either manually or with the preset memory times. Plug unit into the nearest power socket. The ON switch is on the left hand side of the unit.

1. Manual Working:

- Set the time using the '+' and '-' buttons.
- Press 'S' to start. The display will countdown. It will return to the set time when finished.

2. Working with Preset times:

The machine is supplied with 10 preset mixing times:

Estimated time for 2 doses:

Memory 0 = sec. 6	----> IQC Palladium / Contour
Memory 1 = sec. 6	----> IQC Valiant / Dispersalloy
Memory 2 = sec. 10	----> Axia Glass / Amalcap
Memory 3 = sec. 10	----> Axia Base
Memory 4 = sec. 10	----> Axia Silver
Memory 5 = sec. 5	----> Tytin
Memory 6 = sec. 7	----> Duralloy / Ana2000 / Pernite C
Memory 7 = sec. 8	----> Astralloy PD/45
Memory 8 = sec. 15	---->
Memory 9 = sec. 20	---->

- Press 'M' - this will indicate the preset number.
- Press the button until the preset you require is indicated.
- Once selection is made, press the 'S' button to start. The display will countdown. At the end of the cycle it will return to the selected preset time.
- When you have selected the right time of mixing, push the button "S" to make operation. During the mixing, you can see decreasing time. At the end of cycle, display will show the number of memory you used.

The Built in safety feature stops the machine if the cover is opened during mixing. The machine can be stopped by pressing the 'S' button during operation. When the machine is switched off it will retain the last time set. The units takes capsules 30-36mm in length and 14/18 mm diameter. Use a soft cloth with neutral detergent to clean the machine. Avoid solvents and abrasive materials. Do not use the equipment in areas or with materials for which it is not designed.

Built according to EN 61010-2-51 : 1995 – ISO 7488 rules.

Specifications:

Volt: 220/240 AC (110/115 AC) – Hz 50/60
Power: Watt 180 (130)
Oscillation frequency: 4200rpm
Dimensions: 220x230x180 mm
Weight: 4 Kg
Operating temperature: between +5° and +40°
Working time: 30 sec ON – 90 sec OFF
Maximum working time: 30 min

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Henry Schein U.K. Holdings Ltd.