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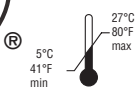
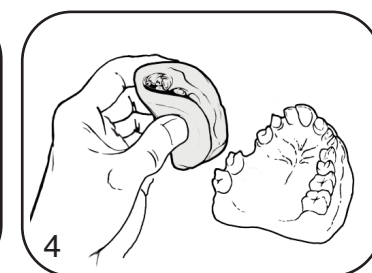
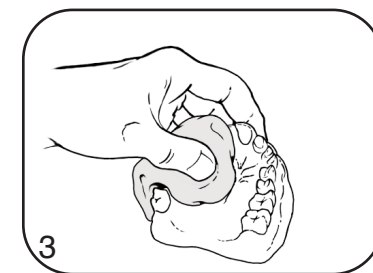
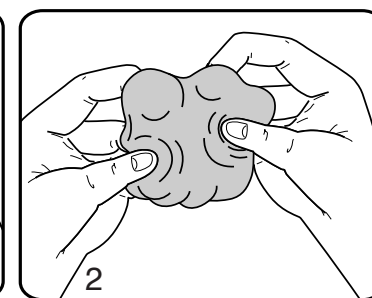
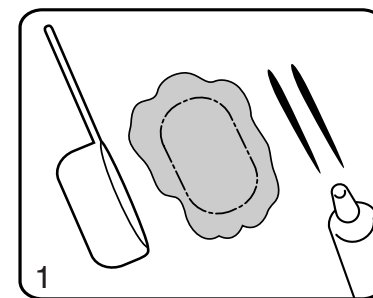
Distributed by: (US only)  
HENRY SCHEIN INC.  
135 DURYEA ROAD  
Melville, NY 11747 USA  
HENRY SCHEIN U.K. HOLDINGS LTD.  
MEDCARE HOUSE, CENTURION CLOSE  
GILLINGHAM BUSINESS PARK  
GILLINGHAM, ME8 0SB U.K.  
www.henryscheinbrand.com  
Rev. 2011/12

**GB****INSTRUCTIONS FOR USE**

**Eurosil 2 Lab** HIGH PRECISION CONDENSATION SILICONES (POLYSILOXANES) FOR THE DENTAL LABORATORIES

EUROSIL 2 Lab which may be mixed with Eurosil 2 paste catalyst, characterised by good final hardness, high stability and capacity of reproduction of details (> 80 Shore A) and resistance to deformation.

**FIELD OF APPLICATIONS**

- Masks for the manufacture of temporary prosthetics; addition or replacement of elements in total or skeletally anchored prosthetics
- Removable gingival masks
- Partial counter-moulding in the muffle for partial or total prosthetics
- Anchorage in prosthetics repair
- Occlusal adjustment key for positioning in the articulator
- Duplication of total prostheses and plaster models without use of insulation
- Bases for working with photo-polymerising compound resins.

**INSTRUCTIONS FOR USE****Dosage:**

Use the measuring spoon to take the required quantity of EUROSIL 2 and spread it on the palm of your hand (note: the measuring spoon must be leveled). Press the edge of the measuring spoon onto the mass once for each spoon used. For every spoon used, apply two strips of Eurosil 2 Paste catalyst as long as the measuring spoon, which is 4 cm (Fig. 1).

**Mixing / use:**

Fold the mass over on itself and knead it energetically with your fingertips for about 30 sec. until colour is homogeneous, without stripes. (Fig. 2). Now begin to model the mass as required. You have about 2 minutes to work with the mass, including mixing time, at room temperature if paste catalyst is used. Hardening takes about 6 min. (Fig. 4).

**WARNINGS / PRECAUTIONS**

The times given are calculated from the start of mixing at a temperature of 23°C (73°F). High temperatures or excessive use of catalyst will speed up hardening. Low temperatures or insufficient catalyst will slow it down. Close indurents properly after use. Do not stain clothing. Use of gloves is recommended. Avoid contact of catalyst with skin and eyes. In the event of accidental contact with the skin, wash well with soap and water. In the event of contact with the eyes, rinse well with water and seek medical attention. If ingested, seek medical attention immediately.

**STORAGE**

Guaranteed for 36 months if stored correctly at temperatures between 5° and 27°C (41°-80°F).

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