S-Spray

- EN Universal spray for handpieces
- ES Aerosol universal para piezas de mano y contra-ángulos
- FR Spray universel pour turbines, pièces à main et contre-angles
- DE HS S-Spray für Hand- und Winkelstücke
- IT Spray universale per manipoli e contrangoli
- NL Universele spray voor hand- en hoekstukken
- PT Spray para peças de mão e contra-angles
- PL Spray do końcówek i kątnic
- CS Sprej pro ruční nástavce a protiúhelníky
- DA Spray til håndstykker og kontra-angler
- SV Spray för handstycken och motvinklar
- NO Spray for håndstikker og vinkelstikker

Danger | Peligro
 Danger | Gefahr
 Pericolo | Gevaar
 Perigo | Nebezpečí
 Niebezpieczeństwo
 Fare / Fara | Fare

UFI: MP52-9V5T-X10D-6DR5

**UK
CA**

UK Responsible Person
 Emergo Consulting (UK) Limited
 c/o C-360 – UK International
 Compass House, Vision Park Histon
 Cambridge CB24 9BZ
 United Kingdom

335 3 MD

GTIN 4064797000465

250 ml e

Exclusively produced for:
 HENRY SCHEIN®

PRISMAN GmbH
 Otto-Hahn-Ring 6-18
 D-64653 Lorsch
 Phone: +49 (0)6251-866 980-0

EN: Type of application: For the care of handpieces and contra-angle handpieces, heads and turbines as well as air engines. **Instructions for use:** To regularly maintain the instruments, spray them before each sterilisation cycle or at least twice a day for approx. 2 seconds. Hold can vertically during use and use with suitable adapter. **Safety instructions:** H 222-H 229 Extremely flammable aerosol. Pressurised container: may burst if heated. Keep out of reach of children. Keep away from heat/sparks/open flames/hot surfaces. No smoking. Do not spray on an open flame or other ignition source. Pressurised container: Do not pierce or burn, even after use. Use only outdoors or in well-ventilated areas. Protect from sunlight. Do not expose to temperatures above 50°C/ 122°F. **Notes on incidents:** All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.

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