DESCRIPTION

VPS CAD/CAM is a vinylpolisiloxane derivative impression material. It has a non-slumping consistency and provides no resistance to a bite registration. It is reflection active and can be used for powderfree optical registration of antagonists, for CAD/CAM/CIM (Computer Aided Design/Computer Aided Manufacturing/Computer Integrated Manufacturing) impression technology.

DIRECTIONS

1. Insert the cartridge into the cartridge dispensing gun. Be sure to seat the cartridge firmly so the plungers of the gun will line up properly. Remove cap and seat the green mixing tip.
2. Seat mixing tip onto cartridge, secure with a twisting motion of 1/4 turn. Extrude 1/4 to 1/2 inch (0,5 to 1 cm) of material to check the mix.
3. Simply extrude VPS CAD/CAM directly onto the occlusal table. You can vary the ribbon thickness by simply overlaying the material. You will need to fill in any bridge or edentulous area with the material to achieve a continuous bite. Check the impression with a blunt instrument and remove when set.
4. Prepare a mesial and a distal extension of the registration and replace it onto the preparation. The mesial, the registered antagonist and the distal occlusal area can be then optically registered.
5. It is suggested to store the multiple use cartridge between applications with the used mixing tip in place. For a new patient, simply replace the mixing tip on the cartridge and use again, you can also replace the mixing tip with the cap.

Prevent contact of the material with latex gloves, this can influence the setting.

For complete protection of personnel, the impression can be desinfected by soaking in e.g. Henry Schein Eurosept MAX-Impressions (9001296) solution as per manufacturer's instructions.

SPECIFICHE

Working Time:	30 seconds
Set Time:	90 seconds (includes working time)
Durometer:	95 Shore-A
Shelf Life:	3 years

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Henry Schein U.K. Holdings Ltd.