Dokuments
K9 4930.

Always on the safe side.

KaVo. Dental Excellence.

Vertrieb/distribution:

KaVo Dental GmbH

Bismarckring 39 • D-88400 Biberach

Tel.: 0 73 51 / 56-1600 • Fax: 0 73 51 / 1488

Hersteller/manufacturer:

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Bismarckring 39

D-88400 Biberach

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KaVo. Dental Excellence.

Vertrieb/distribution:

KaVo Dental GmbH

Bismarckring 39 • D-88400 Biberach

Tel.: 0 73 51 / 56-1600 • Fax: 0 73 51 / 1488

Hersteller/manufacturer:

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Bismarckring 39

D-88400 Biberach

K9 4930.

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We declare that the products manufactured by us to which this statement refers, conform to the essential requirements according to the above-mentioned directive. The specified standards or normative documents were applied to evaluate the product.

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D-88400 Biberach

K9 4930

A 1	User information	2
A 1.1	Meaning of the pictograms	2
A 1.2	Important information	2
A 1.3	Precautions	2
A 1.4	Purpose and applications	2
A 1.5	Technical data	3
A 2	Scope of delivery - accessory	3
A 3	Description	4
A 4	Commissioning	4
A 5	Operation	4
A 5.1	Inserting / changing the tool	5
A 5.2	Operation	5
A 6	Maintenance	5
A 6.1	Weekly cleaning of chuck and handpiece part	6
A 6.2	Replacing the ball bearings	6
A 6.3	Changing the connecting cable	8
A 6.4	Monthly cleaning of the cooling air vents on motor part and locking cap.	9
A 6.5	Changing the carbon brushes	9
A 7	Malfunctions	9
	Guarantee conditions	10
	Spare parts	12
	Declaration de conformity	14

K9 4930.

A 1 User information


A 1.1 Meaning of the pictograms

Situations where failure to follow the instructions may lead to danger, damage to material or operating faults.

 *Important information for operator and engineer.*

 Automatic mode
Automatic sequence

 Close, screw in,
fasten, etc.
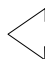
 Open, release, loosen

 more, higher

 less, lower

 Continuous operation

 Time, time sequence

 Disconnect mains plug

A 1.2 Important information

The instructions for use should be read by the user before starting up the unit for the first time, in order to avoid incorrect operation and other damage. If other language versions are required, please request these from your responsible KaVo agent. Duplication and distribution of the instructions for use (IU) require KaVo's prior consent.

All technical data, information and properties of the product described in the IU correspond to the state on going to press.

Modifications and improvements to the product as a result of new technical developments are possible.

This does not imply any right to retrofitting of existing units.

KaVo assumes no responsibility for damage arising through:

- external influences (poor quality of the media or inadequate installation)
- use of incorrect information
- improper use
- improperly performed repairs.

Repair and maintenance work - apart from the activities described in these instructions for use - may be performed only by qualified technical personnel.

In the event of modifications by third parties, the approvals become null and void. KaVo recommends using only original spare parts for operation and for repair.

 *Please note that the EC Directive on waste electrical and electronic equipment applies to this product. Within Europe therefore, this product must undergo special disposal. For more detailed information about this, please contact KaVo or your specialist dental supplier.*

A 1.3 Precautions

Safe operation and protection of the unit are ensured only through proper use in accordance with the instructions for use and using the tools approved for the purpose. The following should also be observed:

- the tool manufacturer's instructions,
- the work safety regulations,
- the accident prevention regulations.

■ *Each time before switching on, check the set speed.*

■ *Observe the permissible maximum speed and maximum pressure of the tools (according to tool manufacturer's instructions).*

■ *Use safety screens when working with rotating tools.*

■ *To avoid danger through accidental switching on, place the handpiece on a suitable shelf or tool support.*

In the event of an unsatisfactory condition of the unit or improper use, e.g.:

- unsuitable tools
- tool shafts not manufactured according to DIN-ISO
- improper use or use not in accordance with the purpose
- unapproved speeds for tools used
- incorrect clamping of the tools in the chuck

- insufficient retaining force of the chuck (wear, soiling, failure to follow the product care instructions for the chuck system, etc.)
- different sizes of tool shaft and chuck
- lack of regular cleaning of the chuck
- failure to follow the maintenance instructions
- failure to comply with the accident prevention regulations (e.g. failure to use safety screens, safety devices, handpiece supports, etc.)
- failure to take into account signs of wear and damage
- tool shafts which have slipped out (potential danger = bending of the tool shafts)

there is a danger of injury and damage to material and unit, e.g. due to:

- Bending of the tool shafts
- Accidental withdrawal of the tools from the chuck
- Breaking or splintering of the tool.

K9 4930


A 1.4 Purpose and applications

K9 4930 handpieces are versatile and very particularly suitable for working on crowns and bridges. Suitable for:

Long-term operation at 5,000 - 25,000 min⁻¹
Brief operation at 1,000 - 5,000 min⁻¹

A 1.5 Technical data

Length:	190 mm
Handpiece part:	max. Ø 27 mm
Motor part:	max. Ø 32 mm
Weight:	approx. 230 g
with connecting cable	approx. 420 g
Output power	max. 42 watt max. 3.3 Ncm
Speed range	1,000-25,000 min ⁻¹

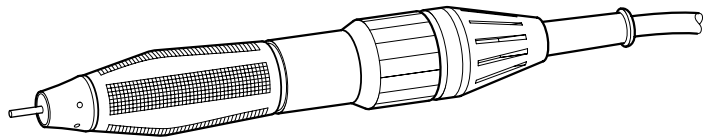i *In the presence of extreme interfering high-frequency fields, speed variations up to max. 20% may occur.*

Ambient conditions:

Permitted in interior rooms

Permissible ambient temperature range of 5° C - 40° C

Permitted to max. relative humidity 80%

Intermittent operation: on 2 min / off 8 min

We reserve the right to make technical modifications.

K9 4930

A 2 Scope of delivery - accessory

Scope of delivery


K9 handpiece 4930
 with chuck \varnothing 2.35 mm
 Care set Mat. No. 0.411.3180
 Instructions for use Mat. No. 1.000.2952

Accessory

Available on request:
 Handpiece support 4850
 Mat. No. 0.642.0352

Tool for changing the ball bearings
D Key Mat. No. 0.411.3962

E Socket wrench Mat. No. 0.411.2310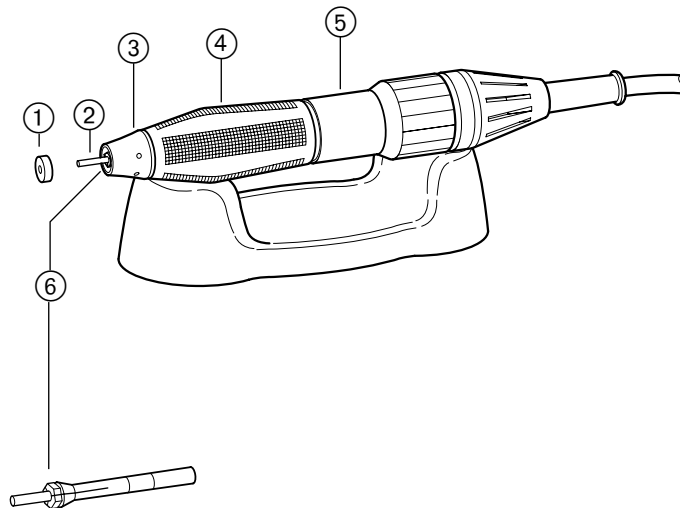

A 3 Description

① Cover (3 covers as spare parts included in care set)

② Straight pin

③ Grip tip

④ K9 handpiece part

⑤ K9 motor part

⑥ Chuck


A Chuck key

B Clip

C Cleaning brush

D Key

E Socket wrench

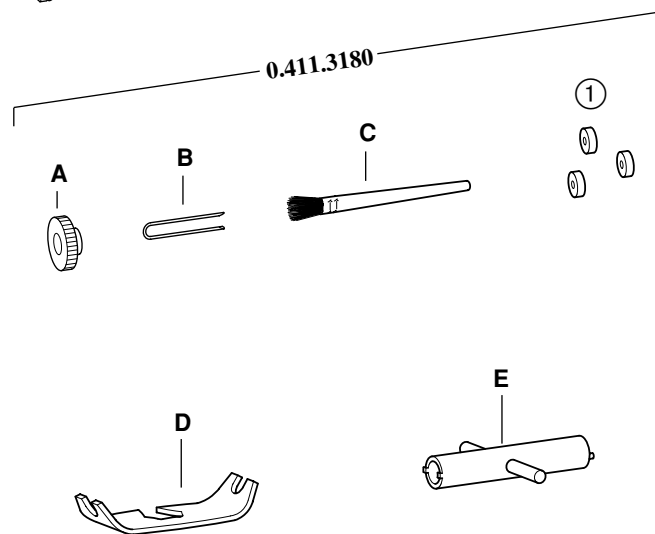
A 4 Commissioning

The K9 handpiece 4930 can be operated with the following controllers:

Knee	K9	4961
Table	K9	4966
Foot	K9	4971

Knee	K9	920
Table	K9	915
Foot	K9	900

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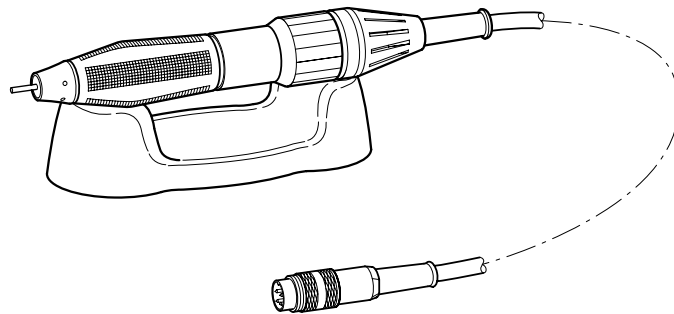
A 5 Operation

A 5.1 Inserting / changing the tool

 During commissioning and each time when operating the K9 handpiece, it is essential to observe the points mentioned in the section «A 1.3 Precautions»!

Opening the chuck


Turn handpiece part ④ and motor part ⑤ in the direction of the arrow until the chuck ⑥ is completely open.

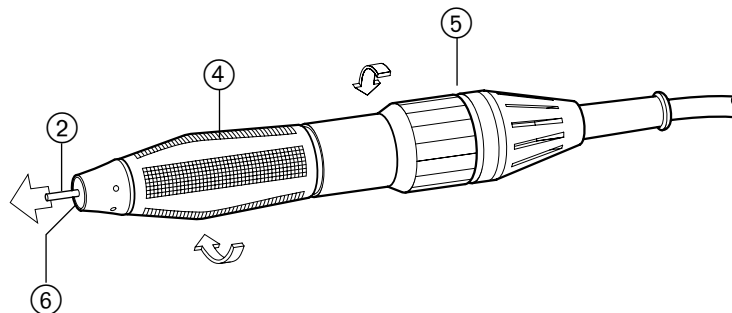Inserting / changing the tool

Closing the chuck

Turn handpiece part ④ and motor part ⑤ in the direction of the arrow as far as they will go.

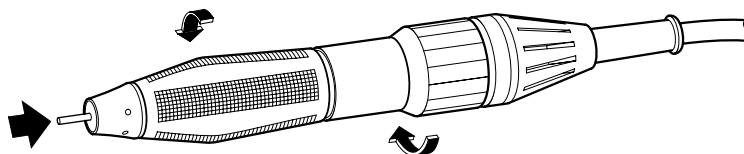A 5.2 Operation

 If the workplace is unsuitably illuminated, a stroboscopic effect may occur. This causes the tool to appear stationary at certain speeds. The situation can be remedied only by acquiring suitable lighting.

Each time before switching on, check the desired speed on the controller and if necessary adjust.

Put the K9 handpiece into operation according to the instructions for use of the up-circuit controller.

K9 4930

A 6 Maintenance

■ Repair and maintenance work on the electrical part of the unit may be performed only by qualified technical personnel or trained persons who have been made aware of the safety regulations. Before maintenance work, disconnect the mains plug or isolate the unit completely from the power supply so that there is no power to the unit.

■ On no account may the K9 handpiece be cleaned with compressed air; use the cleaning brush from the care set.

■ On no account should cleaning agents (such as spray cleaner, degreaser, etc.) be introduced into the interior of the handpiece.

A 6.1 Weekly cleaning of chuck and handpiece part

Removing the chuck

Remove previously used tool or straight pin from chuck and take off cover ① (3 covers as spare parts in care set).

Push tool or straight pin into chuck as far as it will go and close chuck.

Slowly turn the shaft ② until clip **B** can be inserted into grip tip ③ with little resistance.

Open the chuck. Unscrew the chuck using original chuck key **A** and inserted straight pin or tool ②.


Clean the front opening of the K9 handpiece with a brush or with the cleaning brush supplied. Clean chuck thread, surface of the original chuck key **A** and rotor cone and lightly grease.

Installing the chuck

Insert chuck incl. straight pin ② using chuck key **A** and tighten.

Mount cover ① again.

Remove clip **B** from grip tip ③ and close chuck with inserted straight pin again.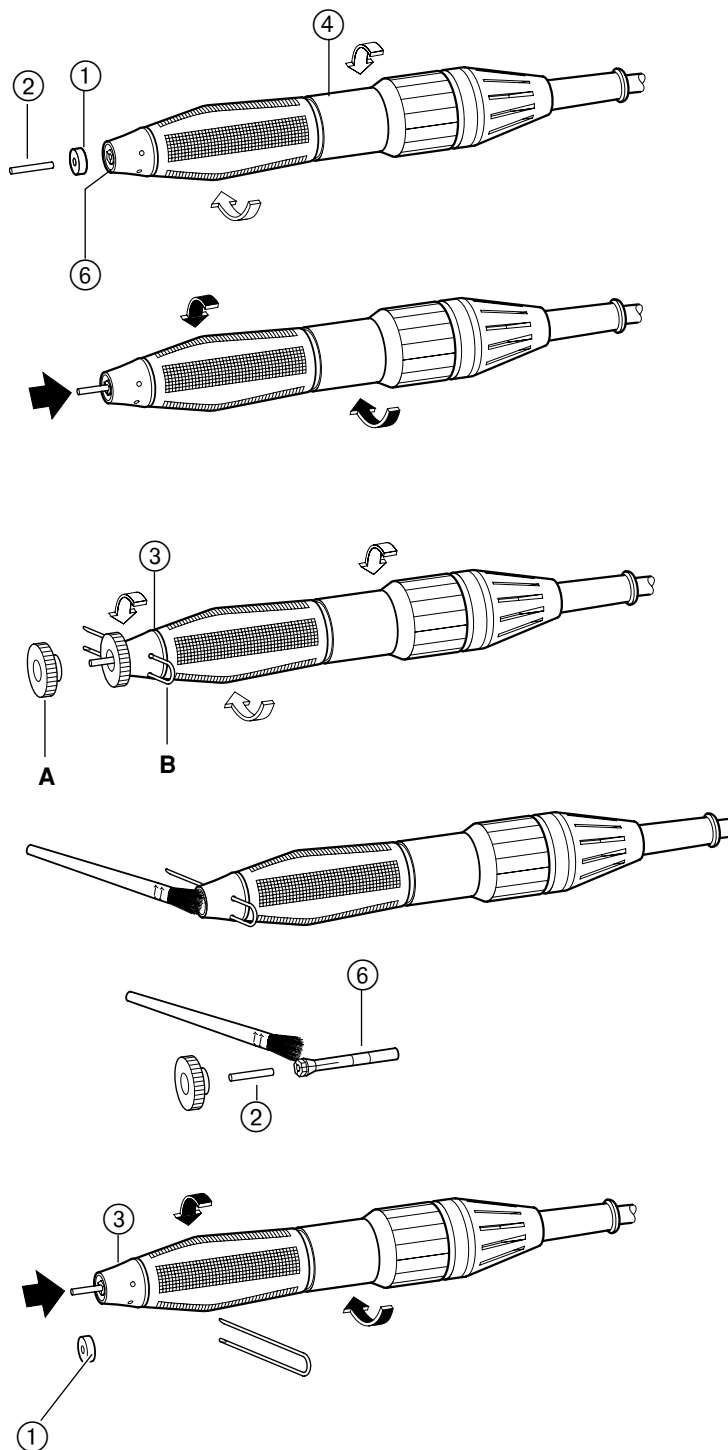


K9 4930


A 6.2 Replacing the ball bearings

Dismantling


Slowly turn the shaft by the straight pin ② until clip **B** can be inserted into grip tip ③ with little resistance.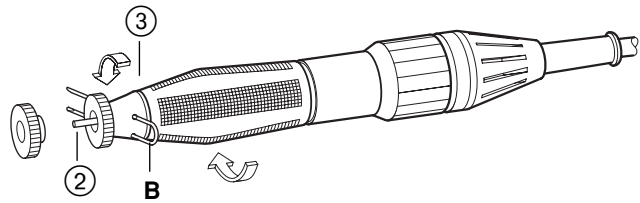


Loosen grip tip in direction of arrow ← and remove clip. Completely unscrew grip tip and remove felt washer ⑦.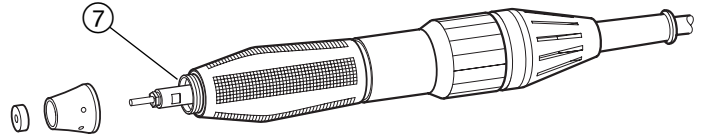


Grip handpiece part ④ and loosen threaded ring ⑧ with socket wrench **E** and unscrew.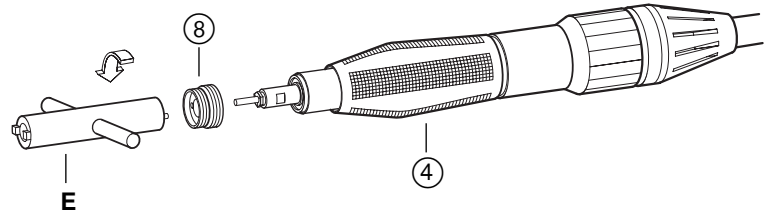


Remove complete shaft unit ⑨ from K9 handpiece.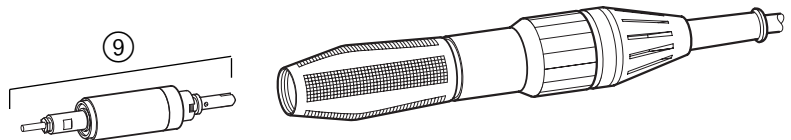


Remove ball bearing ⑩ and bush ⑪ from shaft.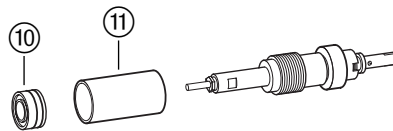

Hold shaft with key **D** and loosen nut ⑫ with socket wrench **E** - Note left-handed thread.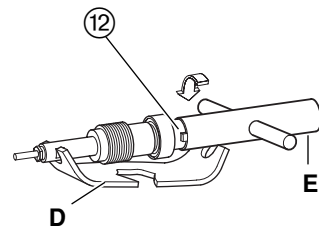


Remove nut ⑫ with ball bearing ⑬ from shaft.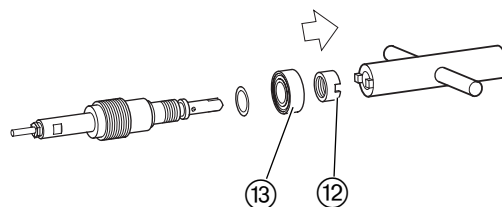


K9 4930

Assembly


Push new ball bearing ⑬ onto shaft. Hold shaft with key **D** and screw on nut ⑫ again with socket wrench **E** - note the left-handed thread.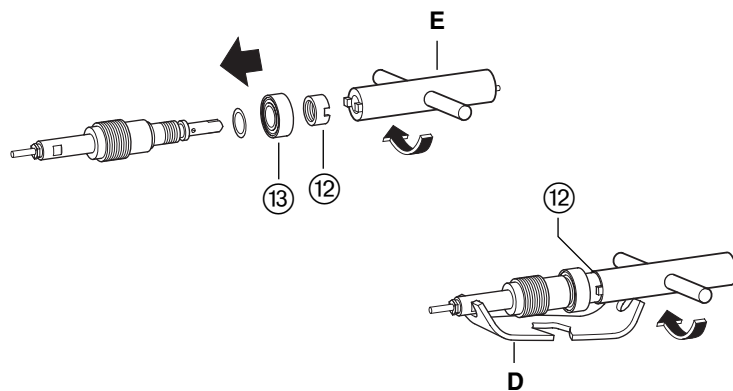


Push ball bearing ⑩ (with the two O-rings facing the inside of the handpiece) and bush ⑪ onto the shaft as far as it will go.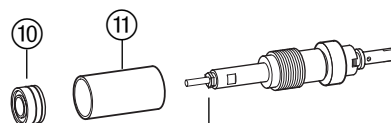


While rotating the shaft unit ⑨, insert it into the K9 handpiece.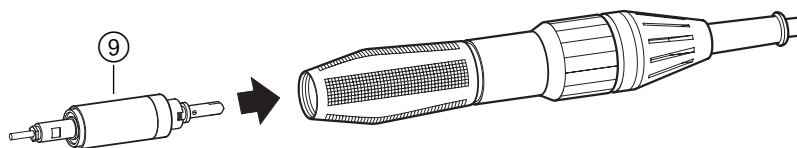


Grip handpiece part ④ and tighten the threaded ring ⑧ with socket wrench **E**.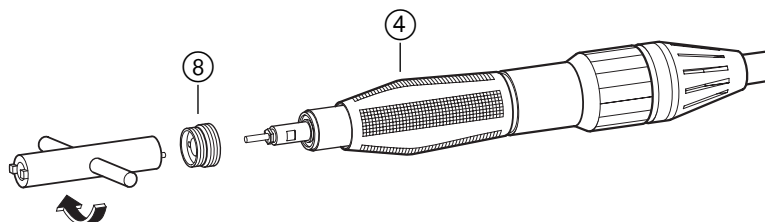

Insert new felt washer ⑦ into threaded ring ⑧.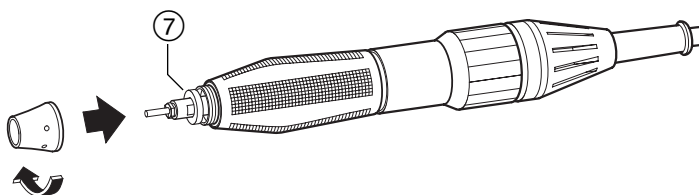

Hold K9 handpiece, screw on grip tip by hand and tighten carefully with clip **B**. Insert cover ①.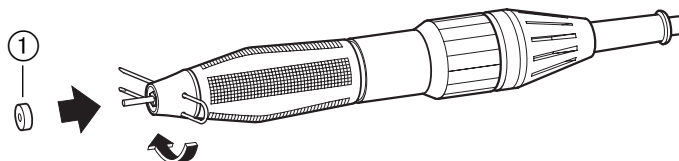


i To increase the life of the ball bearing, KaVo recommends:

■ After changing the ball bearing, allow the handpiece to run without an inserted instrument for about 60 minutes at 10,000 to 15,000 rpm.


K9 4930

A 6.3 Changing the connecting cable

Unscrew locking cap from the motor part and disconnect the connecting cable in the direction of the arrow.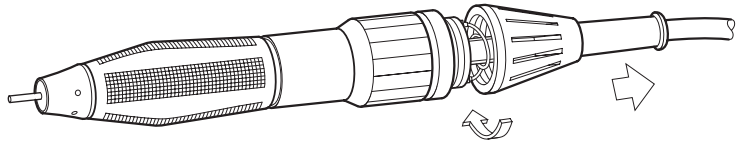

Insert new connecting cable (2-pin) completely into motor part and screw back locking cap.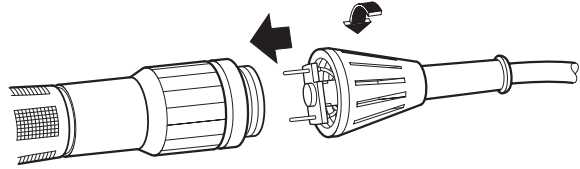


A 6.4 Monthly cleaning of the cooling air vents on motor part and locking cap.

Unscrew locking cap from motor part ⑤ and disconnect connecting cable.

Clean locking cap and interior of motor part with cleaning brush.

 *On no account should cleaning be carried out with compressed air!*

Mount connecting cable and locking cap again.

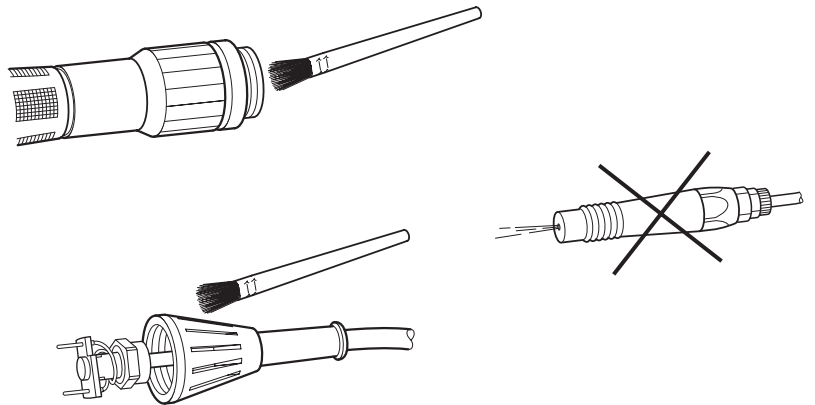
A 6.5 Changing the carbon brushes

Remove connecting cable from motor part (see A 6.3).

Unscrew cheese-head screw ⑭ and remove brush holder ⑮ incl. carbon brushes ⑯.

Replace carbon brushes ⑯ and insert brush holder ⑮ with carbon brushes back into motor part ⑤.

Mount connecting cable again (see A 6.3).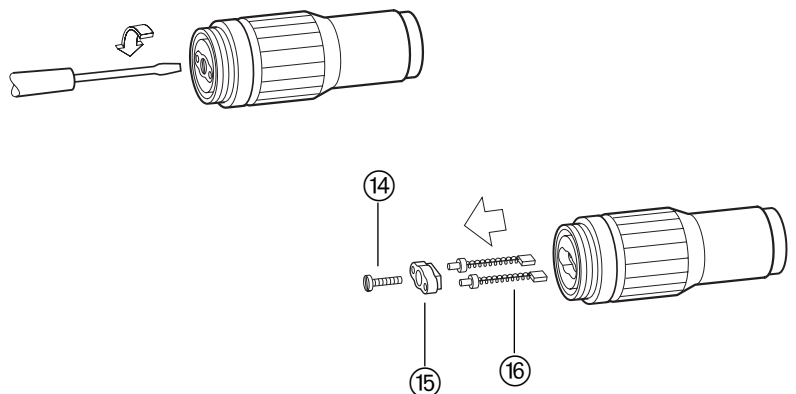

K9 4930

A 7 Malfunctions

 *Repair and maintenance work on the electrical part of the unit may be performed only by specialists or by persons who have been trained in the factory and have been made aware of the safety regulations. Before opening the housing parts, disconnect the mains plug or isolate the unit completely from the power supply so that there is no power to the unit.*

F • K9 handpiece no longer rotates or stops from time to time.

C • K9 handpiece is blocked, possibly due to open chuck system.
R • Close chuck correctly with inserted tool (see **A 5.1**).

C • Break in connecting cable.
R • Check connecting cable incl. plug devices and if necessary replace.

C • Carbon brushes worn.
R • Replace carbon brushes (see **A 6.5**).

C • Fault in the electrical system of K9 controller.
R • Check K9 controller or have repaired.

F • Tool is no longer adequately retained in the chuck..

C • Diameter of tool shaft does not agree with chuck.
R • Use chuck or tool with appropriate diameter.

C • Chuck has become loose.
R • Insert chuck correctly (see **A 6.1**).

C • Chuck is worn.
R • Replace chuck (see **A 6.1**).

F • Loud running noise or K9 handpiece heats up

C • Ball bearing faulty owing to wear or soiling.
R • Check ball bearing and if necessary replace. (see **A 6.2**)

C • Prolonged operation under very high load.
R • Eliminate cause of high load, if necessary increase speed or use tools with smaller diameters.

C • Electronics of K9 controller faulty.
R • Check K9 controller or have repaired.

F = Fault **C** = Cause **R** = Rectification

K9 4930.

Guarantee conditions

Within the framework of applicable KaVo delivery and payment conditions, KaVo guarantees proper function, freedom from flaws in material and manufacturing for a period of 12 months from the date of purchase demonstrated by the purchaser.

In case of justified complaints, KaVo will honor its warranty with a free replacement or repair.

The warranty does not cover defects and their consequences that arose or may have arisen due to natural wear, improper handling, cleaning or maintenance, noncompliance with operating, maintenance or connection instructions, corrosion, contaminated media supply or chemical or electrical influences deemed abnormal or impermissible in accordance with factory specifications.



The warranty does not usually cover lamps, light conductors made of glass and glass fibers, glassware, rubber parts and the colourfastness of plastic parts.

The warranty expires when defects or their consequences can arise from manipulations or changes to the product. Warranty claims can only be asserted when they are immediately reported to KaVo in writing.



This notification must be accompanied by a copy of the invoice or delivery note on which the manufacturing number is clearly visible. In addition to the guaranty, the statutory warranty claims of the purchaser also apply with a warranty period of 12 months.

K9 4930.


A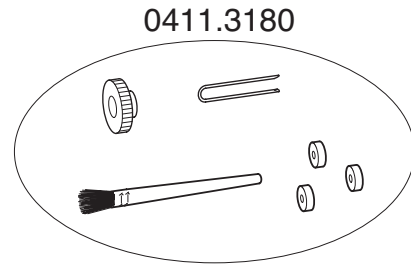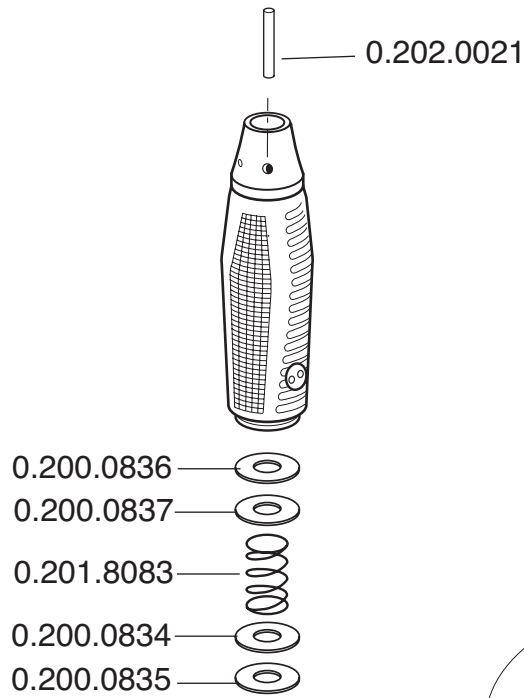

B

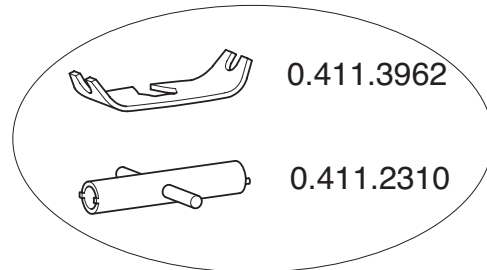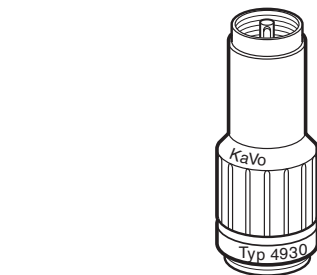
C

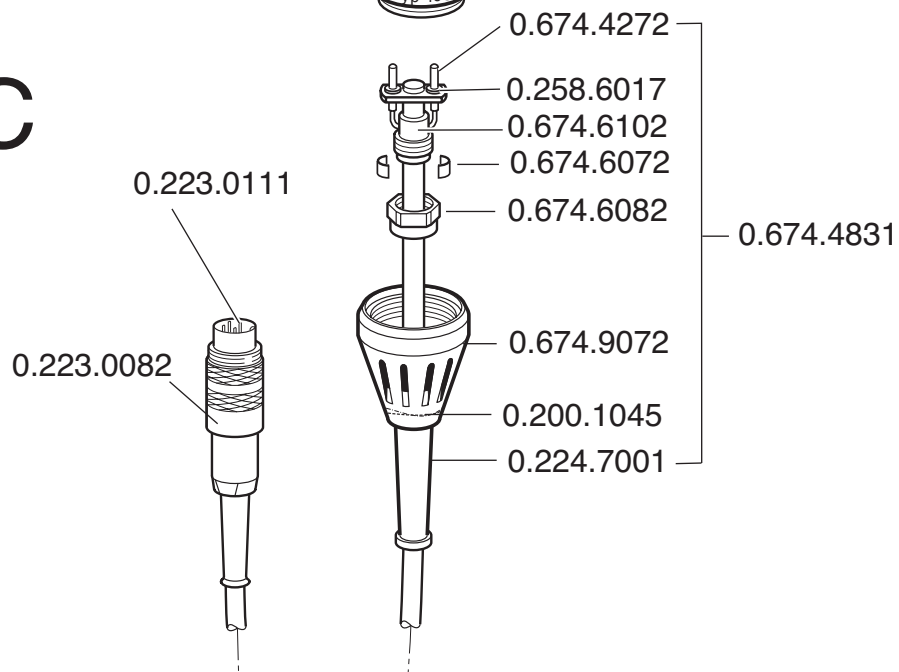
K9 4930.

A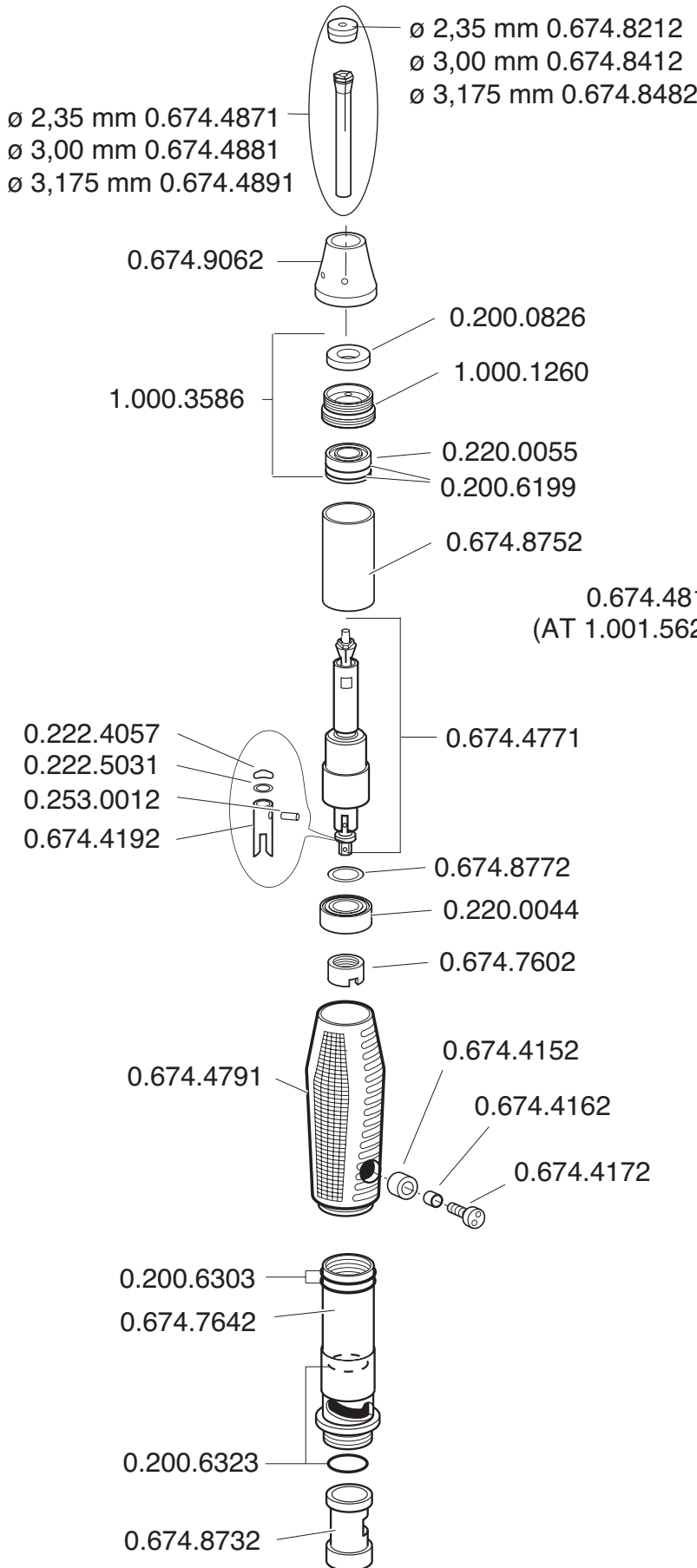

B

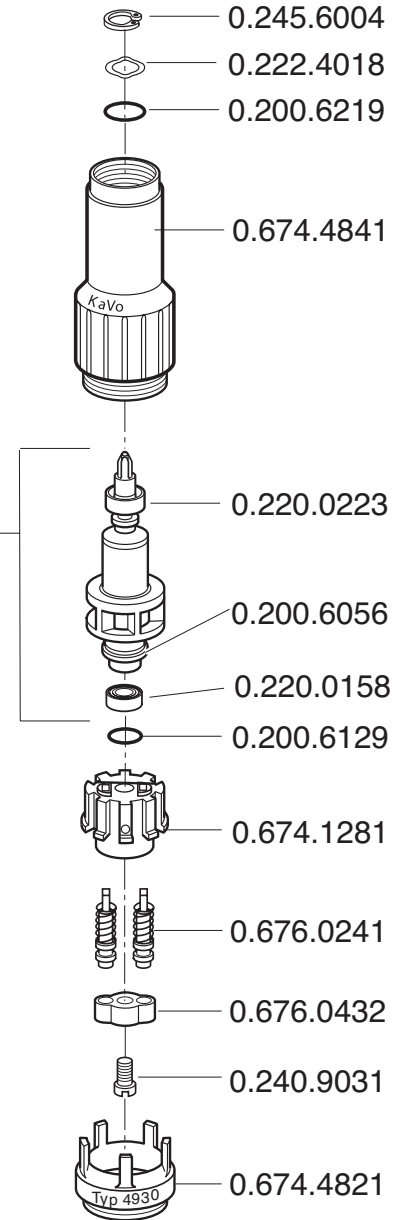KaVo. Dental Excellence.

EG-Konformitätserklärung
EC-Declaration of conformity
CE-Déclaration de conformité
Dichiarazione di conformità con le norme CE
Prohlášení o shodě s normami ES
Declaración de Conformidad de la CE

Manufacturer:
Kaltenbach & Voigt GmbH
Bismarckring 39
D-88400 Biberach

Controller K-Control Type 4960/4965/4970
in connection with handpiece
K-ERGOgrip 4944 / POWERgrip 4941 / K5plus 4911 /
K9 - 950 / K9 - 955 / K9 - 960 / K9 - 970 / K9 - 4930 / SF 4005
K5 - 4910 / K10 - 4950 / K11 - 4990 / K12 - 4940

2006/95/EC
2004/108/EC
EN 61010-1

EC Low Voltage Directive
EMC Directive
Safety requirements for electrical equipment
for measurement, control and laboratory use.
General requirements

EN 61326-1

Electrical equipment for measurement, control
and laboratory use – EMC requirements –
Part 1: General requirements

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