Intended use

Cleaning tablet for external cleaning of dental splints.

(Recommended) Indication

Cleaning product for preservation of dental splints.

Contra indication

The usage of these cleaning tablets is only applicable for dental splints; if any allergy to mint oil exists it may come to an allergic reaction.

Patient target group and designated operators

Cleaning product for dental splints to be used by wearer of dental splints as well as qualified personnel.

Frequency of use

We recommend daily cleaning. This will prolong the durability of your top-quality dental splint and safeguard an impeccable hygiene and function.

Application

Put the cleaning tablet into the MaxiFresh™ bowl filled with 150 ml of water with a temperature of 37 °C.

Please note: In cooler water the cleaning tablet might not dilute completely. After the dissolution of the tablet, the reaction time should be at least 15 minutes, the maximum should be 60 minutes.

When reaction time has passed take the dental splint out of the MaxiFresh™ bowl and – with the help of the insert – rinse thoroughly with clear water. Then dry the surface.

Accordng information

Any critical incidents which occur in conjunction with these products have to be noticed to the manufacturer and the responsible authority.

Keep away from children!**Storage conditions**

At room temperature, i. e. 18 °C–28 °C

Shelf life: see imprint

MaxiFresh™ is not to be used when expiry date has passed.

Forms of delivery

[REF] D4100

24 MaxiFresh™
cleaning tablets incl.
cleaning bowl

[REF] D4101

24 MaxiFresh™
cleaning tablets refill set

Contains

pentapotassium bis(peroxymonosulphate)bis(sulphate), disodium carbonate, compound with hydrogenperoxide (2:3), anionic surfactant

Additional information

According to Regulation (EC) No 648:

Colorant (< 0,1%)

Nonionic Tenside (≤ 0,7%)

Mint oil / Perfume oil (≤ 0,3%)

Contains Mint oil. May produce an allergic reaction.

Hazard statements

H314 Causes severe skin burns and eye damage.

H335 May cause respiratory irritation.

Precautionary statements

P101 If medical advice is needed, have product container or label at hand.

P102 Keep out of reach of children.

P103 Read label before use.

P260 Do not breathe dust / fume / gas / mist / vapours / spray.

P303+P361+P353

IF ON SKIN (or hair): Remove/ Take off immediately all contaminated clothing. Rinse skin with water/ shower.

P305+P351+P338 IF IN EYES:

Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P310 Immediately call a POISON CENTER or doctor/ physician.

P405 Store locked up.

P501 Dispose of contents/ container in accordance with local/ regional/ national/ international regulations.

This product was developed for dental lab use and has to be applied according to the directions for use. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. The user is generally obligated to check the product before use. Especially if the purpose is not mentioned in this directions for use.

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