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0473

EC REP

Distributed by: (US only)
HENRY SCHEIN INC.
135 DURYEA ROAD
Melville, NY 11747 USA
HENRY SCHEIN U.K. HOLDINGS LTD.
MEDCARE HOUSE, CENTURION CLOSE
GILLINGHAM BUSINESS PARK
GILLINGHAM, ME8 0SB U.K.
www.henryscheinbrand.com

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INSTRUCTIONS FOR USE
ETCH GEL 37% IS A WATER BASED,
37% PHOSPHORIC ACID GEL.

DIRECTIONS: Using the syringe with a syringe tip attached, apply the gel directly onto the tooth enamel, being sure to cover the bevels of the preparation. Leave the acid gel in place for 15 seconds. Remove the blue gel with a few drops of water and high volume suction. Rinse the etched surface thoroughly for 20 seconds with air/water spray. Dry completely with oil free, water free air. The etched enamel should have a white chalky appearance. Remove the syringe tip from the syringe and replace the original cap. For application to dentin, follow the instructions that accompany the dentin bonding agent to be used.

NOTE: On primary teeth or teeth with high fluoride content, a 90 120 second etch is recommended.

CAUTIONS: Before dispensing the etchant, be sure the needle tip is securely in place. Do not use excessive force to dispense the etchant. Tightly cap the syringe after use (this prevents clogging). The etchant gel contains phosphoric acid that can be harmful to skin, mucosa and eyes. In case of contact with any of these tis-

sues, flush immediately with copious amounts of water and seek medical assistance for appropriate treatment. Wear protective gloves when handling acid etchant. Reusable syringe needs to be wiped clean after use in a contaminated dental field. Follow standard office hygiene procedures. Acid may be corrosive to metal surfaces. If any metal surface is contacted with the acid, wash the surface immediately.

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