HENRY SCHEIN®

NATURAL ELEGANCE™

BLEACH

IMPROVED FORMULA

BLEACHING • Home Kit

---

BLANQUEAMIENTO • Kit pour patient

---

BLANCHIMENT • Patient kit

---

HS-BLEACHING • Patientenkit




---

SBIANCAMENTO • Per uso domiciliare

**9008873 Bleaching home kit 10% CP (= 3.6% HP)**

**9008874 Bleaching home kit 16% CP (= 5.7% HP)**

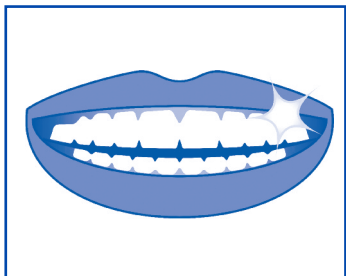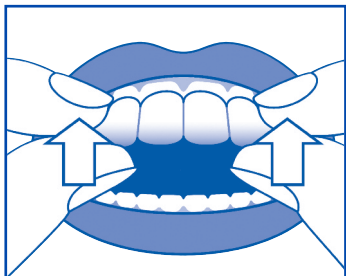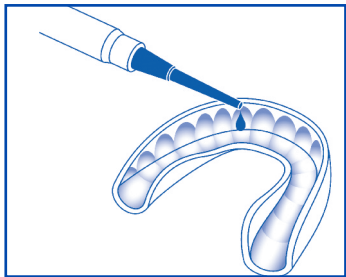## Instructions for Natural Elegance™ BLEACH Home

**Tooth Whitening System for use in customized trays.** Store in a cool, dark place. Do not freeze. Contains hydrogen peroxide. Avoid contact with eyes. Rinse immediately if product comes into contact with them. Not to be used on a person under 18 years of age. To be only sold to dental practitioners.

For each course of treatment, the first application is to be completed by the dentist or under their direct supervision. Additional applications to be completed at home by the patient. With HOME WHITENING you can whiten your teeth in a safe and effective way without causing damage.

Please be aware that crowns, bridges and fillings will not benefit from any bleaching procedure. A professional tooth cleaning is recommended before using this product. The teeth should be in good condition and without cavities. Please consult your dentist if you have questions.

Natural Elegance™ BLEACH Home Gel is available in various concentrations. The higher the carbamide per-oxide content is, the shorter the treatment time. The treatment can be done overnight or for 1–4 hours during the day. The treatment should be repeated over 5–10 days.

### Recommended treatment times

The indicated times are only recommendations. A longer application time per session is possible which can reduce the number of applications. In case of sensitivity a shorter application time per session with more applications is recommended and possibly a lower concentration of carbamide peroxide gel.

Carbamide Peroxide concentration	Minimum time of application	Recommended time of use	Best result for regular discolouration	Indications
10% (~3,6% HP)	2 hours	4 hours overnight	5–10 applications	Minor to regular discolouration, sensitive teeth
16% (~5,7% HP)	1 hour	3 hours or overnight	4–8 applications	Minor to regular discolouration, sensitive teeth

### How to achieve maximum results

- Brush and floss your teeth.
- Take one syringe out of the box and remove the cap from the dispensing tip.

- Place a small drop of Natural Elegance™ BLEACH Home Gel into each tooth compartment (Pic.1). Place the filled trays on your teeth (Pic.2). One syringe fills approximately 6–8 trays.
- After whitening, remove the trays, rinse and brush excess gel from your teeth and gums. Rinse the trays with cold water and clean gently with a toothbrush.
- Avoid tea, coffee, red wine, cigarettes etc. for at least 2 hours after bleaching as this will reduce the effectiveness of the treatment.

**Tip:** To remineralize the teeth and avoid any sensitivity, use Natural Elegance Desensitizing Gel in the cleaned bleaching tray for approximately 10–15 minutes after tooth whitening.

#### **Possible side effects**

Tooth sensitivity, gum irritation, nausea, headache. Symptoms are temporary and should subside in 1–3 days following termination of product use.

#### **Warning**

Avoid contact with the inside of the mouth, eyes and skin. If accidental contact occurs, rinse affected area generously with water. In case of contact with eyes, rinse immediately with water and seek medical advice. Henry Schein does not recommend bleaching treatments on people who are pregnant or breast feeding.

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**HENRY SCHEIN U.K. HOLDINGS LTD.**