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Save time, increase brightness: The new HATHO ThermoPol Set improves the surface of thermoplastic dentures quicker than ever before.

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**The secret
of pre-polishing**

- ★ Attach the HATHO Multi Layer Brush art. no. 4280 to your polishing lathe.
- ★ Adjust the speed to low speed, at 1,800 rpm.
- ★ To fill the multi layer brush with additional polishing agent, hold the pre-polishing compound HATHO Polistar Lintygrey ref. no. 313168 against the rotating multi layer brush for approx. 2 seconds.

- ★ Use the filled brush to evenly polish the thermoplastic denture surfaces for approx. 3 to 4 minutes until smooth.

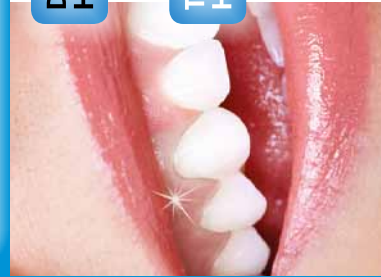
- ★ Finish step 1 as usual by cleaning the workpiece.

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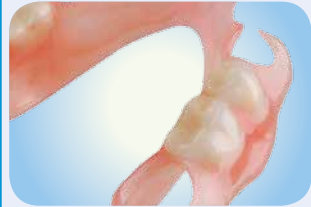
**The technique
of shine polishing**

- ★ Attach the HATHO COSIMA Soft Polishing Buff art. no. 869 100/14 to your polishing lathe.
- ★ Adjust the speed to low speed, at 1,800 rpm.
- ★ Apply approx. 0.5" of HATHO Polistar Cream ref. no. 313200 to the pre-polished thermoplastic dentures as needed.

- ★ Gently and evenly polish the denture surfaces for approx. 3 to 4 minutes until all fine marks have disappeared.

- ★ Finish step 2 as usual by cleaning the workpiece.

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Everyone benefits from the perfect result: Dental technicians, dentists and patients alike.

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**The discovery
of high shine polishing**

- ★ Attach the HATHO MIRA Polishing Buff art. no. 868 4" x 50 H to your polishing lathe.
- ★ Adjust the speed to low speed, at 1,800 rpm.

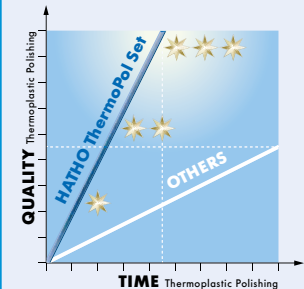
- ★ Fill the polishing buff with polishing agent by holding high shine polishing compound HATHO Polistar Pro Pink ref. no. 313196 against the rotating buff for approx. 2 seconds.

- ★ Gently and evenly polish the denture surfaces for approx. 3 to 4 minutes until they exude the high shine, brilliant appearance that up until now could only be achieved with PMMA acrylic dentures!

- ★ Finish step 3 as usual by cleaning the workpiece.

★ Enjoy the best result in polishing thermoplastic dentures achieved in the shortest time ever needed.

**Blitzschnell, blitzblank
High shine in record time**

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Outstanding advantages in polishing of thermoplastic dentures: **Maximum brightness in minimum time.**

HATHO GmbH
Freiburger Str. 18
79427 Eschbach
Deutschland / Germany
Tel. +49 7634 50 39-0
Fax +49 7634 50 39-22
info@hatho.de
www.hatho.de

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