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Indication for use

Dental thermoforming blank for the production of individual impression-trays as well as base plates for bite registrations or set-ups.

Field of application

Drufoplast is especially suitable for the manufacture of individual trays as well as the basis for bite registrations or set-ups.

Characteristics

Ø 120 mm / 125 mm
thickness 2.0 mm / 3.0 mm

Colours

Nature / pink

Values of the material

Density:	1.18 g / cm ³
Bending strength:	39.00 N / mm ²
Tear strength:	> 30 %
Water absorption:	< 0.1 %
Softening temperature:	> 70 °C

Processing

Place the plate into a vacuum or pressure unit (Vacfomat / Drufoformat). Before thermoforming the plaster model should be watered and undercut areas should be blocked out with a heat resistant material (e. g. Fillin). Depending on the unit and material thickness the heating times are 1:05 to 1:50 min. With the help of vacuum or pressure the plastified plate is formed onto the model. The cooling time under pressure or vacuum should be at least as long as the heating time.

Drufoplast can be treated with hard metal cutters. In order to smooth the cuts especially flexible grinding material can be used (UltraTrimm). For the manufacture of individual impression trays a handle of the same material is available. These handles can be adapted to the tray with heat and compounded with a chemical bonding (TrayFix).

Information on health:

Drufoplast is marked with a CE label and thus fulfills the general requirements of the medical device guideline.

Keep away from children! Only for dental use!

Storage conditions

At room temperature, i. e. 18 °C–28 °C.

Durability: see date of expiry.

Drufoplast is not to be used when expiry date has passed.

Form of delivery

REF **D4210Z**

Drufoplast pink

120 x 2 mm / 15 pieces

REF **D4214**

Drufoplast nature

120 x 3 mm / 10 pieces
with sprayed tray

REF **D4215**

Drufoplast nature

120 x 3 mm / 100 pieces
with sprayed tray

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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