Applicabile per le Maquine Rivelatori:
DÜRR (AC 245, XR-24, XR 24II, XR 24PRO, XR25, XR24easy)
PHILIPS (Clarimat Super S2, S3)
GIRARDELLI (X-20, X-24)
VELOPEX (DE260, Freedom X-Ray, Velopex extra X)
GENDEX (GX 300)

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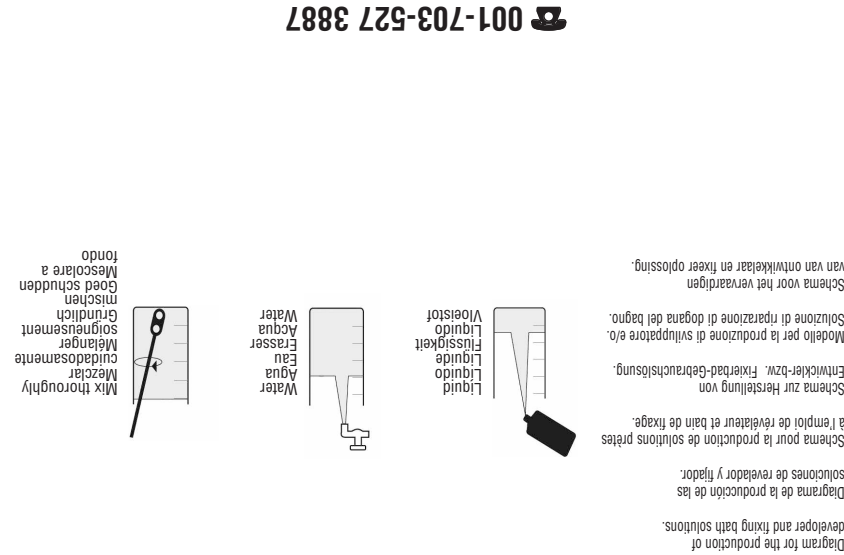
900-2685

AUTOMAT SET radioMAT-XR

Special X-Ray Set for Automatic Film Processors

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Henry Schein U.K. Holdings Ltd.
Gillingham ME8 0SB U.K.

Distribuido por:
Vertrieb durch:

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Rinse empty concentrate bottles with water twice and supply to the Dual System.

Recycling instructions:

Storage: The concentrates must not be stored below +5°C (41°F).

handling the concentrate.

CAUTION: Irritant. Irritating to eyes and skin. In the event of contact with eyes, rinse them thoroughly with water and consult a doctor. Please wear protective gloves when

as the slightest contamination will render the solution unusable.

should be filled carefully into the corresponding tanks. Please keep strictly to this order, Please take care that, after mixing, at first fixing and afterwards developing solution. Before mixing the solutions, the cans must always be rinsed thoroughly with water. exchanged under any circumstances. Even slight contamination will impair the results.

Please take care that cans and caps of the developing and fixing solutions should not be

Important Notice:

sets for cleaning the processors

Processors has to be thoroughly cleaned every 3 months. Please use current cleaning

Cleaning of the machine:

baths have to be renewed every 3 weeks.

often. In such cases, an automatic regenerator is highly recommended. Using this, the a day. With increased output (specialized practice), regeneration will be required more and fixer baths. These regeneration quantities apply to the output of 7 panorama films week 0.5 litre of the remaining solution from the corresponding cans to the developer (approximately 2.5 litres) with regular usage by manual regeneration. Add twice a week 0.5 litre of the remaining solution from the corresponding cans to the developer. Approximately 7 dental panoramic films can be processed a day in one machine

2.2 Manual Regeneration:

which may lead to premature oxidation of the solution.

the regeneration containers completely, otherwise air will be pumped into the baths, cannot be completely balanced due to the fewer regeneration dosages. Never empty months. Moreover with very low film throughput, as the oxidation of the developer For machines with automatic regeneration, the baths must be renewed every 3

2.1 Automatic Regeneration:

2. Regeneration and utilization of the baths:

1. Preparation of the baths — see illustrations.

Preparation instructions for automatic film processors:

GENDEX (GX 300)

VELOPEX (DE260, Freedom X-Ray, Velopex extra X)

GIRARDELLI (X-20, X-24)

PHILIPS (Clarimat Super S2, S3)

DÜRR(AC 245, XR-24, XR 24II, XR 24PRO, XR25, XR24easy)

Applicable for the following Film processors:

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