# STOPPI®

D

GB

F

I

E

## Patienteninformation

## STOPPI® zur Prophylaxe

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
**Patient information**

## STOPPI® for prophylaxis

**STOPPI®** is used similarly to a normal pacifier. However, the silicone shield is placed between teeth and closed lips. Two small lateral bite planes for the molars provide the necessary support. **STOPPI®** is a healthy substitute to the damaging pacifier, thumb-sucking or baby bottles. **STOPPI®** is offered to children as substitute, which they voluntarily accept and wear because they are not forced to.

lateral bite planes

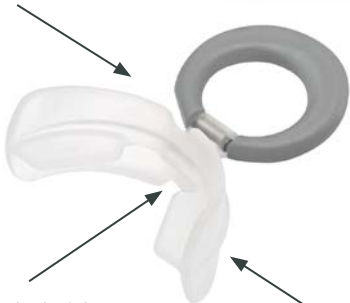no foreign body between  
the incisors

the shield rests in front  
of the teeth

## STOPPI®-rules

Your support and a few rules help your child:

- 1 At the age of two, **STOPPI®** is the only thing that should be worn in the mouth for extended periods. From this age, pacifiers, baby bottle and thumbs should be taboo.
- 2 **STOPPI®** is a substitute for thumb and dummy. The substitution must be voluntary.
- 3 **STOPPI®** should be used as a substitute for a pacifier, thumb or baby bottle and should only be worn at customary sucking times.



- 
- 4 As a rule, weaning is successful within a few weeks. Then the old detrimental sucking habits are forgotten.
  - 5 **STOPPI®** can be stored in the hygiene box
  - 6 Do not forget to brush teeth and **STOPPI®** regularly.
- 

## How is STOPPI® worn?

**STOPPI®** is used like a regular pacifier. But: it must be placed in front of the teeth – that means: between lips and teeth! Only the molars bite on the shield and provide the needed support!

- 1 Initially push one side of **STOPPI®** in front of the teeth,
- 2 then slightly press it and
- 3 now place the other side of **STOPPI®** between teeth and lips.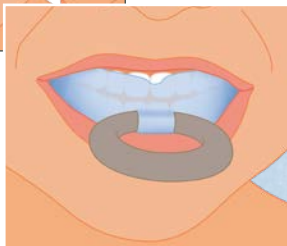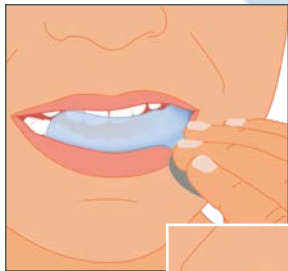

## CAUTION! For your child's safety:

Only use special ribbons which comply with EN 12586. Never attach **STOPPI®**, to other ribbons. Your child could strangle itself with them.

- Cracks, bite marks, stickiness or form changes could lead to detaching and swallowing of parts (risk of suffocation).
- Replace **STOPPI®** after one or two month of usage for reasons of hygiene and safety.

The **STOPPI®**-Box is not a toy. It needs to be used only under adult supervision. **STOPPI®** is not a normal pacifier or soother! It is not suitable for children under the age of 24 months. Risk of suffocation! Only use **STOPPI®** under dental supervision. Do not use **STOPPI®** longer than recommended by the dentist. Please keep safe the instructions for use. In case of complaint: Keep LOT and AN

## Instructions for use

Before using **STOPPI®** for the first time, clean it thoroughly under warm running water. Check **STOPPI®** regularly for places that have been bitten through! Replace **STOPPI®** when necessary. **STOPPI®** never dip in sweet food or medicine (causation of caries and/or material damages)!

## Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, you can also use the OROFAN® dental cleaning tablets. **STOPPI®** is neither dishwasher-safe nor microwave-safe. Only clean the hygienic **STOPPI®**-Box with a damp cloth.

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