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Soft denture relining resin

## Extended intended use

Silicone lacquer for coating of dental soft relining material.

## Indication for use

Coating of permanently soft relining materials (e. g. MegaBase®) based on A-silicone.

## Contraindication

The product may not be used if the patient is allergic to one of the ingredients. There are no side-effects to be expected from this medical device if processed and used properly. However, immune reactions (e.g. allergies) or local disorders of sensation (e.g. irritation of the oral mucosa) cannot be ruled out in principle. If you become aware of adverse side-effects, also in cases of doubt, please be sure to advise us in any case and describe the circumstances and symptoms as accurately as possible. We will investigate any notice.

## Target Patient Group

Patients who need dental care.

## Intended users

Dentist, dental technician

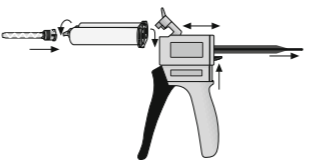
## Product description

MegaBase® 2K Lack forms a thin, highly transparent and scratch-resistant surface. It is also absolutely free of solvents and thus completely odourless.

## Processing

- Prepare the silicone and smoothen the surface (e. g. with UltraTrim medium/light).
- Degrease the surface and let it dry in air for at least 3 min.
- MegaBase® 2K Lack is provided in convenient double cartridges, the mixing tips included in the scope of delivery allow an optimum dosage and mixture of the 2-component material.

- Apply the required quantity either directly onto the surface to be coated or fill the material into a clean vessel. Then spread it thinly and evenly over the silicone with a brush. Possible spillage can be easily removed with a brush. The processing time is 2:15 minutes at room temperature (23 °C; 73 °F).
- The curing will begin independently once the two components have been mixed. The curing time is 10 minutes at room temperature.
- After curing the covered surface has to be brought in a water quench with a temp. of 95–104 °F for final polymerisation and best possible bonding towards the base silicone for 10 min.

## Disinfection

Disinfection of the finished prosthesis with soft relining can be done with the usual disinfectants for dental silicone found in dental practices and dental laboratories.

## Cautions

- The product is exclusively intended for the above-mentioned range of use and shall only be used by professionally qualified and instructed persons.
- Due to technical filling reasons there might be an unequal transportation of the two components of new cartridges. For this reason it is recommended to dispose the initially mixed pea-sized amount.
- Pre-convey the material before attaching the mixing cannula.

• Polymerisation, referring to the hardening of addition curing silicones, may be inhibited by latex gloves, creams, plastics, cleaning agents, etc. For this reason, we recommend that you wear commercially available polyethylene gloves.

- Wear protective clothing! Set material is chemically resistant. Marks on your clothing cannot be removed.
- After use, leave the mixing cannula on the double cartridge as a cap.
- Never dispense into a mixing cannula which has hardened.

## Keep away from children!

## Note for one-time use

Due to the physical characteristics, the cured silicone lacquer cannot be used for further coating.

## Disposal

Disposal of contents / container in accordance with local / regional / national / international regulations and safety data sheet.

## Hazard identification

Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

## Serious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

Le MegaBase® 2K Lack forms a thin, highly transparent and scratch-resistant surface. It is also absolutely free of solvents and thus completely odourless.

- Prepare the silicone and smoothen the surface (e. g. with UltraTrim medium/light).
- Degrease the surface and let it dry in air for at least 3 min.
- MegaBase® 2K Lack is provided in convenient double cartridges, the mixing tips included in the scope of delivery allow an optimum dosage and mixture of the 2-component material.

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## Batch number / use-by date

The batch number and the use-by date are located on both the outer packaging and the product. When submitting a complaint about a product, please always quote the batch number. Do not use the product after expiration of the use-by date.

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and

possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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