
# MTA BIO-CERAMIC ROOT CANAL SEALER

- ES Cemento obturador endodóntico biocerámico
- FR Ciment d'obturation endodontique biocéramique
- DE HS-Biokeramisch endodontischer Füllzement
- IT Cemento endodontico bioceramico per otturazioni
- NL Bio keramisch wortelkanaal afdichtmiddel
- PT Cimento obturador endodóntico biocerámico
- CS Biokeramická impregnace kořenového kanálku
- PL Bioceramiczny wypełniacz endodontyczny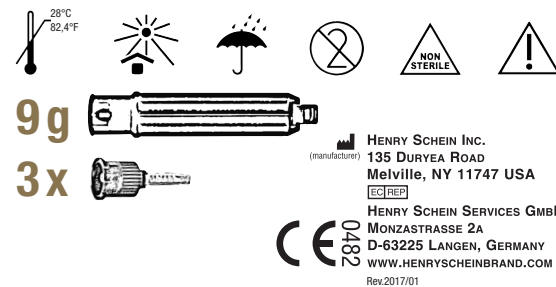

**ATTENTION:** This product must be used according to the instructions described in this manual. The manufacturer is not responsible for failure or damage caused by incorrect handling or use.

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## BIOCERAMIC ROOT CANAL SEALER (EN)

### INTRODUCTION:

Henry Schein's MTA is a bioceramic root canal sealer based on MTA (mineral trioxide aggregate). Its composition provides the following qualities:

- Indicated for the permanent sealing of root canals;
- High radiopacity;
- Low expansion during setting;
- Low solubility in contact with tissue fluids;
- Optimal flow characteristics for insertion;
- Does not stain the tooth;
- Eugenol-free material: does not interfere with the setting of resin cements

CAUTION: Federal law restricts this device to sale by or on the order of a dentist.

### COMPOSITION:

- **Base Paste:** Salicylate Resin, Natural Resin, Calcium Tungstate, Nanoparticulated Silica, Pigments;
- **Catalyst Paste:** Diluting Resin, Mineral Trioxide Aggregate, Nanoparticulated Silica, Pigments.

### INDICATION:

Henry Schein's MTA is a root canal sealer intended for the permanent sealing of root canals and may be used in combination with root canal obturation materials.

### CONTRAINDICATION:

Hypersensitivity to the resin or other components.

### WARNINGS:

- Henry Schein's MTA contains resins which may cause sensitivity in susceptible individuals. Do not use on patients allergic to the resins or other components of the product;
- Avoid contact with eyes or skin. In case of contact, rinse immediately with water;
- Avoid contact with oral mucosa. In case of contact, rinse with water and prevent swallowing of product. In case any sensitivity persists, seek medical attention promptly;
- If the syringe becomes contaminated with saliva or blood during application, dispose of the syringe and do not use on an additional patient.
- Do not mix up the base and catalyst lids as this will cause hardening of the product inside the tube.

### PRECAUTIONS:

Both the patient and professional should use suitable protective clothing, eye protection and gloves when using this product.

### STORAGE:

- Keep in dry and cool place, away from humidity and sources of heat.
- Do not Refrigerate.
- Keep the product tightly capped.

### TECHNIQUE OF USE:

#### 1. Root canal preparation:

Prior to insertion of Henry Schein's MTA, the root canal should be prepared and cleaned according to the selected endodontic technique. The device requires an isolated dry field; therefore, any moisture found in the canal or in the ambient humidity will be sufficient to initiate Henry Schein's MTA hardening;

#### 2. Mixing:

- Dual syringe: The dual syringe ensures equal mixing in a 1:1 ratio. Press plunger to extrude material directly onto a glass slab, mixing pad or into the tooth canal. The cement should be used immediately after mixing;
- Tubes: Mix equal volume units (1:1). Mix for 30 seconds to a homogeneous consistency;

**3. Insertion:** Use Henry Schein's MTA with gutta-percha or silver points, coating them with a thin layer of the cement and filling the root canal according to the selected technique. Henry Schein's MTA can be applied in the root canal with a spiral paste filler or a suitable applicator tip that fits directly on the mixing tip.

**WARNING:** Discard mixing tip after use.

#### 4. Removal of the root canal filling:

Use the conventional techniques for the removal of gutta-percha fillings;

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