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Intended use

Dental impression material on the basis of vinyl polysiloxane

Indication for use

Dental impression material for the reproduction of the patient's oral situation.

Dynax® dental impression material is intended to be placed on a preformed impression tray and to reproduce the structure of a patients teeth and gums.

Contraindication

The product may not be used if the patient is allergic to one of the ingredients. There are no side-effects to be expected from this medical device if processed and used properly. However, immune reactions (e.g. allergies) or local disorders of sensation (e.g., inflammations of the oral mucosa) cannot be ruled out in principle. If you become aware of adverse side-effects, also in cases of doubt, please be sure to advise us in any case and describe the circumstances and symptoms as accurately as possible. We will investigate any notice.

Target Patient Group

Patients who need dental care.

Intended users

Dentist

Product description

Properties Dynax® putty, Dynax® heavy body, Dynax® mono, Dynax® clear and Dynax® light:

- scannable surface
- hydrophilic surface characteristics
- high tensile strength
- highest precision and dimensional stability
- high elastic recovery
- long processing times
- short intraoral curing

Dynax® putty is a kneadable, pressure-forming pre-impression material especially for the correction and double mixing technology.

Dynax® heavy body is a thixotropic, pressure-forming precision pre-impression material especially for the sandwich, correction and double mixing technology.

Dynax® light is a light bodied correction material with high detail precision for the sandwich, correction and double mixing technology. Combinable with Dynax® putty, Dynax® heavy body, Dynax® mono and Dynax® clear.

Dynax® mono is a medium bodied precision impression material for the monophase impression.

The cured impression can be disinfected with disinfectants for dental silicones usually used in dental practices and dental laboratories.

Dynax® mono is a medium bodied precision impression material for the monophase impression.

Dynax® clear is a clear transparent, medium bodied precision impression material for the monophase impression, especially for the implant impression.

Impression tray: Preparation and adhesives

All standard (in metal or hard plastics) and / or individual impression trays are applicable corresponding to the impression technique. To improve the adhesion of the impression material to the metal or plastic trays, a tray adhesive for A-silicones should be applied.

Processing 50 ml double cartridge

The impression material is automatically dosed out of the double cartridge in a mixing ratio of 1:1. Put the double cartridge into the injector. Remove the cap of the cartridge and dispense a bit of the material until both components flow equally. Now, attach the mixing canula onto the double cartridge and start dispensing the material into the prepared impression tray.

Processing tubs

Mixing: Take both components A and B out of the tubs in the ratio 1: 1 by using the supplied trays and mix both components 45 sec. by hand until a homogeneous color arises. The A and B components of a packaging must only be used in this combination. Tubs should be closed well after use.

Impression taking

The area where the impression is taken has to be cleaned and dried carefully. Choose impression technique according to indication.

Position the prepared tray inside the mouth within the working time, applying slight pressure. Fix the impression without pressure until the material has set completely.

Please find processing times in „technical data“ overview.

Cleaning / disinfecting

The cured impression can be disinfected with disinfectants for dental silicones usually used in dental practices and dental laboratories.

Casting

After disinfecting dry the impression and store it at normal room temperature. The impression should not be cast any sooner than 30 min. after removal from the mouth. Using special purpose hard plaster (ISO 6873, type 3) respectively an ultra hard plaster for the form (ISO 6873, type 4) is recommended.

Caution

- Dynax® products are not to be used in combination with condensation-vulcanising impression materials.
- The product is exclusively intended for the above-mentioned range of use and shall only be used by professionally qualified and instructed persons.

• Polymerisation, referring to the setting of cross-linked silicones, can be disrupted by latex gloves, creams, cleaning agents etc.. For this reason, we recommend that you wear commercially available polyethylene gloves.

• The setting times can be increased by lower temperatures or reduced by higher temperatures.

• Residue of retraction aids have to be removed by use of water spray.

• Wear protective clothing! Set material is chemically resistant. Marks on your clothing cannot be removed.

• Pre-convoy the material before attaching the mixing canula.

• Never convey to a mixing canula which has been cured.

• After use, leave the mixing canula on the double cartridge as a closure.

Keep away from children!

Note for one-time use

After impression taking, the cured silicone cannot be used for a further impression due to its physical physical features.

Disposal

Disposal of contents / container in accordance with local / regional / international regulations and safety data sheet.

Hazard identification

Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

Serious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

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