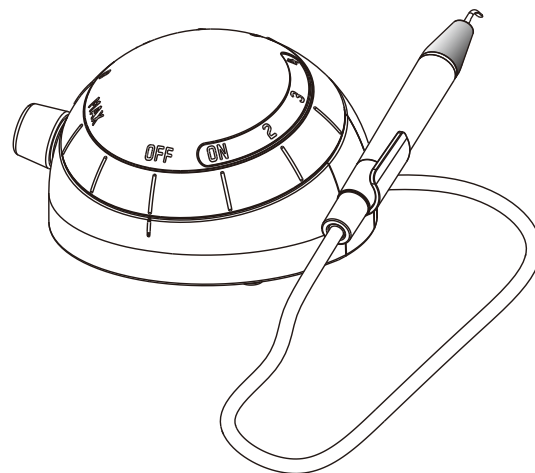
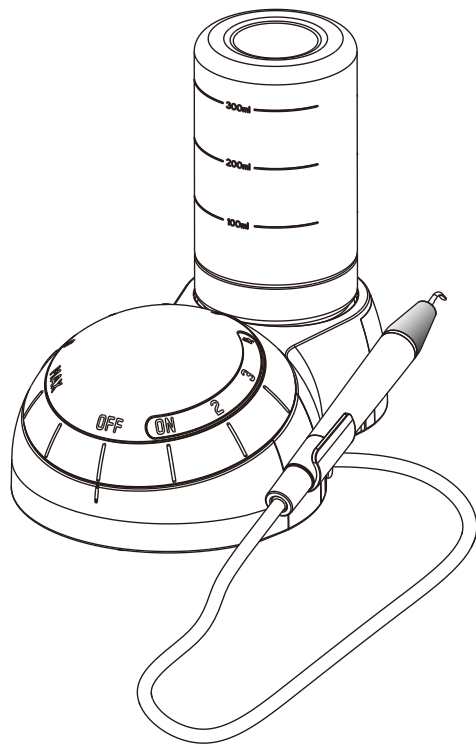
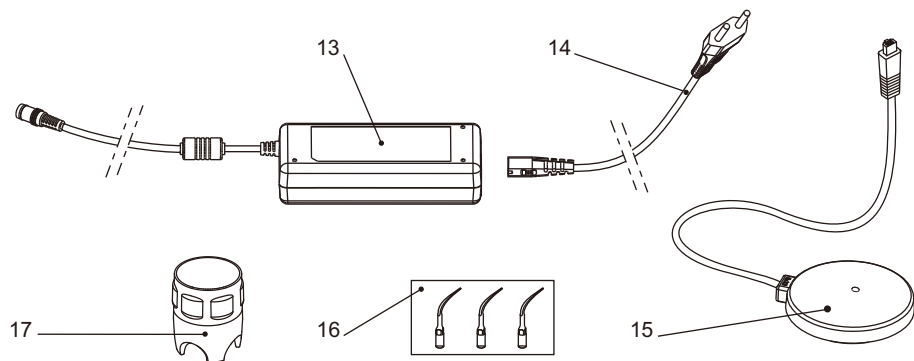
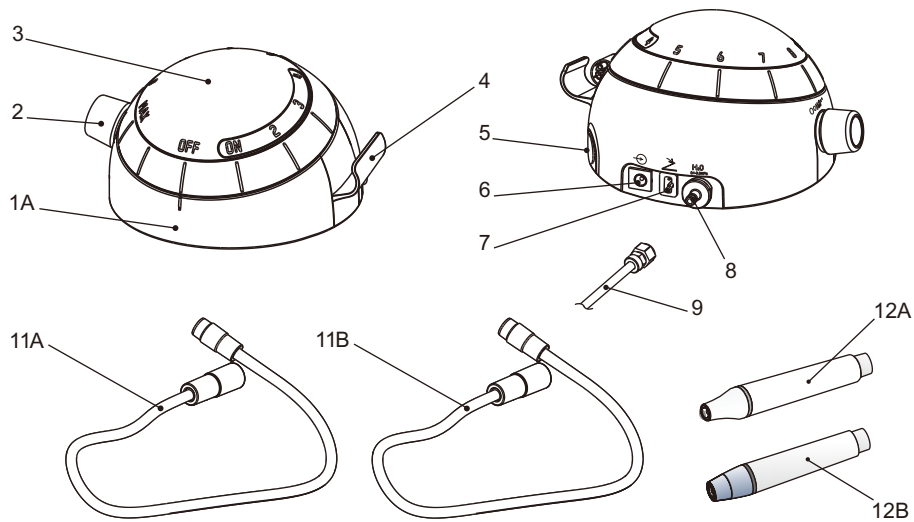
OPERATION INSTRUCTIONS
MODE D'EMPLOI
BEDIENUNGSANLEITUNG
INSTRUCCIONES DE EMPLEO
ISTRUZIONI PER L'USO
BEDIENINGSINSTRUCTIES

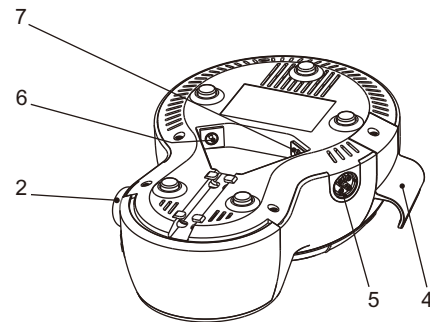
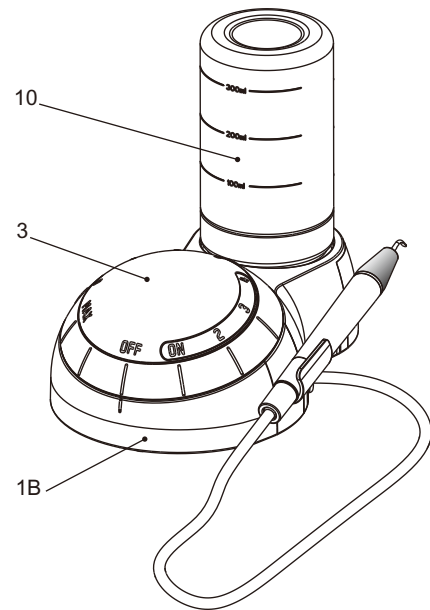
OPTIMA BAC100 OPTIMA BAC200

OPTIMA BAC100

OPTIMA BAC200

COMPONENTS

- 1 Unit
- 1A OPTIMA BAC100
- 1B OPTIMA BAC200
- 2 Water flow rate adjustment
- 3 Power dial
- 4 Handpiece holder
- 5 Handpiece connection
- 6 External power supply connection
- 7 Foot control connection
- 8 Water hose connection
- 9 Water supply hose
- 10 Bottle
- 11 Handpiece cord
- 11A Handpiece cord BAC100E&BAC200E
- 11B Handpiece cord BAC100EL&BAC200EL
- 12 Handpiece
- 12A Handpiece BAC100E&BAC200E
- 12B Handpiece BAC100EL&BAC200EL
- 13 Power supply
- 14 Power cord
- 15 Foot pedal
- 16 Scaling tips
- 17 Torque wrench

BA International supplies units with various accessories. The "Packing list" shows exactly what is included with your unit.

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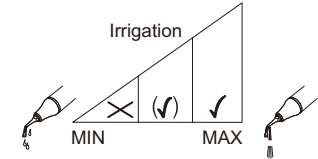
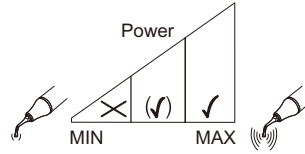
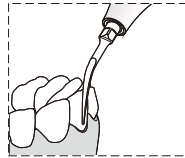
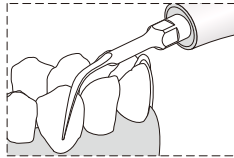
BAC261E

Instrument BAC261E for universal scaling

Removal of supragingival deposits in all quadrants.

Use a high power setting for hard concretions and for initial treatment, or a medium power setting for pain-sensitive patients. Use a high to medium irrigation flow rate.

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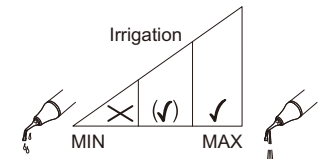
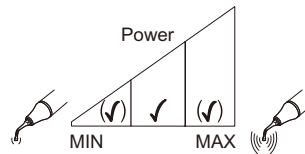
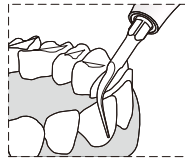
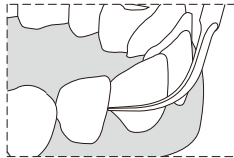
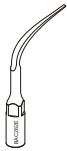
BAC262E

Instrument perio BAC262E

Removal of supra and subgingival deposits in all quadrants, including the interproximal and sulcus areas.

Use a medium power setting, or low power setting for pain-sensitive patients and for supragingival maintenance-type treatments. Use a high power setting for hard concretions. Use a high to medium irrigation flow rate.

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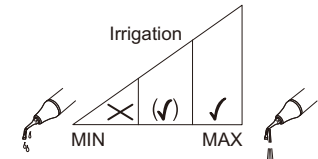
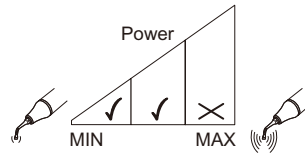
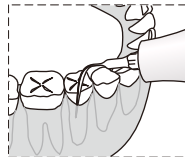
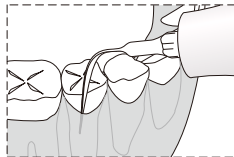
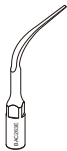
BAC263E

Instrument perio slim BAC263E

Fine removal of subgingival deposits on root surfaces and irrigation of periodontal pockets with an antimicrobial solution. May also be used for supragingival maintenance-type treatments. Use a low power setting or medium power for hard concretions. Do not exceed a medium power setting. Use a high to medium irrigation flow rate.

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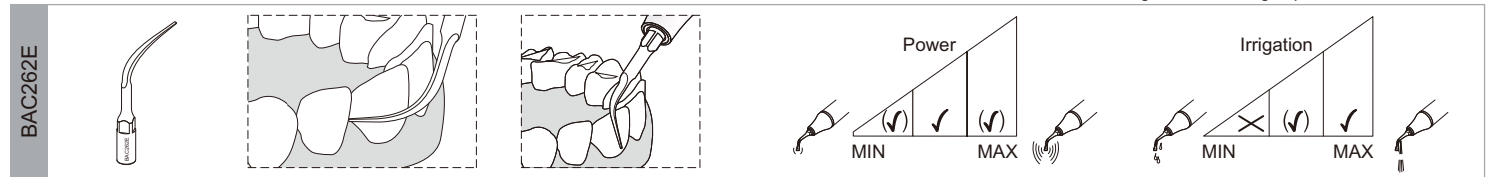
DEAR CUSTOMERS,

Thank you for buying a new BA International product. This device meets the highest quality and safety standards.

In order to get the best out of this high-end dental product and ensure the longest working life possible, please read this manual carefully before using or maintaining the product.

Qualified use

This product is only intended for qualified use in dental medicine according to the applicable health and safety regulations, accident prevention measures and in observance of these instructions for use. The product must only be prepared and maintained by persons who have been instructed in infection control, personal protection and patient safety. Improper use (e.g. due to lack of hygiene or maintenance), non-compliance with our instructions or the use of accessories and spare parts not approved by BA International invalidates all claims under warranty and any other claims.

Qualifications of the user

The product has been developed and designed for the target group "dentists, dental hygienists, dental employees (prophylaxis) and dental assistants".

 For further details on the various instruments and their scopes of application, please refer to the operation instructions supplied with your BA International system or available from the distributor in your country.

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ABOUT THIS MANUAL

The illustrations in these instructions represent the OPTIMA BAC100 (unless otherwise specified); however, these instructions also apply to the OPTIMA BAC200.

These operating instructions are to ensure the correct installation and use of this product. Always keep this document close at hand.

Please read these operating instructions carefully as they explain the most important details and procedures. Please pay special attention to the safety precautions.

Misuse can damage the product and hence cause risks and hazards for patients, users and third parties.

Instructions concerning the components of the system are provided separately in the relative instructions for use. Read the instruction manuals originally delivered with each of the components before using this product. Do not hesitate to contact your local BA International dealer for further information.

This product needs special precautions regarding electromagnetic compatibility and needs to be installed and put into service according to the "Electromagnetic compatibility" manual.

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To prevent possible patient or user injury or damage to the OPTIMA BAC100 and BAC200, please note the corresponding directives:

-  Danger
Risk of patient or user injury
-  Caution
Risk of damage to the device or environmental harm
-  Please note
Useful additional information and hints
-  Prohibited
-  Authorized

Note that the English version of this manual is the master from which the translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call your approved BA International repair center or your dealer directly.

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BA International

CONTENTS

Treatment	2
Indications	2
Contraindication	2
Installation and connection	4
Connecting the unit	4
Electrical safety precautions	8
Preparing for treatment	8
Mode and instruments	8
Connecting the handpiece	10
Attaching the instruments	12
Installing the bottle	14
Solutions compatible with the circuit	16
Operating the unit	18
Setting the ultrasound power	20
"Perio" mode	22
"Scaling" mode	24
Setting the liquid flow rate	26
PIEZO handpiece LED (Option)	28
End of treatment	30
Circuit washing	32
Handpiece disassembly	34
Cleaning and sterilizing	36
Cleaning	36
Packaging for sterilization	38
Sterilization	38
Cleaning the bottle and the unit	40
Disinfecting the circuit	44
Maintenance	44
Verification of O-rings	46
Replacing the pump	48
Safety precautions	50
Storing the product	54
Disposal	56
Warranty	56
Accessories	56
BA International Service	58
Symbols	60
Technical data	66
Troubleshooting	69

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TREATMENT

Indications

This product is designed for use in dentistry with BA International ultrasound instruments for the following indications:

Scaling

- Removal of supragingival calculus
- Removal of stains

Endo

- Preparation, cleaning and irrigation of root canals
- Retrograde preparation of root canals
- Condensing gutta-percha
- Removal of crowns, bridges and restorations

Restorative

- Preparation of cavities
- Cementation of restorations
- Condensing of amalgams

Periodontics

- Scaling and root planing
- Periodontal therapy

Contraindication

Ultrasonic oscillations may prevent cardiac pacemakers and defibrillators from functioning properly. Therefore, we recommend that patients with a cardiac pacemaker or a defibrillator should not be treated with this product.

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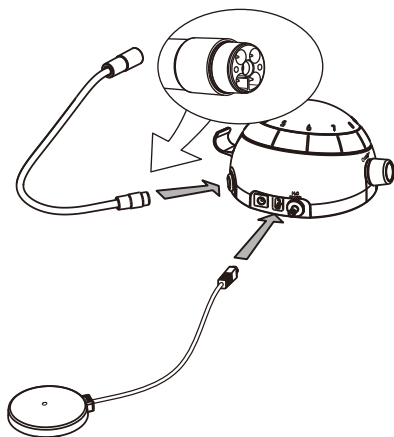
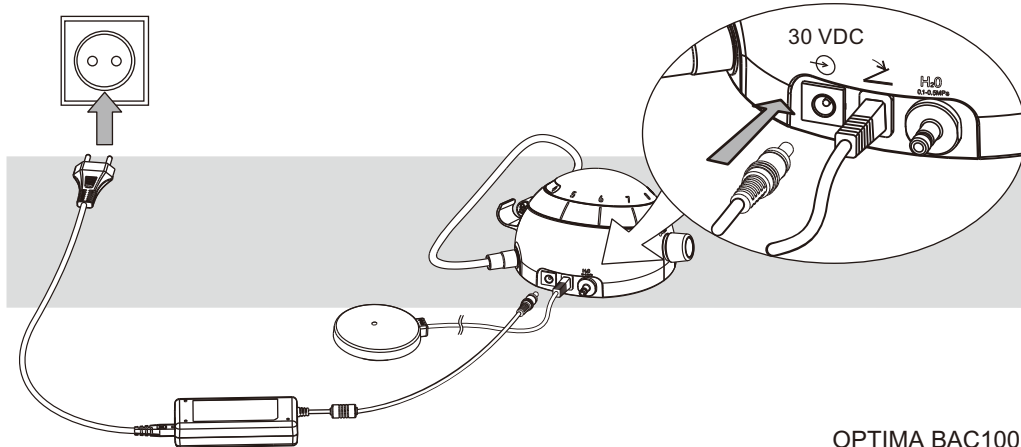
Brancher l'appareil

OPTIMA BAC100

 Always check that the unit is installed in a safe and stable position. A possible fall may suddenly pull on the cord and the handpiece, hurting the patient and the user, and damaging the unit.

To connect the foot pedal and the power supply to the OPTIMA BAC100.

Connect the unit to a water supply with the original BA International water hose supplied with your product.

 It is recommended to use an independent water supply.

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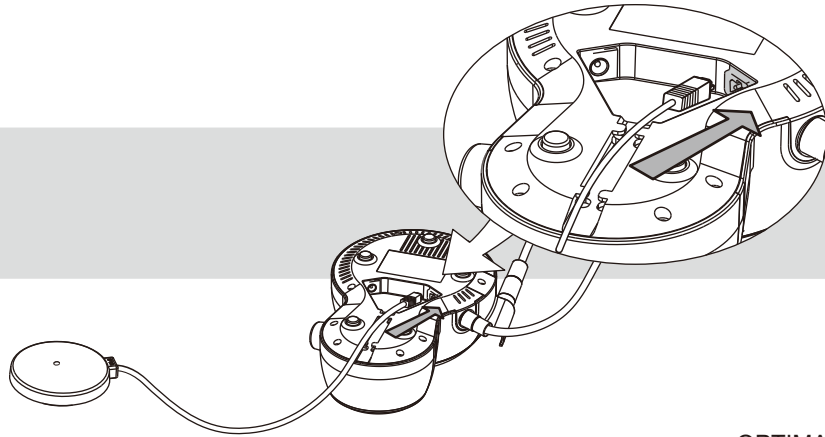
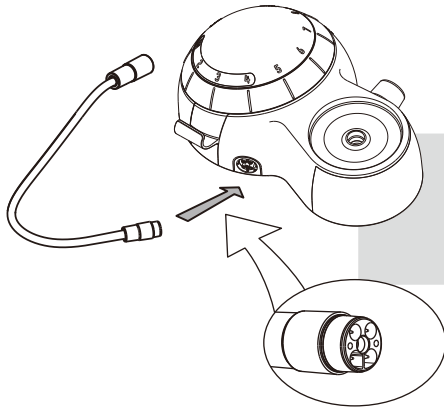
OPTIMA BAC200

 Check that the rated voltage of the power supply corresponds to the local line voltage to prevent damaging the unit.

Use the unit only in a FI protected mains supply (FI = Residual current protection).

 For USA or Canada: connect only to a Hospital Grade outlet.

To connect the foot pedal and the power supply to the OPTIMA BAC200, turn over the unit as shown in the pictures above.

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Electrical safety precautions

 No modification of the unit and the accessories supplied with the unit are allowed.

 Do not touch the connector of the handpiece cord and the patient simultaneously.

 The power supply must not be fixed tightly by screw.

The power supply can be placed on a table or hung on the wall. So it is accessible and can easily be detached.

 To disconnect the unit from the supply network, unplug the power supply.

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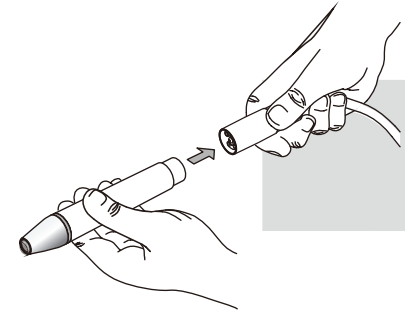
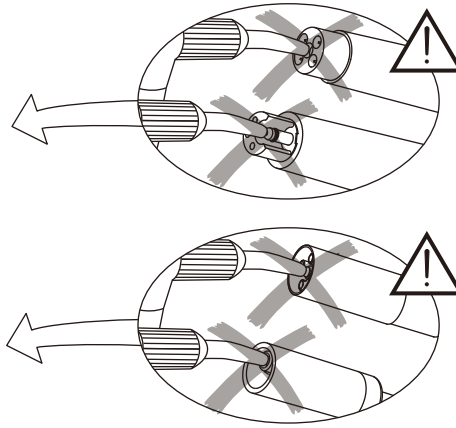
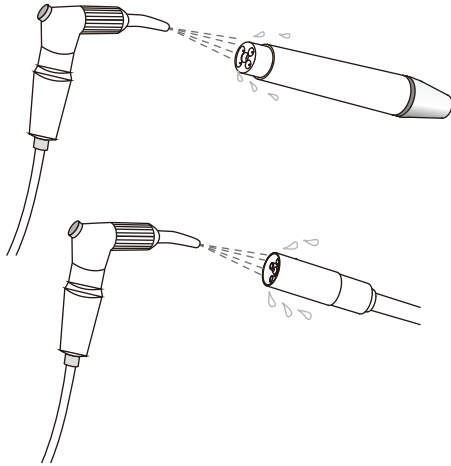
Connecting the handpiece

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Anschliessen des Handstücks

 Clean, disinfect and sterilize the different pieces and accessories of this product before each use. Please refer to the "Cleaning, disinfecting and sterilizing" chapter. Non-sterile pieces and accessories may cause bacterial or viral infections.

 Blow dry the connections to remove the possible presence of liquid to ensure a proper electrical contact. Never blow compressed air in the irrigation connections as this will irretrievably damage internal parts.

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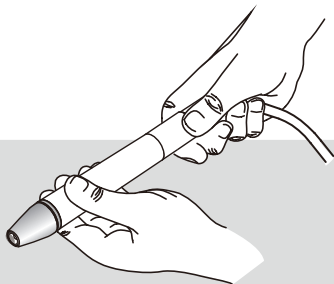
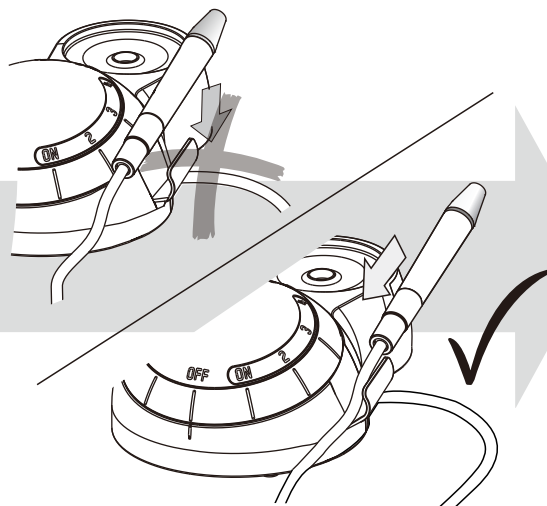
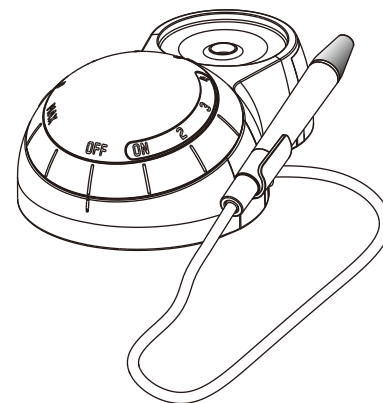
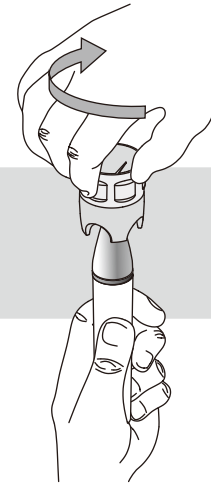
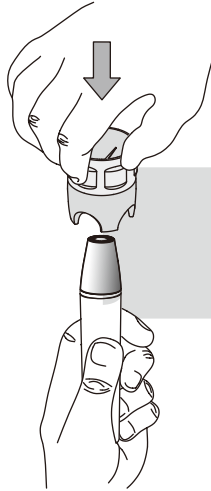
Fixation des instruments

Befestigen der Instrumente

 Use only the Torque wrench to tighten the BA International instrument on the handpiece to the correct torque. Once the instrument is screwed all the way in, give an extra quarter of a turn to obtain the required torque.

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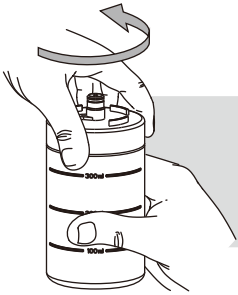
Installation de la bouteille

Installieren der Flasche

 Always check the condition of the bottle cap O-ring before installing the bottle. Please refer to the "Maintenance" chapter.

 Use the graduations on the bottle to adjust the quantity of solution necessary for the treatment.

Install the bottle using a rotating movement until it comes to a stop.

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Solutions compatible with the circuit

Please read the instructions for use of the disinfecting products.

Please refer to standards applicable in your country regarding the purchase and use of these products.

Disinfecting solutions for the treatment

- Chlorhexidine ($\leq 0.3\%$)
- Physiological water ($\text{NaCl} \leq 0.9\%$)
- Hydrogen peroxide ($\text{H}_2\text{O}_2 \leq 3\%$)
- Carbanilides
- Quaternary ammonium compounds
- Citric acid
- Aromatic essence
- Ethanol

Examples of common products

BacterX[®] pro / Triclosan / Listerine[®] / Alodont[®] / Meridol[®] / BreathRx[®] / CloSYS[™] / Scope[®] / Cepacol[®] /.....

Disinfecting solutions for washing the circuit

- Ultra Clean ($\text{H}_2\text{O}_2 6\%$)
- Hydrogen peroxide ($\text{H}_2\text{O}_2 5\%$)
- Sterilox (hypochlorous acid)

Incompatible mixing

Different disinfecting solutions can be used during treatment. Chemical reactions might occur between these solutions and damage the unit. Wash the liquid circuit for 20 seconds with distilled (or demineralized) water before using a different disinfecting solution (please refer to the "End of treatment" chapter).

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Handpiece disassembly

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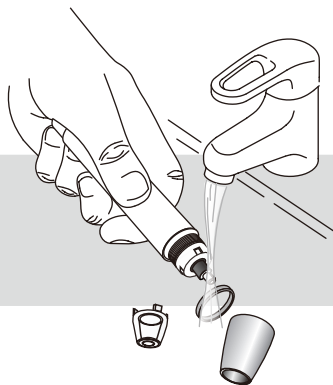
 Unscrew the nozzle and plug out the LED module (when equipped) under running water before cleaning.

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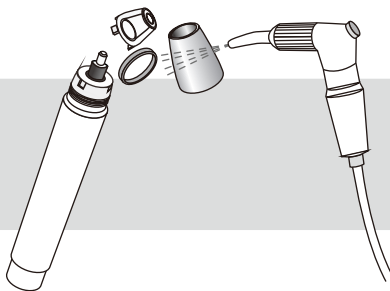
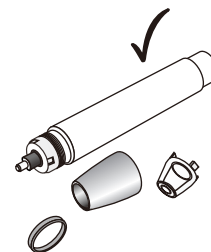
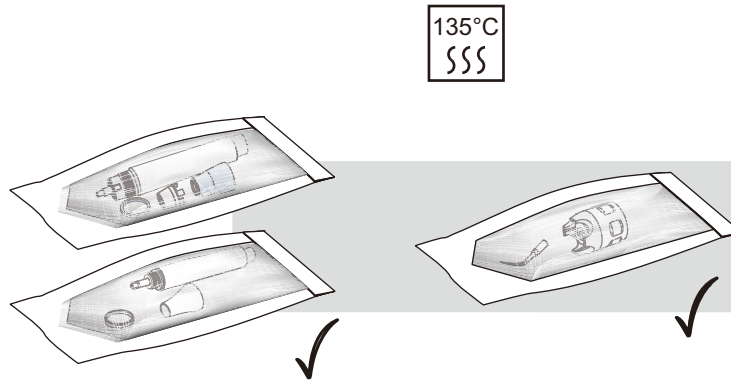
Recommended cleaning, packaging for sterilization, and sterilization procedures are provided below.

Cleaning

- Remove any externally attached soiling by brushing carefully with a soft surface brush or cloth.
- Remove the instruments from handpiece.
- Soak all parts in the cleaning solution for, at a minimum, the time and the concentration specified by the manufacturer of the cleaning agent.

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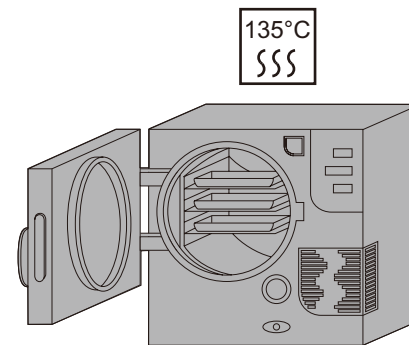
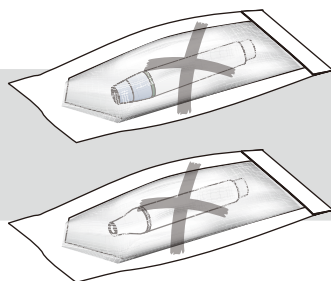
- Brush the parts in the cleaning solution if any visible soiling is detected.
- Submerge all components completely in deionized water.
- Rinse through all product tubes with deionized water.
- Repeat the cleaning process if the last rinse does not run clear, or if stains are still visible on the product.
- Dry all components at room temperature.

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Packaging for sterilization

The clean, dry instruments should be placed in a sterilization container. The container should be bagged in a sterilization pouch that is validated for use in a 135°C steam sterilization cycle. The sterilization pouch should be sealed according to the manufacturer's instructions.

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Sterilization

BA international recommends sterilization in a fractional cycle at 132°C for at least 3 min. BA International recommends use of biological indicators and chemical indicators that have been validated for use in a 135°C steam sterilization cycle.

 The hot air sterilization and radio-sterilization procedure may not be used (as it causes the destruction of products).

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 Bottles can be washed in a washer and disinfected at up to 95 °C in a washer disinfectant. Bottles cannot be sterilized.

 Clean the unit only with an alcohol-based, commercially-available, colorless disinfectant (ethanol, isopropanol). The use of scouring powder or an abrasive sponge will damage its surface.

 The unit is not protected against water spray. It cannot be sterilized.

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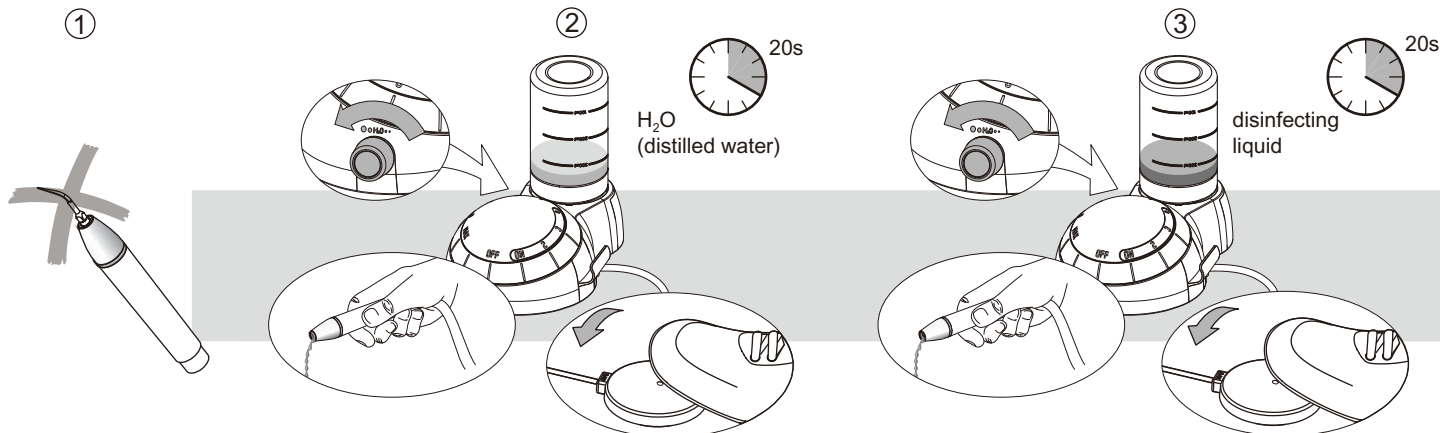
Disinfecting the circuit

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 It is recommended to disinfect the liquid circuit daily.

 Disinfection is necessary to reduce crystalline deposits and the number of bacteria in the liquid circuit of the unit; the following procedure achieves a 5 log reduction of target pathogens (using disinfecting solutions for washing the circuit as listed at the beginning of this manual):

- 1 Remove the instrument,
- 2 Wash the circuit during 20 seconds with distilled (or demineralized) water,
- 3 Disinfect the circuit during 20 seconds with an appropriate disinfecting solution for washing the circuit
- 4 Allow the disinfecting liquid to act for 5 minutes,
- 5 Rinse the circuit during 20 seconds with sterile water.

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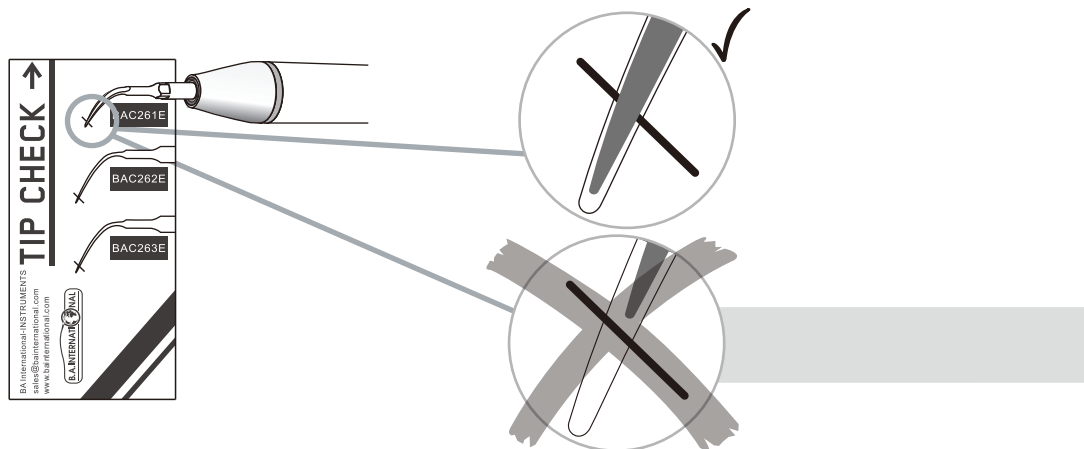
The BA International PIEZO instruments must be checked regularly using the BA International "Tip Card". Ultrasound instruments wear during use and become shorter. Worn instruments are ineffective and are a cause of discomfort to the patient.

Instruments showing wear on the diamond coating must be replaced.

For precautionary reasons, do not exceed the validated lifespan of the components (see the Technical Data chapter).

Always use BA International original parts. Using nonoriginal components may damage the equipment, and practitioner or patient may be injured.

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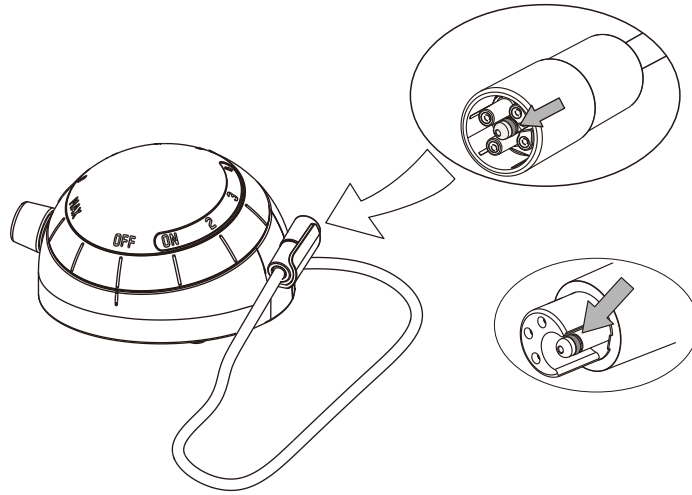
Verification of O-rings

Vérification du joint torique

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 A damaged or worn out O-ring must be replaced immediately.

 Check that both connectors of the cord are dry before connecting it to the unit.

 Please refer to the instructions provided with the maintenance set to replace an O-ring.

 Always use BA International original parts. Using nonoriginal instruments may damage the unit, and practitioner or patient may be injured.

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SAFETY PRECAUTIONS

BA International and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

Use for the intended purpose only: before using the product, make sure that you have studied the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

Risk of explosion: do not use this product in the presence of inflammable anaesthetics or gases.

The operator shall not touch the cord connector and the patient simultaneously.

During treatment, avoid touching the patient with the handpiece body, as the heat from the handpiece can be uncomfortable for the patient.

This product must be used only by trained and qualified personnel.

Before each treatment, check the contents of the bottle as well as the liquid present in the circuit.

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 Clean, disinfect and sterilize the different pieces and accessories of this product before each use. Please refer to the information provided in this operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 This product has been investigated with regard to safety from electrical shock and fire hazard. ETL (Electrical Testing Laboratories) has not investigated the physiological effects.

 Always examine the product for damage before commencing treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original BA spare parts and accessories only.

 Have this product exclusively repaired by an approved BA International repair center.

 Use exclusively the external power supply (ref.BAC20044072) delivered with the BA International unit.

 Use exclusively the pedal (ref.BAC20044071) delivered with the BA International unit.

 Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the "Electromagnetic compatibility" manual must be taken into account.

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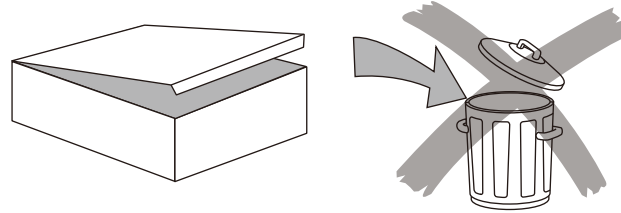
 Keep the original packaging until the product is to be disposed of permanently. It can be used for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the "Cleaning, disinfecting and sterilizing" chapter,
- Proceed to a washing of the circuit without bottle to empty it,
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data" chapter.

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DISPOSAL

The product must not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

Waste Electrical and Electronic Equipment belonging to customers located in the European Union may be shipped to BA International for recycling in accordance to the WEEE regulations. The costs of recycling, exclusive of shipping fees, are covered by BA International.

WARRANTY

Damage due to non-adherence to the operating instructions or resulting from the wear out of parts are excluded from warranty.

The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from BA International or any authorized dealers. Please contact your customer service directly.

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BA International SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or to an authorized BA International Repair Center.

In the case of non-authorized repairs or damages due to non-adherence to the operating instructions, BA International accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Risk of transmitting micro-organisms! To protect the personnel of the repair center and for safety reasons during transport and shipment, the product and the accessories returned to the factory for repair or revision must be cleaned, disinfected and sterilized in accordance with the operating manual. Repair can be refused for product or accessories received in a contaminated condition.

When sending your product directly to the approved BA International repair center, please include the name of your dealer. This simplifies processing.

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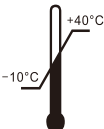
TECHNICAL DATA

Description

Manufacturer	BA International Ltd. Unit 9, Kingsthorpe Business Centre, Studland Road, Kingsthorpe, NORTHAMPTON, NN2 6NE, UK
Models	OPTIMA BAC100 & OPTIMA BAC200
Classification EN 60601-1	- Class II - Applied part, BAC100 Type B, BAC200 Type BF - IP 20, unit; IP X1, foot pedal
Classification 93 / 42 EEC	Class IIa
Operating mode: With irrigation Without irrigation	Continuous operation Duty cycle 10% during 10 min. maximum
Power Supply: External voltage unit	100-240 VAC / 50-60 Hz 30 VDC
Power consumption	45 VA
Ultrasonic specifications: Maximum output Frequency range	8 Watts 24-32 kHz
Weight	OPTIMA BAC100: Approx 0.6 kg OPTIMA BAC200: Approx 1 kg
Dimensions in mm (H x W x D)	OPTIMA BAC100: 96 x 170 x 135 OPTIMA BAC200: 96 x 153 x 215
Operating conditions	+10 °C to +40 °C 30% to 75% relative humidity Altitude max. 3000 m
Storage and transport conditions	-10 °C to +40 °C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure
Expected service life	5 years
Sterilization for handpiece	This product is conformed to ISO 22374 which is capable of withstanding a minimum of 250 sterilization cycles.

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TROUBLESHOOTING

Symptom	Solution
The LEDs of the power supply do not illuminate when the unit is switched on	<ul style="list-style-type: none">• Check that the mains cord of the power supply is properly connected to the mains supply.• Check the power supply of the dental office.• Turn the power dial of the unit to ON and check if the LED is lit.• Replace the transformer.• Send the unit and the accessories to an approved BA International repair center.
No irrigation flow when the foot pedal is pressed	<ul style="list-style-type: none">• Turn the liquid flow rate knob to the maximum.• Make sure that water comes in external device for the OPTIMA BAC100.• Make sure that the bottle is properly installed for the OPTIMA BAC200.• Disconnect the piezo handpiece and check that the liquid exits from the cord.• Use compressed air (max. 3 bar) and check that air comes out of the handpiece.• Press the bottle by hand and push the air through to tube to change the negative pressure status in whole system for the OPTIMA BAC200.• Replace the water filter for the OPTIMA BAC100.• Replace the pump head for the OPTIMA BAC200.• Check the connection of the foot pedal.• Replace the foot pedal.• Send the unit and the accessories to an approved BA International repair center.
No ultrasound oscillations when the foot pedal is pressed	<ul style="list-style-type: none">• Check that the handpiece is connected to the handpiece cord.• Check that the instrument is tight.• Check that the instrument is not cracked.• Check the connection of the foot pedal.• Replace the foot pedal.• Send the unit and the accessories to an approved BA International repair center.
Ultrasound power decreasing or inadequate	<ul style="list-style-type: none">• Check that the selector is beyond the position 4.• Check that the instrument is tight.• Check the wear of the instrument and replace if necessary.• Replace the handpiece.• Send the unit and the accessories to an approved BA International repair center.
Water leaks between the handpiece cord and the handpiece	<ul style="list-style-type: none">• Replace the O-ring on the handpiece cord.• Replace the handpiece.• Send the handpiece to an approved BA International repair center.
Only OPTIMA BAC200: Water leaks from the bottle or the device	<ul style="list-style-type: none">• Check the O-ring of the bottle cap.• Replace the bottle.• Send the unit and the accessories to an approved BA International repair center.
Insufficient lighting of the handpiece	<ul style="list-style-type: none">• Send the handpiece to an approved BA International repair center.

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BA International Ltd.
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
NORTHAMPTON, NN2 6NE, UK
Telephone: +441604 777700
Fax: +441604 721844
E-mail: sales@bainternational.com

bainternational.com