Eurosept® Plus Impression Disinfection Liquid

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(CS) Tekutina k dezinfekci otisků

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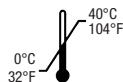
EN: Area of application: Disinfection of all dental impression materials. **Instructions for use:** Immerse the respective impression into the ready-for-use disinfectant solution, and remove again after the recommended application time. The impression must be fully covered by the solution. Only immerse alginates and hydrocolloids for a short time and let them dry. Then rinse the impression with water. Do not mix with cleaning agents or other disinfectants. Depending on the degree of contamination, the working solution is to be replaced (RRI recommendation: at least once per working day). The working solution can be disposed of as sewage water. **Application times:** Bacteria (incl. MRSA) as per VAH, EN 13727 and EN 14561 (dirty conditions); Conc. 1 min.; Yeastocidal (Candida albicans) as per VAH, EN 13624 and EN 14562 (dirty conditions); Conc. 1 min.; Limited virucidal (Enveloped viruses e.g. HBV, HCV and HIV) as per RAI/ DWY (with biological load); Conc. 1 min. (with contamination); T68 (M. terrae in a suspension test, clean conditions); Conc. 15 min.; Noroviruses as per EN 14476 (clean conditions); Conc. 30 min.; Rotaviruses as per BGA/DWI (clean conditions); Conc. 1 min. **Safety guidelines:** Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of contents/ container in accordance with all local, regional, national and international regulations. **Directive 649/ 2004/ EC:** Contains < 5 % cationic surfactants, fragrance. **100 g solution contain:** 0.24 g of benzyl C 12-16-alkyl [ethylphenyl] methyl dimethylammonium chloride, 0.24 g of N, N-didecyl-N, N-dimethylammonium chloride, 0.24 g of C12-14-alkyl [ethylphenyl] methyl dimethylammonium chloride, 0.35 g chlorhexidine gluconate.

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