RGI-GLASS IONOMER FILLING MATERIAL 2.5"

• Fast Setting

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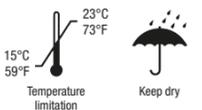
15°C
59°F
Temperature limitation

Keep dry

Caution: Federal law restricts this device to sale by or on the order of a physician.

Do not reuse

Caution, consult accompanying documents

Distributed by (US):

HENRY SCHEIN INC.
135 DURYEA ROAD
Melville, NY 11747 USA

EC REP

HENRY SCHEIN SERVICES GmbH

MONZASTRASSE 2A

D-63225 LANGEN, GERMANY

Made in Germany

Rev.2018/01

Instructions for use

(In accordance with DIN EN ISO 9917-1 : 2007)

Product description:

RGI-Glass Ionomer Filling Material 2.5" - Fast Setting is a radiopaque glass ionomer cement for fillings in capsules for direct application. It adheres chemically to enamel and dentine and continuously releases fluoride ions. Usable quantity of cement per capsule is at least 0.15 ml.

Indications:

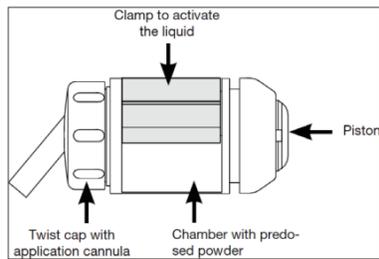
- Restoration of V-shaped defects and cervical enamel or root erosions including class V cavities
- Restoration of deciduous teeth (especially class II)
- Fillings of class II cavities (not occlusion-bearing) by means of tunnel preparations
- Base/liner, Core build-up
- Temporary long-term treatment of class I and II cavities

Method of Use:

Preparation:

The adhesion of RGI-Glass Ionomer Filling Material 2.5" - Fast Setting to enamel and dentine prevents formation of marginal gaps and creates good retention. Undercuts give additional hold to the filling. The layer thickness of the restorations must not be less than 1 mm.

Avoid feather-shaped margins. If there is any doubt opt for conventional preparation. Clean the cavity (e.g. 3 % hydrogen peroxide), thoroughly rinse with water and dry (warm air). Do not overdry the dentine: it should have a moist, shiny appearance. If the remaining dentine is thicker than 1.5 mm additional lining is not necessary. If the residual dentine thickness is insufficient, the area close to the pulp should be covered with a suitable calcium hydroxide product. The capsule must be activated, mixed and opened before application.

Activation:

For correct activation the liquid from the sachet needs to be fully emptied into the inner chamber of the capsule, carried out with a capsule activator. Place the capsule in the activator with the nozzle end protruding ready to extrude. Hold the lever for 2 s in a fully depressed position.

Mixing:

Mix the activated capsule for 10 s and remove from the mixer (4.000 - 4.500 oscillations per min). Insert it into the application device. Working time and consistency can be affected by the speed of the capsule mixer and the mixing time.

Opening:

To open the cannula for extrusion twist cap fully to the right until it stops.

Application:

Place the opened capsule into the applicator. The cannula can be directed as required by rotating the entire capsule within the applicator. Within 30 s of mixing, apply the material directly into the cavity by pressing the lever slowly and evenly. Do not apply excessive pressure.

Mixing time: 10 s

Working time: 1.5 min

Setting time: 2.5 min

After mixing RGI-Glass Ionomer Filling Material 2.5" - Fast Setting can be worked for about 1.5 min (at room temperature). When placing a filling the working area must be kept dry. The use of a rubber dam is recommended. Glass ionomer cements are susceptible to humidity (breath, saliva) or, respectively, dehydration during the setting time, especially directly after application. Where possible, cover immediately with a strip or matrix or apply a protective varnish e.g. Henry Schein Fluoride-Containing Dental Desensitising Varnish.

Finishing and polishing (flexible polishing discs) of fillings should not be carried out earlier than 2.5 min (setting time) after placement of the filling. Recommendation: The best results for fine finishing will be achieved approximately 24 h after placement of the filling. Re-apply protective varnish afterwards.

Information, precautionary measures:

RGI-Glass Ionomer Filling Material 2.5" - Fast Setting contains polyacrylic acid and fluoroalicate. It should not be used in case of known hypersensitivities (allergies) to this/ these ingredient/ s. RGI-Glass Ionomer Filling Material 2.5" - Fast Setting should only be mixed immediately before application to ensure full working time for insertion and shaping to avoid unnecessary pre-drying and cracking of material. Ensure that the application capsule is mixed immediately after activation.

Storage instructions and application method:

Store RGI-Glass Ionomer Filling Material 2.5" - Fast Setting in a dry place at temperatures of 15 °C - 23 °C. Only remove it from the foil immediately before use. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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