RGI-GLASS IONOMER FILLING MATERIAL 3.5"

• Regular Setting

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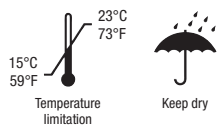
50 x

Rx only

Caution: Federal law restricts this device to sale by or on the order of a physician.

Do not reuse

Caution, consult accompanying documents

Distributed by (US):

HENRY SCHEIN INC.
135 DURYEA ROAD
MELVILLE, NY 11747 USA

EC REP

HENRY SCHEIN SERVICES GmbH
MONZASTRASSE 2A
D-63225 LANGEN, GERMANY

Made in Germany
Rev.2018/01

Instructions for use

(In accordance with DIN EN ISO 9917-1)

Product description:

RGI-Glass Ionomer Filling Material 3.5" - regular setting is a radiopaque glass ionomer bulk filling material in capsules for direct application. In addition to high compressive strength, the material has good adhesion to enamel and dentine and releases fluoride ions. Usable quantity of cement per capsule is at least 0.15 ml.

Indications:

- Restoration of non occlusion-bearing class I cavities
- Semi-permanent restorations of class I and II cavities
- Restorations of cervical lesions, class V cavities, root caries
- Restoration of class III cavities
- Restoration of deciduous teeth
- Base/liner
- Core build-up
- Temporary restorations

Application:

Preparation:

Prepare the cavity using a minimally invasive approach (removing only carious tooth substance). There is no need for undercuts. The layer thickness of the restorations must not be less than 1 mm. Avoid thinly tapering margins. Clean the cavity, rinse with water and dab or blow dry. Do not overdry the dentine: it should have a moist, shiny appearance. Place a matrix, if necessary. If the residual dentine thickness is insufficient, the area close to the pulp should be covered with a suitable calcium hydroxide product.

Activating the application capsule and mixing the material:

Do not remove the capsule from the foil bag until it is ready to be used. During activation, the plunger needs to be fully depressed into the capsule. Press the plunger end of the capsule on a hard surface (e.g. work counter) to activate. Fully depress (see fig. 1). The capsule should only be activated immediately before use and then placed in a Capsule Mixer with approximately 4,000 rpm. Mix the activated capsule for 10 - 15 s and remove from the mixer. Insert it into the application device.

Application:

Once opened (see fig. 2), and within 15 s of mixing apply the material directly into the cavity slowly and evenly. The cannula can be directed as required by rotating the entire capsule in the applicator. Working time and consistency can be affected by the speed of the capsule mixer and the mixing time.

Fig. 1: Activating the capsule

Fig. 2: Opening the capsule

After mixing, RGI-Glass Ionomer Filling Material 3.5" - regular setting can be worked for at least 1.5 minutes (at room temperature). Avoid contamination with excessive moisture and prevent from drying out. To avoid ingress of moisture, apply a protective varnish after the contours/ modelling has been completed. The restoration can be finished with a diamond bur or polisher, but no earlier than 5 - 6 minutes after mixing. A protective varnish should then be applied again.

Mixing time (23°C):	10 - 15 s
End of mixing to application (23°C):	15 s
Working time (23°C):	1.5 min
Setting time (37°C):	3 - 4 min
Total setting time:	5 - 6 min

Information, precautionary measures:

RGI-Glass Ionomer Filling Material 3.5" - regular setting contains polyacrylic acid. RGI-Glass Ionomer Filling Material 3.5" - regular setting should not be used in case of known hypersensitivities (allergies) to this ingredient. Avoid direct application to dentine near the pulp or to exposed pulp. In case of contact with the eyes, rinse immediately with plenty of water and consult a medical specialist. Any instruments used should be thoroughly rinsed with cold water immediately after use to remove material before it sets. Ensure that the application capsule is mixed immediately after activation.

Storage instructions and application method:

RGI-Glass Ionomer Filling Material 3.5" - regular setting in a dry place at temperatures of 15 °C - 23 °C. Only remove it from the foil immediately before use. Do not use after expiry date. Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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