RGI-Glass Inomer Filling Material • LC

- (ES)** Material de ionómero de vidrio para obturaciones
 - fotopolimerizable
- (FR)** Matériau d'obturation verre ionomère de restauration
 - photopolymérisable
- (DE)** HS-Glasionomer Füllungsmaterial
 - lichthärtend
- (IT)** Materiale vetroionomero per otturazioni
 - fotopolimerizzabile
- (NL)** Glasionomer vulmateriaal
 - lichtuithardend

- (PT)** Ionómero de vidro para restauração • fotopolimerizável
- (EL)** Υαλο-ιονομερικό υλικό πλήρωσης • φωτο-πολυμεριζόμενο
- (TR)** Cam İyonomer malzeme • epkila sertlegen
- (CS)** Sklioinomerní výplňový materiál • světlem tuhnutí
- (HR)** Staklenoinomerni materijal • svjetlom tvrdnjenje
- (HU)** Üvegionomer töltőanyag • fényreköthető
- (PL)** Szkło-ionomerowym materiałem do wypełnień • światłoutwardzalny
- (RO)** Material ionomer de sticlă pentru umplere • fotopolimerizabil cu rășină
- (DA)** Glassionomer fyldningsmateriale • lysghærdende
- (FI)** Lasi-ionomeeritähtemateriaali • lasipolymerisoitav
- (NO)** Glasionomer fyllingsmateriale • lysghærdende
- (SV)** Glasjonomermaterial för fyllningar • ljushärdande

Assortment:

900-4394 **A2**

900-4395 **A3**

50 x

Instructions for use

(In accordance with DIN EN ISO 9917-2 : 2010)

Product description:
RGI-Glass Inomer Filling Material - LC is a light-curing, radiopaque glass ionomer restorative material in capsules. The material has outstanding aesthetics, good handling properties and high chemical adhesion. Usable quantity of cement per capsule is at least 0.17 ml.

- Indications:**
- Class III and V restorations, primarily cervical fillings and root caries
 - Restoration of deciduous teeth
 - Small class I fillings
 - Temporary fillings
 - Core build-up
 - Linings

Application:
Preparation:
Prepare the cavity using a minimally invasive approach (removing only carious tooth substance). There is no need for undercuts. Class V cavities should have a minimum depth of 1 mm. Clean the cavity, rinse with water and dab or blow dry. Place a matrix, if necessary. If the residual dentine thickness is insufficient, the area close to the pulp should be covered with a suitable calcium hydroxide product.

Activating the application capsule and mixing the material:
Do not remove the capsule from the foil bag until it is ready to be used. During activation, the plunger needs to be fully depressed into the capsule. Press the plunger end of the capsule on a hard surface (e.g. work counter) to activate. Fully depress (see fig. 1). The capsule should only be activated immediately before use and then placed in a Capsule Mixer with approximately 4000 rpm. Mix the activated capsule for 10 s and remove from the mixer. Insert it into the application device.

Application:
Once opened, and within 10 s of mixing apply the material directly into the cavity slowly and evenly. The cannula can be directed as required by rotating the entire capsule in the applicator. Working time and consistency can be affected by the speed of capsule mixer and mixing time.

Avoid unnecessary delays between application and finishing as the setting process may begin and make working with the material difficult. Apply the selected shade of RGI-Glass Inomer Filling Material - LC in layers with a maximum thickness of 2 mm, adapt with a suitable instrument and then light cure. After mixing, RGI-Glass Inomer Filling Material - LC can be worked for 2.5 minutes (at room temperature). Higher temperatures shorten and lower temperatures extend these working times.

Light-curing:
Conventional polymerisation devices are suitable for light curing the material. The curing time using an LED-/halogen light (minimum light-output of 500 mW/cm²) is 20 s per layer. Hold the light tip as close as possible to the surface of the filling (5 mm or less).

Quantity per capsule:	minimum 0.17 ml
Net weight per capsule:	minimum 0.35 g
Mixing time (23 °C):	10 s
Working time (23 °C):	minimum 2.5 min.
Light-curing:	20 s

Finishing:
After light-curing, remove excess material and finish and polish the filling as usual.

Information, precautionary measures:
Ensure that the application capsule is mixed immediately after activation. In case of contact with oral mucosa or skin, remove immediately with a cotton swab soaked in alcohol. Rinse with water. In case of contact with the eyes, rinse immediately with plenty of water and consult a medical specialist. RGI-Glass Inomer Filling Material - LC contains polyacrylic acid, amines, BHT and methacrylates. RGI-Glass Inomer Filling Material - LC should not be used in case of known hypersensitivities (allergies) to these ingredients. Phenolic substances, especially preparations containing eugenol or thymol, disturb the polymerisation of the restorative cement. Do not use zinc-oxide eugenol cements or other eugenol-containing materials together with RGI-Glass Inomer Filling Material - LC.

Storage instructions and application method:
Store RGI-Glass Inomer Filling Material - LC in a dry place at temperatures of 4 °C - 23 °C. Only remove it from the foil immediately before use. Do not use after expiry date. Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

4°C
39°F

Temperature limitation

23°C
73°F

Keep dry

Rx only

Caution: Federal law restricts this device to sale by or on the order of a physician.

Do not reuse

Caution, consult accompanying documents

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