## LGI-GLASS IONOMER LUTING CEMENT

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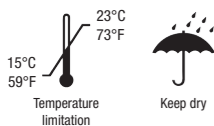
# 50 x

# Rx only

Caution: Federal law restricts this device to sale by or on the order of a physician.

Do not reuse

Caution, consult accompanying documents

Distributed by (US):  
HENRY SCHEIN INC.  
135 DURYEA ROAD  
Melville, NY 11747 USA  
EC REP

HENRY SCHEIN SERVICES GmbH  
MONZA STRASSE 2A  
D-63225 LANGEN, GERMANY  
Made in Germany  
Rev.2018/01

## Instructions for use

(In accordance with DIN EN ISO 9917)

(EN)

### Product description:

LGI-Glass Ionomer Luting Cement is a radiopaque glass ionomer cement for luting in capsules for direct application. LGI-Glass Ionomer Luting Cement has good rheological properties, high levels of compressive strength, fluoride release and low film-thickness. Usable quantity of cement per capsule is at least 0.19 ml.

### Indications:

- Luting of metal-based crowns, bridges, inlays and onlays
- Luting of high-strength all-ceramic crowns and bridges (such as aluminium oxide, lithium disilicate and zirconium dioxide ceramics)
- Luting of metal-based and high-strength all-ceramic crowns and bridges (such as aluminium oxide, lithium disilicate and zirconium dioxide ceramics) on implant abutments
- Luting of metal root posts or indirect metal post core build-ups
- Luting of prefabricated steel crowns
- Luting of orthodontic bands

### Method of Use:

#### Preparation:

Pre-treatment of the restoration according to the manufacturer's instructions (e.g. sandblasting of the inside of metal restorations). If necessary, degrease using alcohol or acetone. The area close to the pulp should be covered with a suitable calcium hydroxide product. Clean the cavity, rinse with water and dab or blow dry. Do not overdry the dentine: it should have a moist, shiny appearance.

### Activating the application capsule and mixing the material:

Do not remove the capsule from the foil bag until it is ready to be used. Before use, shake the capsule by hand to loosen the powder. Press the plunger end of the capsule on a hard surface (e.g. work counter) to activate. Fully depress (see fig. 1). The capsule should only be activated immediately before use and then placed in a capsule mixer with approximately 4000rpm. Mix the activated capsule for 10s and remove from the mixer. Insert it into the application device. Working time and consistency can be affected by the speed of the capsule mixer and the mixing time.

Fig. 1: Activating the application capsule

### Application:

Carefully open the outlet by lifting the application tip until you feel a stop (see fig. 2). Place the opened capsule into the applicator, following the instructions for use. The cannula can be directed as required by rotating the entire capsule within the applicator. Dispense the material directly on the prepared surfaces using the lever. Distribute evenly via the application tip. Always apply a slight excess of material. Immediately put the restoration into place and hold it applying light pressure until the curing process is finished. Do not delay between activation, mixing and application or dispensing the material will be difficult.

Fig. 2: Opening the application capsule

At room temperature, LGI-Glass Ionomer Luting Cement has a working time of 3 minutes, before setting begins after 5 to 7 minutes (3 - 5 mins. at 37 °C). Higher temperatures shorten and lower temperatures extend these working times. Prevent the ingress of water and saliva during application and setting phase. Excess material can be easily removed during the rubber-elastic phase (before complete curing) using a scaler or a curette.

### Information, precautionary measures:

LGI-Glass Ionomer Luting Cement contains polyacrylic acid. LGI-Glass Ionomer Luting Cement should not be used in case of known hypersensitivities (allergies) to this/ these ingredient/s. Avoid direct application to dentine near the pulp or to exposed pulp. In case of contact with the eyes, rinse immediately with plenty of water and consult a medical specialist. LGI-Glass Ionomer Luting Cement is not intended for non-reinforced all-ceramic restorations such as silicate ceramics, nor for indirect composite restorations, orthodontic brackets, ceramic inlays, ceramic onlays or veneers. Any instruments used should be thoroughly rinsed with cold water immediately after use to remove material before it sets. Once orthodontic bands are removed, any residues can be gently removed with an ultrasonic scaler. Ensure that the application capsule is mixed immediately after activation.

### Storage instructions and application method:

Store LGI-Glass Ionomer Luting Cement in a dry place at temperatures of 15 °C - 23 °C. Only remove it from the foil immediately before use. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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