# LGI-GLASS IONOMER LUTING CEMENT

• Resin Reinforced

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LY  
50x

## Instructions for use (In accordance with DIN EN ISO 9917-2, class 1)

**Product description:**  
LGI-Glass Ionomer Luting Cement, Resin reinforced is a radiopaque, chemically curing, resin-modified glass ionomer luting cement in capsules. The material characteristics include easy handling, low solubility in the mouth and a high self-adhesion. Usable quantity of cement per capsule is at least 0.14 ml.

**Indications:**  
Luting of  
- porcelain-fused-to-metal crowns, bridges on hard dental tissue and on cores rebuilt with amalgam, composites or glass ionomer materials  
- Metal inlays, onlays, crowns and bridges  
- root posts  
- all-ceramic crowns manufactured from silicate ceramic, zirconium dioxide-only or alumina-only ceramic cores  
- orthodontic appliances

**Method of Use:**  
Thoroughly clean and dry enamel, dentine and cavity surfaces. Clean cores before luting (e.g. pumice, water), thoroughly rinse and dry with a gentle stream of air or cotton pellets (do not overdry). Before application, the capsule should be activated, mixed and opened.

**Activating the application capsule and mixing the material:**  
Do not remove the capsule from the foil bag until it is ready to be used. During activation, the plunger needs to be fully depressed into the capsule. Press the plunger end of the capsule on a hard surface (e.g. work counter) to activate. Fully depress. The capsule should only be activated immediately before use and then placed in a Capsule Mixer with approximately 4000 rpm. Mix the activated capsule for 10 sec and remove from the mixer. Insert it into the application device.

**Application:**  
Once opened, and within 10 sec of mixing apply the material directly into the cavity slowly and evenly. The cannula can be directed as required by rotating the entire capsule in the applicator. Working time and consistency can be affected by the speed of the capsule mixer and the mixer time.

Fig. 1: Activating the capsule

Fig. 2: Opening the capsule

**Luting of restorative pieces:**  
Apply the material onto the prepared area and/or into the prosthesis. Place the prosthesis in situ and set fully, ensure pressure is maintained during setting. Any unnecessary waiting times between activation, mixing and application should be avoided, as the process of curing may impede the extraction of the material and the positioning of the restoration. LGI-Glass Ionomer Luting Cement, Resin reinforced has a working time of approx. 2 min at room temperature and sets in the mouth in 3 - 5 min. After approx. 4 min the material reaches a gel-like consistency, making it easy to remove any excess material with the aid of a probe. Higher temperatures shorten and lower temperatures extend these working times.

**Luting of posts:**  
Apply LGI-Glass Ionomer Luting Cement, Resin reinforced only to the post and then place the post directly into the root canal. Agitate the post slightly during placement to avoid trapping air. Do not apply the cement into the canal with a lentulo since this could increase the speed of setting.

Quantity per capsule:	min. 0.14 ml
Net weight per capsule:	min. 0.27 g
Mixing time (23 °C):	10 s
Working time (23 °C):	min. 2 min
Setting time (37 °C):	3 - 5 min

**Information, precautionary measures:**  
Ensure that the application capsule is mixed immediately after activation. Avoid contact with eugenol, thymol or other substances which may inhibit the setting of LGI-Glass Ionomer Luting Cement, Resin reinforced. Use eugenol free temporary cement. Before final luting, remove all residues of varnishes and other substances (e.g. insulating agents) completely from the enamel as they may impair adhesion. Ensure sufficient residual dentine thickness to prevent pulp irritation. If necessary, protect the pulp or build up the core. Any instruments used should be thoroughly rinsed with cold water immediately after use to remove material before it sets. Once orthodontic bands are removed, any residues can be gently removed with an ultrasonic scaler. Avoid contact of LGI-Glass Ionomer Luting Cement, Resin reinforced with soft tissue. LGI-Glass Ionomer Luting Cement, Resin reinforced contains polyacrylic acid, peroxide and methacrylates. LGI-Glass Ionomer Luting Cement, Resin reinforced should not be used in case of known hypersensitivities (allergies) to this/ these ingredient/ s. In case of contact with the eyes, rinse immediately with plenty of water and consult a medical specialist.

**Storage instructions and application method:**  
Store LGI-Glass Ionomer Luting Cement, Resin reinforced in a dry place at temperatures of 15 °C - 23 °C. Only remove it from the foil immediately before use. Do not use after expiry date. Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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