# VISALYS® CEMCORE TRY IN PASTE

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### ENGLISH

### Product description / Indication

Visalys® CemCore Try In Paste serves to simulate the shade effect of the selected Visalys® CemCore shade on ceramic restorations and for covering the cement joint to avoid a layer of inhibition. The water-soluble paste corresponds in shade and translucence with the completely cured Visalys® CemCore.

Visalys® CemCore Try In Paste is offered in the shades Universal (A2/A3), Dark (A4), Opaque, Bleach and Translucent.

Note: Only once the cement composition Visalys® CemCore is fully polymerized and has attained its final shade, the shade matching with the Visalys® CemCore Try In Paste is attained.

Behave while that water-soluble version is, is Kettenbach GmbH & Co. KG neither liable for damage or loss resulting from the use of the product, nor does it guarantee the quality of the product.

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As a result of the hydroscopic property of the product, fluid shifts can arise in the dentin tubules and cause brief pain sensations in sensitive persons.

Residues of the Visalys® CemCore Try In Paste on the surfaces of the restoration or the core can impair the adhesive strength of Visalys® CemCore. Visalys® CemCore Try In Paste does not set and is not suitable for cementation of restorations.

Keep out of reach for children.

Cautions

Do not use after the expiration date.

Do not swallow material. If the material is inadvertently swallowed: Consult a medical doctor if problems arise or persist.

Avoid contact with skin and mucous tissue.

Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with an eye wash or water. Take the appropriate protective measures (e.g. wear safety glasses). Consult a medical doctor if problems arise or persist.

Safety data sheets are available at [www.kettenbach.com](http://www.kettenbach.com).

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As a result of the hydroscopic property of the product, fluid shifts can arise in the dentin tubules and cause brief pain sensations in sensitive patients.

Residues of the Visalys® CemCore Try In Paste on the surfaces of the restoration or the core can impair the adhesive strength of Visalys® CemCore. Visalys® CemCore Try In Paste does not set and is not suitable for cementation of restorations.

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