# Automat XR/C+

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## Cleaning



In order to achieve optimal picture quality we strongly recommend regular cleaning of the roller sets and the baths. Please refer also to the relevant appliance documentation.

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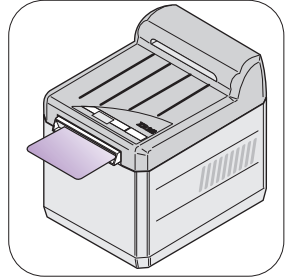
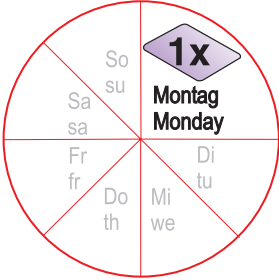
using at least weekly cleaning films, e.g. „Clear“ (Dürr Dental) before starting work.

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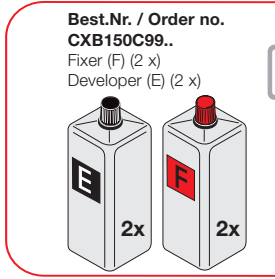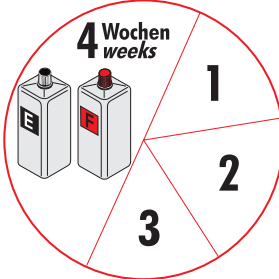## Every 2 months rinse developer tank and roller packet

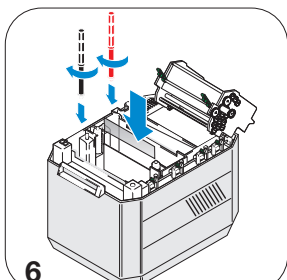
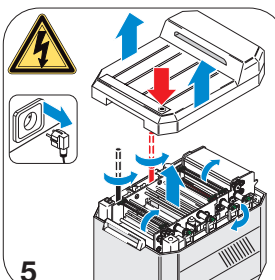
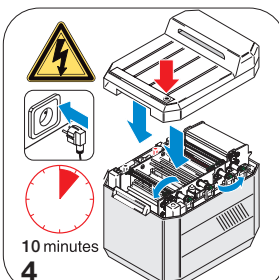
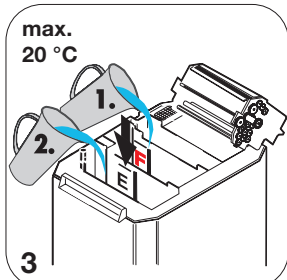
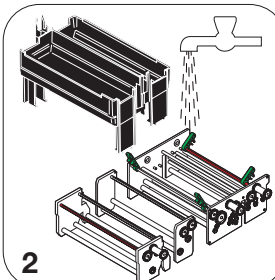
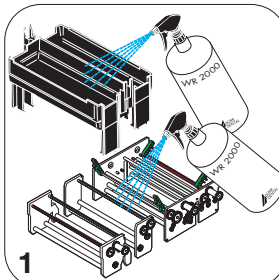
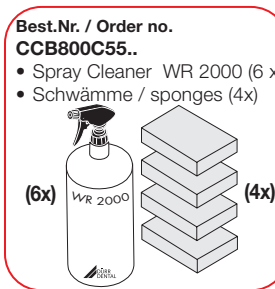
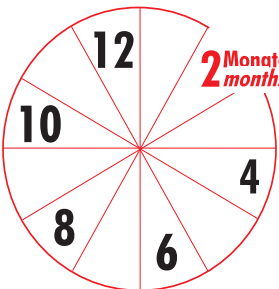
- Rinse according to requirements roller packets and lifting tank, e.g. with spray cleaner WR 2000.
- Pouring in chemicals, see next page.

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**NOTICE**  
The developer bath is rendered unusable by cleaning agents containing thiourea.  
Even the smallest amount of contamination by the cleaner in the x-ray developer means that a new solution will have to be made.  
› Never allow thiourea cleaners, such as Universal cleaner (Dürr Dental) to come into contact with the „Automat XR/C+“ X-ray chemicals.

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## Pouring in chemicals with „Automat XR/C+“

Order no. CXB 150C99..

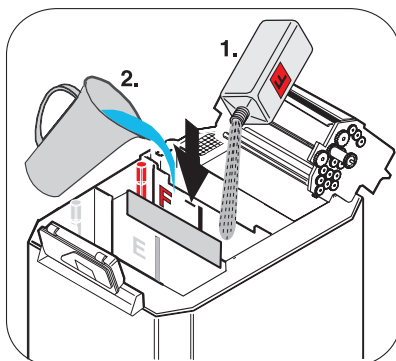
In order to avoid a bad solution or bath-mixing we strongly recommend to observe the following sequence.

Please refer also to the relevant appliance documentation!

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
### 1. Fixingbath-Setting up

- Place partition between developing and fixing tank.
- Carefully pour the fixing concentrate into the fixing tank (avoid splashes so that chemical do not mix together). Info: The productivity from 1 bottle concentrate is 5 l ready to use solution
- Fill bottle with water and pour into the fixing tank.
- Fill with water (max. 20 °C) to upper marking

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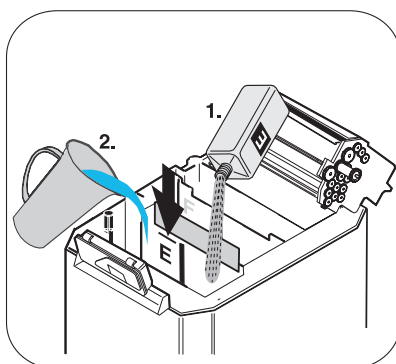### 2. Developingbath-Setting up

- Remove any splashes from making the fixing bath !
- Carefully pour the developing concentrate into the developing tank.
- Fill bottle with water and pour into the developing tank
- Fill with water (max. 20 °C) to upper marking.

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EAK 09 01 01 (Entwickler)  
EAK 09 01 04 (Fixierer)

### 3. Disposal

The empty concentrate bottles can be recycled via the Green Dot recycling system.

Used developer and fixer ready to use solutions must be collected and disposed of in accordance with the applicable national regulations.

#### Disposal key:

EAK 09 01 01 (developer)  
EAK 09 01 04 (fixer)

duerrdentaleurope GmbH + Co. KG  
Max-Planck-Straße 27  
70806 Kornwestheim, DE  
a DÜRR DENTAL company  
Tel: +49 7142 705-0  
info@duerrdental.com  
www.duerrdental.com  
Made in Germany

2019/11

9000-600-91/01

DÜRR  
DENTAL