# WASH CHECK

- ⓔᗺ WASHING TEST - INSTRUCTIONS FOR USE
- ⓔᗺ PRUEBA DE LAVADO - INSTRUCCIONES DE USO
- ⓔᗺ TEST DE LAVAGE - INSTRUCTIONS D'UTILISATION
- ⓔᗺ WASCHTEST - GEBRAUCHSANWEISUNG
- ⓔᗺ TEST DEL LAVAGGIO - ISTRUZIONI PER L'USO
- ⓔᗺ ENSAIO DE LAVAGEM - INSTRUÇÕES DE UTILIZAÇÃO
- ⓔᗺ TEST MYCIA - INSTRUKCJA UŻYCIA

Washing test strips      REF 988-2298  
 Wash test metal frame      REF 988-2299

**HENRY SCHEIN INC.**  
(manufacturer)  
 135 DURYEA ROAD  
 Melville, NY 11747 USA

**CE** **HENRY SCHEIN SERVICES GmbH**  
 MONZASTRASSE 2A  
 D-63225 LANGEN, GERMANY  
 WWW.HENRYSCHEINBRAND.COM  
 Rev.2023/02

## ⓔᗺ INSTRUCTIONS FOR USE

1. Insert the wash test strip in the metal frame.
  - a) Red test soil 1 is shielded: the water stream contacts indirectly. This part provides a gradually more difficult cleaning process.
  - b) Red test soil 2 is open: the water stream contacts directly.
2. Place the Washing test into the basket or tray.
3. Run a complete washing process.
4. After the process remove the test strip for monitoring the results and visually check for residues.

Compare the condition of the red test soil by the table “Guidance for visual determination “.

5. If the results are approved according to the guidance tabel , you can continue with the sterilization process and strip can be applied into sterilization documentation record.

Place wash test into basket  
 (Recommended placement as shown in the picture)

## Guidance for visual determination

(indirect method)

spot 1 is used for determination

Reference Standard	Spot 1 Color Difference (covered) $\Delta E$	Sample of color change spot 1      spot 2	Spot 2 Color Difference (un-covered) $\Delta E$
	$82 \pm 3$ (before wash)		$82 \pm 3$
	$40 \pm 3$		$40 \pm 3$
<b>Acceptable Value</b> ★	<b>Residual volume of protein substance below 200 <math>\mu\text{g}</math> (<math>\Delta E 21 \pm 1</math>)</b>		
	19.2		13.5
	11.6		5.6
<b>Targeted Value</b> ★	<b>Residual volume of protein substance below 100 <math>\mu\text{g}</math> (<math>\Delta E 9.3 \pm 1</math>)</b>		
	8.5		6.6
	8.4		1.9
	4.6		0.5
	1.6		0.3
<b>Conditions</b>	Washer Disinfectant : Miele G7882 – program : 60~93°C/ 10 minutes (RKI Program) – Placement Point : Center of rack Alkaline Detergent (pH Value Approx. 11.5) is used.		
<b>Note</b>	<p>The reference standard was made out based on our own internal data. Washing conditions, such as maximum capability, may differ depending on the model and the production year of the WD. Please use this as a guideline for controlling the within-day fluctuation and day-to-day variation of the washing performance of your own WD.</p> <p>★ Residual volume of protein substance – ( Targeted Value : 100 <math>\mu\text{g}</math> &amp; Acceptable Value : 200 <math>\mu\text{g}</math>)</p> <p>When variations are observed in the washing performance, it is recommended to verify the followings.</p> <ol style="list-style-type: none"> <li>1. Verify the washing conditions such as flow of detergent, clogging in the spray arm, and over loading, etc.</li> <li>2. Verify the residual volume of protein substance by direct method in your own facility.</li> </ol>		

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