BiodentineTM XP

Active Biosilicate TechnologyTM

(fr)	Substitut dentinaire bioactif	1
(en)	Bioactive dentine substitute	4
(es)	Sustituto bioactivo de la dentina	6
(de)	Bioaktiver Dentinersatz	9
(it)	Sostituto bioattivo della dentina	12
(pt)	Substituto bioativo de dentina	15
(nl)	Bioactief dentinesubstituut	18
(pl)	Bioaktywny substytut zębiny	21
(el)	Βιοενεργό υποκατάστατο οδοντίνης	24
(bg)	Биоактивен заместител на дентин	27
(cs)	Bioaktivní náhrada dentinu	30
(da)	Bioaktiv dentinerstatning	33
(et)	Bioaktiivne dentiini aseaine	36
(hu)	Bioaktív dentin helyettesítő	38
(lt)	Biologiškai aktyvus dentino pakaitalas	41
(lv)	Bioloģiski aktīvs dentīna aizstājējs	44
(ro)	Substitut de dentină bioactiv	47
(sk)	Bioaktívna náhrada dentínu	50
(sl)	Bioaktivni nadomestek dentina	53
(sv)	Bioaktiv dentinersättning	56
(hr)	Bioaktivna zamjena za dentin	59
(no)	Bioaktiv dentinerstatning	62

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SYMBOLE

CH REP Mandataire CH

Biodentine™ XP is a bioactive dentine substitute from the “Active Biosilicate Technology™” innovation.

PRESENTATION

Biodentine™ XP 500 (red piston): box of 10 foils, each foil contains 1 cartridge of 700 mg of powder and 190µL of Liquid.

Biodentine™ XP 200 (black piston): box of 10 foils, each foil contains 1 cartridge of 468 mg of powder and 133 µL of Liquid.

COMPOSITION

Cartridge contains both the powder & liquid parts:

- Powder part is composed of tricalcium silicate, zirconium oxide, calcium oxide, calcium carbonate and iron oxides.
- Liquid part is composed of calcium chloride, polycarboxylate and purified water.

INTENDED USE

Bioactive dentine substitute.

TARGET GROUP(S)

Target population

Biodentine™ XP is intended to be applied on temporary teeth (from 2 years old), on permanent teeth (mature or immature) or on non-vital immature permanent teeth depending on indication. Please refer to the dedicated section (§Indications for use).

Intended users

Dental healthcare professional only.

CLINICAL BENEFIT

- Preservation of the tooth vitality (except for apexification, perforations and furcations)
- Preservation of the tooth functionality
- Revitalization of the necrotic tooth

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

Summary of Safety and Clinical Performance is available, using Basic UDI-DI Code 37601407000004G9, in the European database on medical devices (Eudamed): <https://ec.europa.eu/tools/eudamed> or upon request (sscp@septodont.com) until EUDAMED will be fully functional.

PERFORMANCE CHARACTERISTICS

Technical performances / specifications

- Compressive strength:

At 24 h, compressive strength \geq 150 MPa

After 28 days, compressive strength: between 260 MPa and 300 MPa which is similar to the sound dentine.

- Sealing ability : creation of a tight seal with the dentin tubule.
- Setting time: the average setting time of the product is 12 minutes, starting from the end of mix. Depending on some conditions, that include storage time and conditions, the setting time may vary (from 9 to 25 minutes).
- Working time : \geq 1 min

Biological performance

Bioactivity of Biodentine relies on hydroxyapatite formation promoting a favorable environment to the angiogenesis and osteogenesis, isolating the root canal from the surrounding tissues and stimulating the healing process of the damaged apical tissues.

Clinical performance

- Preservation of the pulp
- Preservation of the amelo-dentinal structure
- Preservation of the root canal structure
- Root edification
- Sealing ability necessary to create a favorable environment required in Revitalization procedure.

INDICATIONS FOR USE

Biodentine™ XP 500 and Biodentine™ XP 200 are used for the same indications, only difference in quantity exists.

In the crown:

- For temporary teeth (children from 2 years of age) and permanent teeth (immature or mature):
- Permanent dentine restoration under composites or Inlay/Onlay.
- Temporary dentine-enamel restoration.
- Restoration of deep and/or large coronal carious lesions (sandwich technique).
- Restoration of cervical or radicular lesions.

On the pulp:

- For temporary teeth (children from 2 years of age) and permanent teeth (immature or mature):
- Pulp capping (direct and indirect)
- Pulpotomy for diagnosed symptoms of reversible pulpitis and irreversible pulpitis where bleeding is controlled within 5 minutes.

In the root:

For permanent teeth (immature or mature):

- Repair of root perforations
- Repair of furcation perforations
- Repair of perforating internal resorptions
- Repair of external resorption
- Root-end filling in endodontic surgery (retrograde filling)

For immature permanent teeth:

- Apexification
- Revitalization procedure by means of revascularization of permanent immature teeth with necrotic pulp.

CONTRA INDICATION

- Hypersensitivity to one of the ingredients.
- Restoration of large loss of tooth substance and subjected to high stresses.
- Pulp space that does require post or core for the final restoration.

EXPECTED LIFETIME

At least 10 years

INSTRUCTIONS FOR USE

Refer to Precaution for use 1 and 2

 Scan the QR code on the Protocol Card (Document 'Protocol Card') for watching the video

1. Activation of the cartridge with Biodentine™ Mixer

- 1.1. Open the lid of Biodentine™ Mixer.
- 1.2. Put and align the Biodentine™ XP cartridge in the activation socket (see Protocol Card step 3).
- 1.3. Lock the cartridge manually by making a **clockwise quarter turn**.
- 1.4. Manually pull the lever down through all its course, (until it stops) a slight click may be heard (see Protocol Card step 4).
- 1.5. Put the lever back to its initial place.
- 1.6. Unlock the cartridge from the activation socket by making a **counterclockwise quarter turn** (See Protocol Card step 7).
- 1.7. Check if the cartridge is well activated: the cartridge piston has to be pushed inside. **Refer to Precaution for use 4.**
- 1.8. Once the cartridge is activated, proceed immediately to the Mixing. **Refer to warning 9.**

2. Mixing

- 2.1. Transfer the cartridge from the activation socket to the Mixing arm of Biodentine™ Mixer (see Protocol Card steps 7 and 8).
- 2.2. Lock the cartridge manually by making a **clockwise quarter turn**.
- 2.3. Close the lid of Biodentine™ Mixer.
- 2.4. Press the START button of Biodentine™ Mixer, (see Protocol Card step 9).
- 2.5. The START button will progressively become green indicating the mixing time elapsed (time of the mix : **30 secondes**).
- 2.6. At the end of the mix, the START button will blink green and will beep twice.
- 2.7. Open the lid of Biodentine™ Mixer.
- 2.8. Unlock the cartridge from the mixing arm making a **counterclockwise** quarter turn and remove it.
- 2.9. Close the lid of Biodentine™ Mixer Turn off the mixer when further operation is no longer needed.

3. Insertion of Biodentine™ cartridge in Biodentine™ Gun

- 3.1. Immediately after mixing, raise the Biodentine™ Gun lever until it stops in horizontal position.
- 3.2. Align the Biodentine™ XP Cartridge in the Biodentine™ Gun (see Protocol Card step 10).
- 3.3. Insert the cartridge into the Gun and lock the cartridge making a **clockwise quarter turn** (see Protocol Card step 11).

4. Priming

- 4.1. Hold Biodentine™ Gun: the lever with one hand and the handle with the other hand (see Protocol Card step 12) and **progressively** push down the lever. **Refer to warning 11.**
- 4.2. Stop priming when the dentine substitute appears in the nozzle (see Protocol Card step 13).

5. Extrusion

- 5.1. Hold Biodentine™ Gun with one hand (see Protocol Card step 14).
- 5.2. The cartridge nozzle can be bent manually using adequate strength (due to the thickness of the nozzle) to facilitate the placement of the dentine substitute (see Protocol Card step 15) **Refer Warning 10.**
- 5.3. Progressively push down the lever to allow the dentine substitute to flow out of the cartridge. A homogeneous and creamy dentine substitute is obtained. **Refer to warning 4.**

6. Application

Apply the dentine substitute directly into the affected tooth cavity with Biodentine™ Gun. (see Protocol Card step 16). You may alternatively handle Biodentine™ XP with an amalgam carrier, a spatula, or a Root Canal Messing Gun.

7. Cartridge removal

Release the lever and unlock the cartridge from Biodentine™ Gun making a **counterclockwise** quarter turn then remove the cartridge (see Protocol Card step 17).

8. Disposal

Dispose the cartridge in an appropriate bin. Refer to Disposal section.

WARNINGS AND PRECAUTIONS FOR USE

Precautions for use

1. Wear the appropriate personal protective equipment while manipulating the device because it may cause eye irritation.
2. All the instructions for use must be carried out within the working area, away from the patient.
3. Water contamination slows down the setting of the material. Prevent exposure to water and fluids during the initial setting stage.
4. If the cartridge is not correctly activated, it may fall from the mixing arm.
5. Do not prime a cartridge which is not locked after a **clockwise quarter turn** in the Biodentine™ Gun.
6. Refer to Biodentine™ Mixer and Biodentine™ Gun instructions respectively for use for Biodentine™ Mixer and Biodentine™ Gun preparation.

Warnings

1. Use a rubber dam to ensure that the dentine substitute is not in contact with the mucosa or swallowed.
2. Single-use product: do not use the mixed dentine substitute on several patients to avoid cross contamination.
3. Biodentine™ XP is only intended to be mixed with Biodentine™ Mixer and extruded with Biodentine™ Gun. Otherwise, technical specifications are not guaranteed.
4. If the consistency of the obtained dentine substitute is other than homogeneous and creamy, do not use it.
5. Wait until the end of the setting time before performing the permanent enamel restoration or removing the matrix (when used).
6. For revitalization procedure, wait until the formation of blood clot before applying Biodentine™ XP. Anesthetic with vasoconstrictor may prevent blood clot formation.
7. For endodontic procedures, flush with sterile saline solution to remove any residue of irrigant and dry without totally dehydrating the root canal.
8. If the cartridge falls on the floor, clean it with ethanol to avoid contamination.
9. If **3 min** has elapsed between cartridge activation and mixing, do not use the cartridge, otherwise the dentine substitute technical performances are not guaranteed.
10. Do not bend the nozzle more than 4 times at 90° to avoid any breakage of the nozzle.
11. During priming step, do not push down the lever strongly but progressively to avoid unintended extrusion of the dentine substitute.

SIDE-EFFECTS

Possible hypersensitivity to Biodentine™ XP or ingredients.

Please report any serious incident that occurred in relation to the device to Septodont (vigilance@septodont.com) and to the local competent authority for medical devices.

STORAGE

Store between 5°C and 25°C and protect from moisture.

Any remaining material shall be discarded.

DISPOSAL

Dispose the Biodentine™ XP in an appropriate bin if it is contaminated, otherwise it can be disposed in a regular bin.

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by Arnold Bott AG