
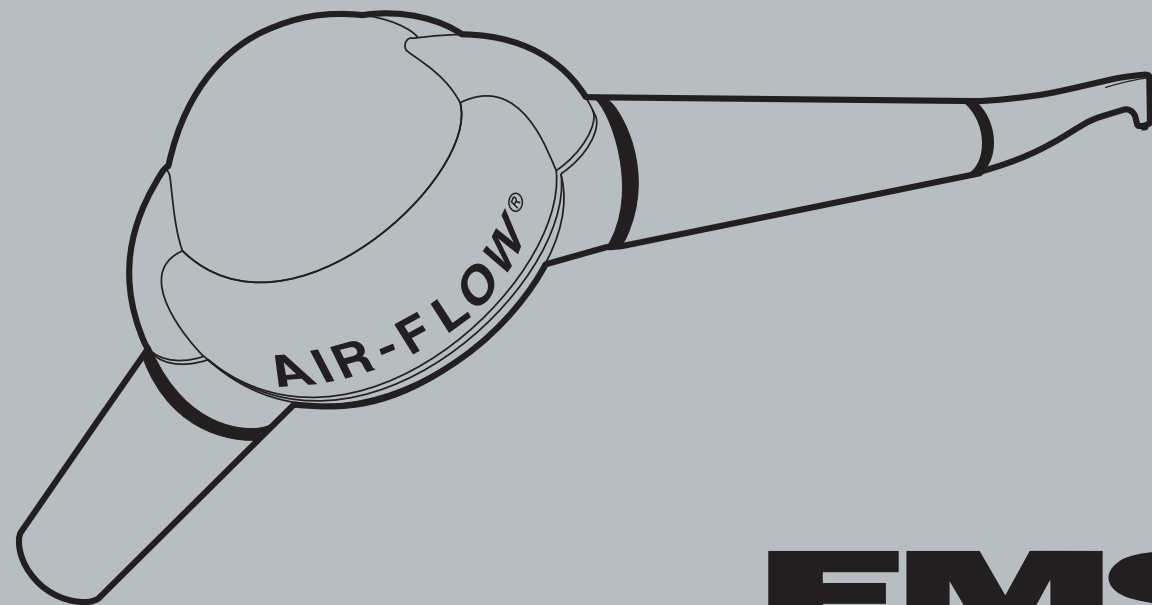
OPERATION INSTRUCTIONS  
 MODE D'EMPLOI  
 BEDIENUNGSANLEITUNG  
 INSTRUCCIONES DE EMPLEO  
 ISTRUZIONI PER L'USO  
 BRUKSANVISNING

# AIR-FLOW® handy 2+

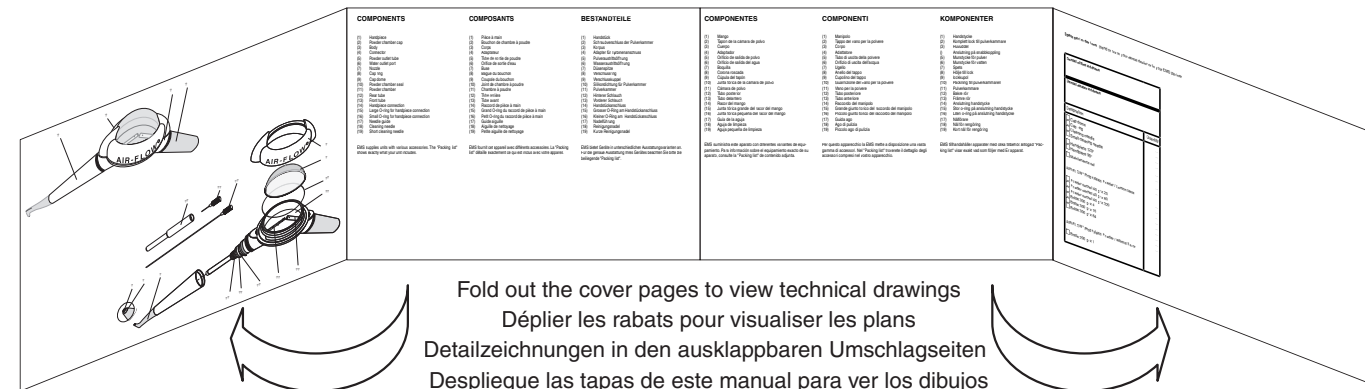**EMS**   
 ELECTRO MEDICAL SYSTEMS

ADDITIONAL SAFETY ADVICE FOR USA:  
 Caution! Federal (USA) law restricts this unit to sale by or on the order of a dentist

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## TECHNICAL MODIFICATIONS

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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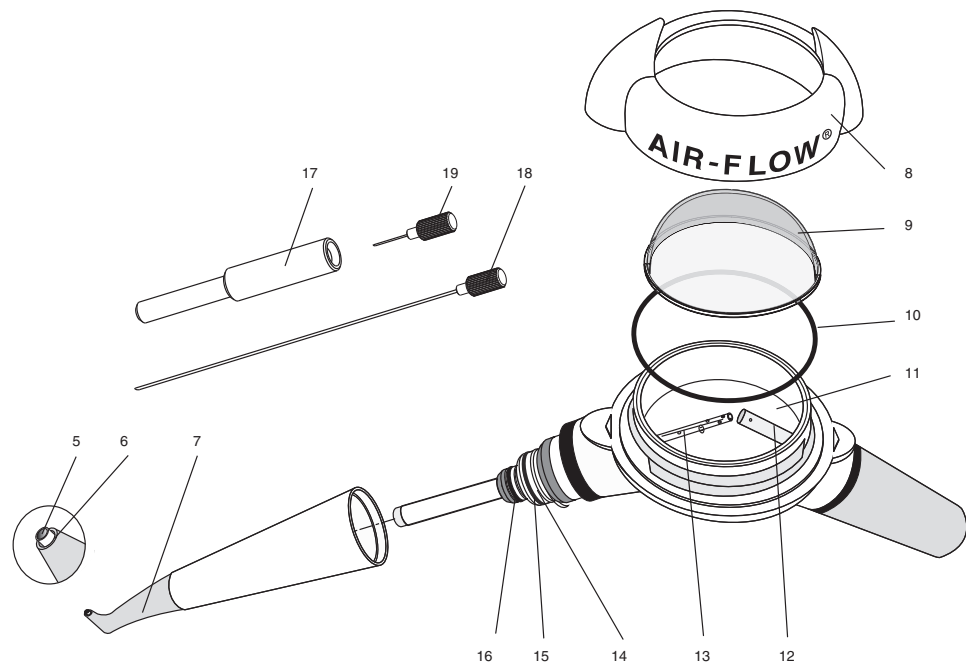
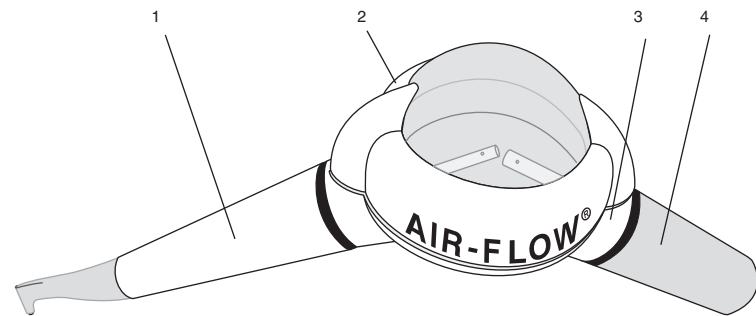
## COMPONENTS

- 1 Handpiece
- 2 Powder chamber cap
- 3 Body
- 4 Connector
- 5 Powder outlet tube
- 6 Water outlet port
- 7 Nozzle
- 8 Cap ring
- 9 Cap dome
- 10 Powder chamber seal
- 11 Powder chamber
- 12 Rear tube
- 13 Front tube
- 14 Handpiece connection
- 15 Large O-ring for handpiece connection
- 16 Small O-ring for handpiece connection
- 17 Needle guide
- 18 Long cleaning needle
- 19 Short cleaning needle

EMS supplies units with various accessories. The "Packing list" shows exactly what your unit includes.

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**Spare part order form** (send or fax to your dental dealer or to your EMS partner)

<b>Dental office address</b>	
<b>Dental dealer address</b>	
Designation	Quantity
<input type="checkbox"/> Cap dome	....
<input type="checkbox"/> Cap ring	....
<input type="checkbox"/> Long cleaning needle	....
<input type="checkbox"/> Short cleaning needle	....
<input type="checkbox"/> Handpiece 120°	....
<input type="checkbox"/> Handpiece 90°	....
<input type="checkbox"/> Maintenance set	....
AIR-FLOW® Prophylaxis Powder / Lemon taste	....
<input type="checkbox"/> Powder sachet 40 gr x 20	....
<input type="checkbox"/> Powder sachet 40 gr x 80	....
<input type="checkbox"/> Powder sachet 40 gr x 320	....
<input type="checkbox"/> Bottle 300 gr x 4	....
<input type="checkbox"/> Bottle 300 gr x 16	....
<input type="checkbox"/> Bottle 300 gr x 64	....
AIR-FLOW® Prophylaxis Powder / without flavor	....
<input type="checkbox"/> Bottle 200 gr x 1	....

**Spare part order form** (send or fax to your dental dealer or to your EMS partner)

<b>Dental office address</b>	
<b>Dental dealer address</b>	
Designation	Quantity
<input type="checkbox"/> Cap dome	....
<input type="checkbox"/> Cap ring	....
<input type="checkbox"/> Long cleaning needle	....
<input type="checkbox"/> Short cleaning needle	....
<input type="checkbox"/> Handpiece 120°	....
<input type="checkbox"/> Handpiece 90°	....
<input type="checkbox"/> Maintenance set	....
AIR-FLOW® Prophylaxis Powder / Lemon taste	....
<input type="checkbox"/> Powder sachet 40 gr x 20	....
<input type="checkbox"/> Powder sachet 40 gr x 80	....
<input type="checkbox"/> Powder sachet 40 gr x 320	....
<input type="checkbox"/> Bottle 300 gr x 4	....
<input type="checkbox"/> Bottle 300 gr x 16	....
<input type="checkbox"/> Bottle 300 gr x 64	....
AIR-FLOW® Prophylaxis Powder / without flavor	....
<input type="checkbox"/> Bottle 200 gr x 1	....

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## DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

Adaptable on the turbine connection of your dental unit, the air polishing AIR-FLOW® handy 2+ functions with the powder provided by EMS.

The confinement of the powder by a water jet makes it possible to steer the water jet with great precision and thus ensures a pleasant treatment for your patient.

This product removes dental plaque, soft deposits and surface stains from pits, grooves, interproximal spaces or smooth surfaces of the teeth.

It is also recommended for the following cleaning procedures :

- Plaque removal for placement of sealants
- Surface preparation prior to bonding/cementation of inlays, onlays, crowns and veneers
- Surface preparation prior to placing composite restorations
- Effective plaque and stain removal for orthodontic patients
- Cleaning prior to bonding ortho brackets
- Clean implant fixture prior to loading
- Stain removal for shade determination
- Plaque removal prior to fluoride treatment
- Plaque and stain removal prior to whitening procedure.

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## PLEASE READ THIS BEFORE BEGINNING WORK!

This operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this instruction close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

### **Danger**

Risk of injury

### **Caution**

Risk of property damage or environmental harm

### **Please note**

Useful additional information and hints

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### **Förbjudet**

### **Tillåtet**

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly in your approved EMS repair center or with your dealer.

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EMS

## CONTENTS

Installation and setting up.....	2
Water supply.....	2
Compressed air supply.....	2
Checking the turbine connection.....	4
Connecting to the dental unit.....	6
Setting the water flow rate.....	8
Filling the powder chamber.....	10
Closing the cap.....	12
Handling and setting water/air.....	14
General hints for treatment.....	16
General information.....	16
Wear a mask and eye protection.....	18
Patient mouthwash.....	20
Applying lip cream.....	20
Soft tissue protection.....	20
Placement of saliva pump.....	22
Positioning and using the unit.....	22
Working Technique.....	24
Precautions to be taken at the end of the treatment.....	28
Fluoride Application.....	30
Patient Information.....	32
Disinfecting, cleaning and sterilizing.....	34
Cleaning the unit.....	34
Cleaning the handpiece.....	36
Disinfecting and sterilizing the handpiece.....	38
Drying and connecting the sterilized handpiece.....	40
Regular cleaning and maintenance.....	42
Safety precautions.....	48

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max. 40°C

### Compressed air supply

Set the pressure of your unit to the maximum value allowed by your turbine to get a working pressure of 3.5 to 4.5 bar (3500-4500 hPa).

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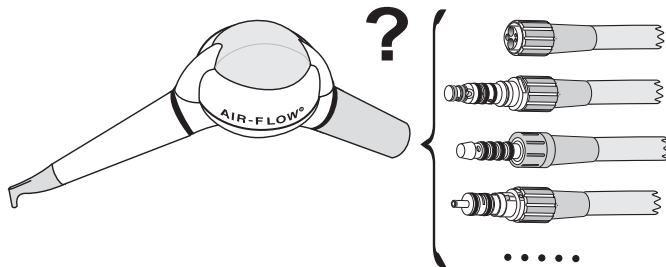
## Checking the turbine connection

⚠ The unit is equipped with an adapter specially designed for the turbine connection of your dental unit. Only use the unit with this special turbine connection. To connect it to another type of turbine will damage it.

⚠ The turbine of your dental unit must not be under pressure when to connect the unit. Do not activate the turbine footpedal. If you have a turbine equipped with light, turn the light off.

⚠ Make sure the o-rings of your turbine connection are in good condition. A turbine connection with o-rings in bad condition can damage the unit.

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## GENERAL HINTS FOR TREATMENT

### General information

 Contraindication: Patients suffering from chronic bronchitis or asthma must not, under any circumstances, be treated with an airpolishing device. The jet of air and powder could cause respiratory difficulties.

 Contraindication: Patients on a low salt diet must not be treated with the powder containing sodium bicarbonate. For patients on a low salt diet use the powder without sodium bicarbonate provided by EMS.

 The aromatized powder provided by EMS can lead, in specific cases, to allergic reactions to the flavour. For patients known to suffer such reactions, use the powder without flavour provided by EMS.

 Do not direct the jet of powder toward fillings, or crown and bridgework as this could damage the restorations.

 The powder containing sodium bicarbonate provided by EMS can be used only for supra-gingival application. For sub-gingival application, please use the specific powder provided by EMS and refer to its operation instructions.

 Never use "EMS Abrasive Powder" for the treatments carried out with this product.

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To avoid getting powder in the eyes under the contact lenses, the person wearing such contact lenses must take them off.

 A powder jet accidentally directed into the eye can cause severe ocular injury. During treatment, we strongly advise to all persons concerned dentist, hygienist, patient, to wear eye protection.

 To limit the risk of bacterial or viral infections and the powder inhalation, we recommend to the dentist and the hygienist to wear a safety mask of protection.

 The contact lenses or glasses of the patient may get dirty during the treatment. We recommend to take them off.

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**Mundspülung des Patienten**

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**Watterollen einlegen**

 Patient mouthwash during at least 30 seconds with the BacterX<sup>®</sup> pro\* solution will greatly prevent bacteria development during the treatment.

\* BacterX<sup>®</sup> pro is not available in the USA.

 We strongly recommend applying vaseline on the patient's lips in order to avoid their getting dried and chapped.

 The cotton rollers stop the flow of saliva, part the lips and protect the gums.

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 The air/powder jet is powerful. It can cause injury to the gums or an emphysema caused by the introduction of air into the soft tissue spaces. It is strongly recommended that the operator never direct the nozzle directly at the gum tissue or into the gingival sulcus.

 During treatment, make small circular motions.

Finally after the treatment polish all dental surfaces by setting the water flow rate to the maximum.

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

## Precautions to be taken at the end of the treatment

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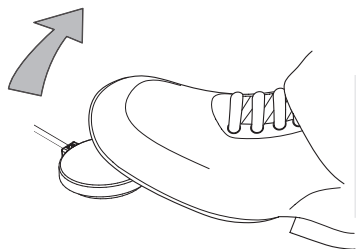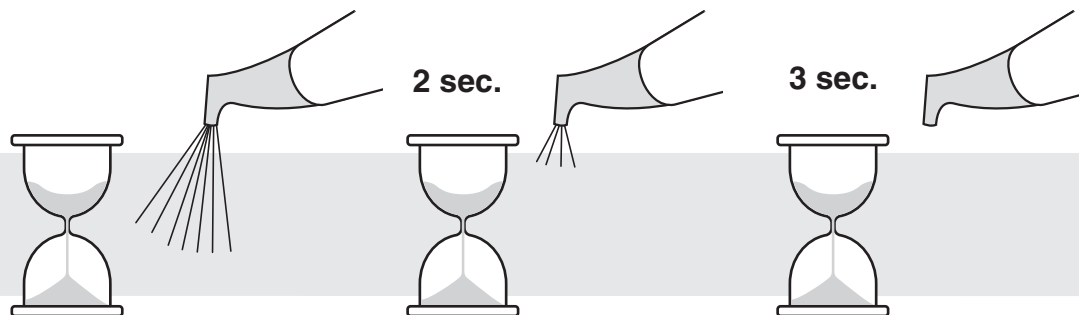 When you remove your foot from the control pedal, the air/powder jet continues for a few more seconds.

You can finish your treatment by taking these seconds into account.

 You can also introduce the nozzle still in the patient mouth into the high-speed evacuator. Let some time for the unit to decompress, with no risk of damaging patient mouth.

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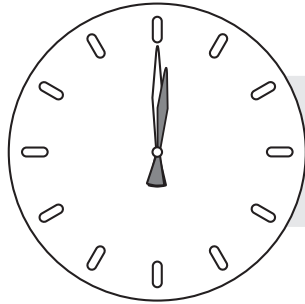
## Information au patient

## Patientenunterweisung

 After the treatment, the teeth are clean and the dental cuticle is completely eliminated. Its reconstitution by salivary proteins requires between 2 and 3 hours. During this time, the teeth do not have any more natural protection with respect to colouring.

 Tell your patient that during 2 to 3 hours following treatment, he should neither smoke, nor consume food or drinks which could strongly colour the teeth (tea, coffee...).

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
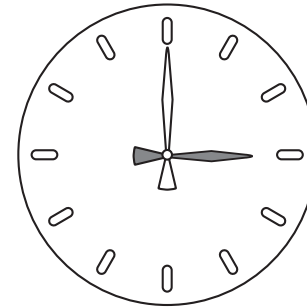
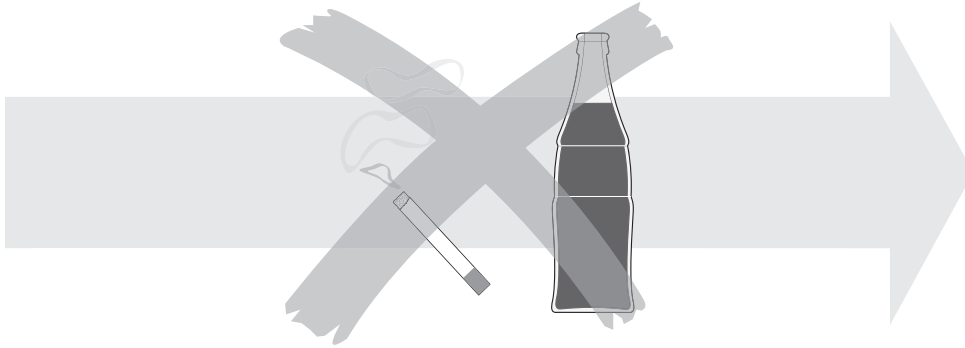
## DISINFECTING, CLEANING AND STERILIZING

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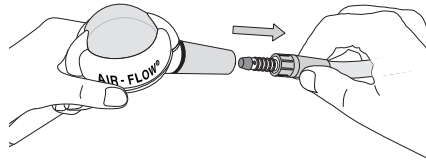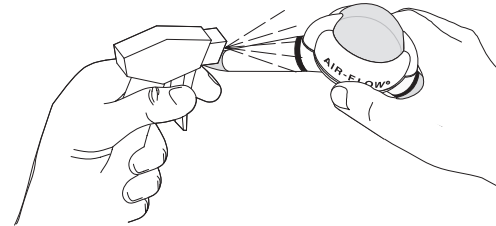
⚠ Clean the unit only with an alcohol-based, commercially available (ethanol, isopropanol), colourless disinfectant. The use of scouring powder or an abrasive sponge will damage its surface.

⚠ Do not submerge the unit in a disinfectant bath. It would be likely to be damaged.

⚠ The unit is not protected against water spray. It cannot be sterilized.

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
## Cleaning the handpiece

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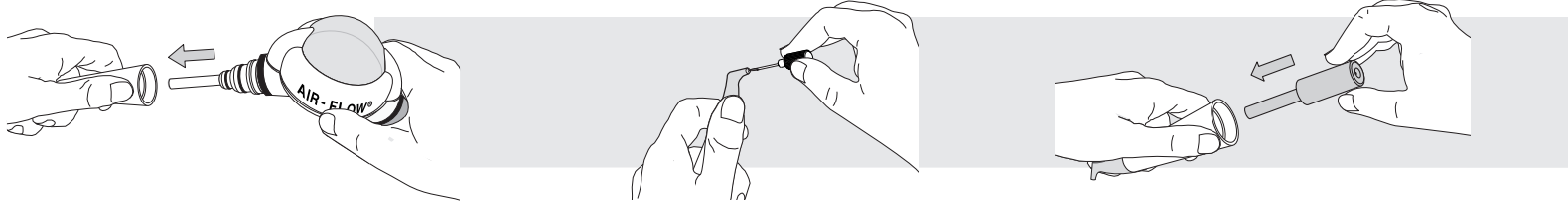## Reinigen des Handstücks


Remove the residual powder present in the tubes by using the cleaning needles. Proceed with precaution and do not use force because the needles can easily break. Use only the tools which are provided to you.

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
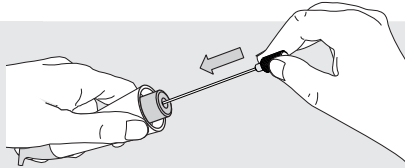
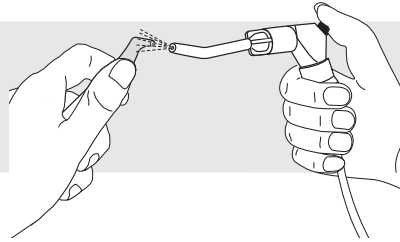
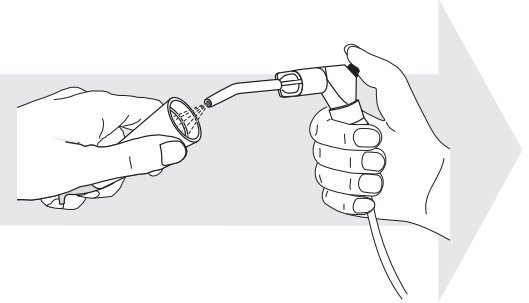
## Disinfecting and sterilizing the handpiece

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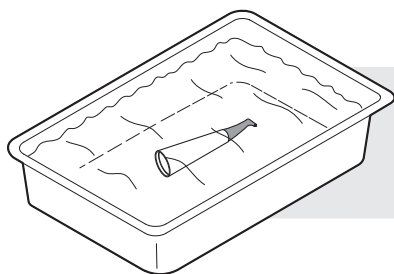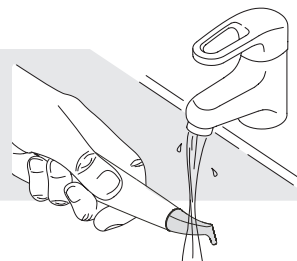 Submerge only the handpiece in a disinfectant bath.

Clean the disinfected handpiece with running water before sterilizing.

 After use, always sterilize the handpiece, only in the autoclave at 134°C (135°C maximum) for at least 3 minutes.

 For sterilizing, please refer to the present-day rules in your country.

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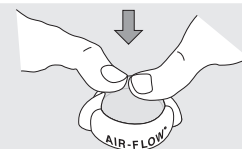
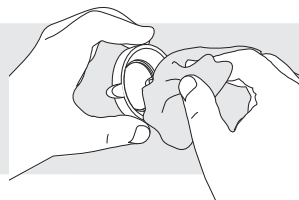
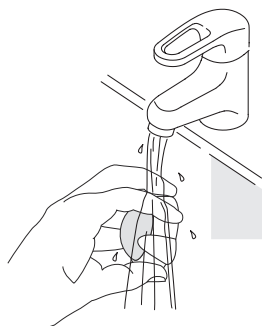
 Clean and disinfect regularly the cap. First rinse it with water, then disinfect it with alcohol (ethanol, isopropanol).

 To facilitate cleaning you can dismount the dome.

 Dome must be correctly positioned in the ring of the cap. The ribs of the dome must be aligned with the lugs of the ring. The two pieces must be correctly aligned to prevent leakage and for pressurization to occur.

 The cap and its seal must be replaced at least once a year.

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 Byt ut defekta delar omedelbart.

## SAFETY PRECAUTIONS

EMS and the distributor of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 Use for the intended purpose only: Before using the product, make sure that you have studied and understood the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 This product must be used only by trained and qualified personnel.

 Always examine the product for damage before commencing treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

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 Before each use, disinfect, clean and sterilize the different pieces and accessories of this product. Please refer to the information provided in the operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 The powders provided by EMS are especially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

 Never use "EMS Abrasive Powder" in the unit as this would damage it.

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## STORING THE PRODUCT WHEN OUT OF USE FOR EXTENDED PERIODS

 Keep the original packaging until the product is to be disposed of permanently. You can use it for shipping or storing your unit at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the chapter "Disinfecting, cleaning and sterilizing"
- Pack the product and all accessories in the original packaging

Please refer to the storage and transport conditions in the "Technical data".

 The powder should not be stored near acids or heat sources.

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## **DISPOSING OF THE PRODUCT, ACCESSORIES AND PACKAGING**

The product, its accessories and its packaging do not contain any substances that are dangerous for the environment.

Should you wish to definitively dispose the product, please comply with the regulations which apply in your country.

## **WARRANTY**

The warranty will be valid for one year from the date of purchase of your product and accessories.

Damages due to non-adherence to the operating instructions or wearout of parts are excluded from warranty.

## **ACCESSORIES**

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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## EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or our approved EMS repair center.

In the case of non-authorized repairs or damaged due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operating instructions.

The connector must be dismantled or changed only by an approved EMS repair center.

You can ask for a modification of the type of adapter of the apparatus, in the range available near an approved EMS repair center only.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing for us.

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## TECHNICAL DATA

### DESCRIPTION

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	AIR-FLOW® handy 2*
Classification 93 / 42 EEC	Class IIa
Mode	Continuous operation
Service Pressure to the turbine connection : Water	0.7 to 2.2 bar (700-2200 hPa) with a service flow of 50-80 ml/min.
Service Pressure to the turbine connection : Air	Static pressure 3.5 to 4.5 bar (3500-4500 hPa)
Weight	Approx. 0.160 kg
Operating conditions	+10° C to +40° C 30% to 75% relative humidity 500 hPa to 1060 hPa air pressure
Storage and transport conditions	-10°C to +40°C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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### Type of problem

### Solutions

Water comes into the powder chamber or leaks from the cap

- Check the dental unit connection
- Check the state of the o-rings of the turbine connection
- Clean the cap and the powder chamber
- Clean the handpiece
- Connect the unit to the dental unit (be careful, the powder chamber must be empty)
- Check the quality of the spray
- Fill the powder chamber
- Send the unit to an approved EMS repair center

---

No powder/air jet coming from the unit

- Cut immediately the air input releasing the foot pedal
- Wait for 1-2 minutes for the system to decompress
- Disconnect the unit from the dental unit
- Press the foot control pedal of the dental unit
- If no air comes out for the turbine connector, it is your dental unit that causes the problem
- If air comes out, it is the device that causes the problem
- Unscrew the cap with the top of the wash-hand basin, but of the powder can be ejected. A sealed apparatus, even disconnected, can indeed remain under pressure
- Empty the powder chamber and screw the cap back on
- Connect the unit to the dental unit (be careful, the powder chamber must be empty)
- Connect the handpiece to the unit
- Press the foot control pedal of the dental unit
- If air comes through the handpiece connector, the handpiece is blocked. Clean the handpiece
- If no air comes out of the handpiece connector, the unit is blocked
- Send the unit to an approved EMS repair center

---

Air and/or powder is leaking from the threads of the cap

- Check the seal and the cleanliness of the thread on the powder chamber and on the cap
- Replace the seal if necessary

---

The unit efficiency diminishes

- A powder refill is perhaps necessary
  - Clean the handpiece
  - Send the unit to an approved EMS repair center
-

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