MIGMA 200-E

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D

PLANNING / MOUNTING GUIDELINES

E

INSTRUCTIONS DE PLANIFICATION / MONTAGE

F

430.8755.10 / 1502

Inhaltsverzeichnis	Table of contents	Sommaire
Einbau und Installation3	Mounting and installation3	Montage et installation3
Anschlüsse (Verkabelung der Geräte)4	Connections (cabling of devices)4	Raccordements (câblage des appareils)4
Bedienung5	Operation5	Maniement5
Einbauabmessungen und Geräteposition Möbelbreite 500mm6	Mounting measurements and device position- ing for device width 500mm6	Dimensions de montage et position de l'appareil, largeur du meuble 500m6
Einbauabmessungen und Geräteposition Möbelbreite 350mm7	Mounting measurements and device position- ing for device width 350mm7	Dimensions de montage et position de l'appareil, largeur du meuble 350mm7
Einbau ohne Versenkung des Rahmens8	Mounting without countersinking the frame8	Montage sans enfoncement du cadre8
Einbau mit Versenkung des Rahmens8	Mounting with countersinking the frame8	Montage avec enfoncement du cadre8
Mischerunterlage8	Mixing device board8	Support de mélange8
Ausbau Bedienpanel9	Dismounting the service panel9	Démontage panneau de commande9
Geräte-Demontage / -Montage bei Einbau ohne Frontöffnung9	Dismounting / mounting the device without front opening9	Démontage / montage de l'appareil en cas de montage sans ouverture frontale9
Austausch MIGMA 100-E zu Neugerät MIGMA 200-E10	Exchanging MIGMA 100-E with new device MIGMA 200-E10	Remplacement MIGMA 100-E par le nouvel appareil MIGMA 200-E10
Wechsel des Bedienpanels MIGMA 100-E zu Neugerät MIGMA 200-E11	Exchanging the service panel MIGMA 100-E with new device MIGMA 200-E11	Remplacement du panneau de commande MIGMA 100-E par le nouvel appareil MIGMA 200-E11
Wechsel des Deckels MIGMA 100-E zu Neugerät MIGMA 200-E11	Exchanging the cover MIGMA 100-E with new device MIGMA 200-E11	Remplacement du couvercle MIGMA 100-E par le nouvel appareil MIGMA 200-E11
Technische Daten12	Technical data12	Données techniques12

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Mounting and installation

Montage et installation

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1. Clue the table cover for MIGMA 200-E to
the worktop of the furniture using silicone.

2. Insert the built-in mixing device into the
furniture. Beforehand, remove the carriage
safety screws (2 pieces). For detailed in-
formation refer to the MIGMA 200 instruc-
tion manual 430.8750.02 which comes
with the device.

3. Cable the device with the table cover.

⚠ Do not connect the device to the mains
power supply (100-230V) before finishing
the mounting.

For detailed information refer to page 4.

4. Bring the mixing device into position.

5. Lift the mixing device and insert the mixing
device board.

🔧 This board helps to remove the mixing
device more easily.
For detailed information refer to page 8.

6. Align the opening of the mixing device
symmetrically with the table cover.

⚠ The mixing device must stand free. No
contact with table cover or furniture side
walls!

A distance of at least 5mm must be kept
to the cover and the walls.

Only use the mains power supply to con-
nect the device (100-230V)

⚠ Do not commission the device before
bringing it into its end position. It is impe-
rative to disconnect the device from the
mains power supply (100-230V) before
maintaining or dismantling any of the built-
in appliances!

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1 Mains power supply 100-230V 50/60Hz at table cover.

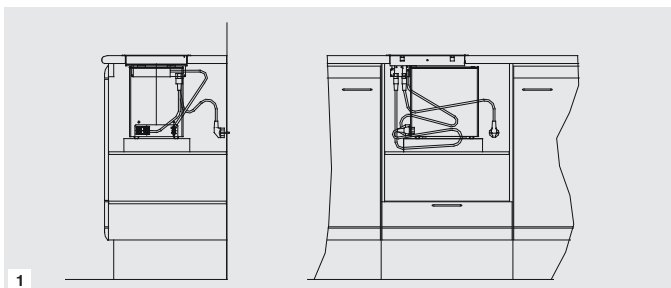
2 Connection line between mains power supply and mixing device (100-230V 50/60Hz).

3 Interconnection of control and safety system.

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

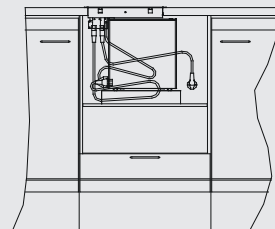
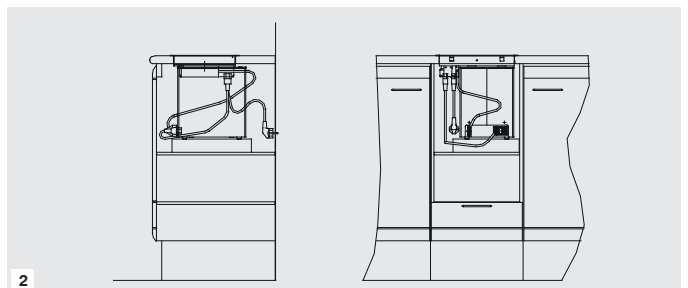
2

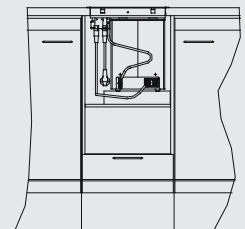1 Mounting in furniture width 500mm
2 Mounting in furniture width 350mm

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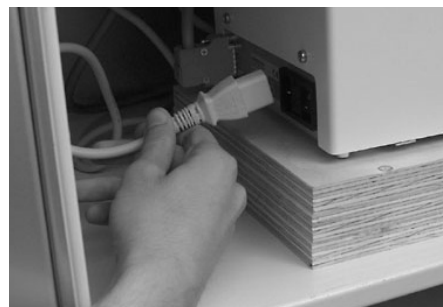⚠ Do not commission the device before bringing it into its end position. It is imperative to disconnect the device from the mains power supply (100-230V) before maintaining or dismantling any of the built-in appliances!

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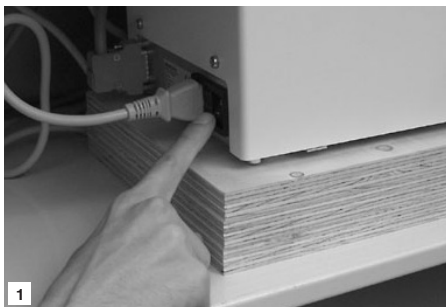
- [1] Main switch
[2] Mixing time regulator 4 - 28 seconds
[3] Start button

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- 1 Always keep the main switch on „ON“.
Use the main switch on the table cover [1]
to activate the device.

- 2 Activate the main switch [1]. The LED on
the start button [3] appears green. Regu-
late the mixing time [2] 4 - 28 seconds.
Insert the mixing bowl with alginate or
another mixing compound.

- 3 Close the cover and press the start
button [3].

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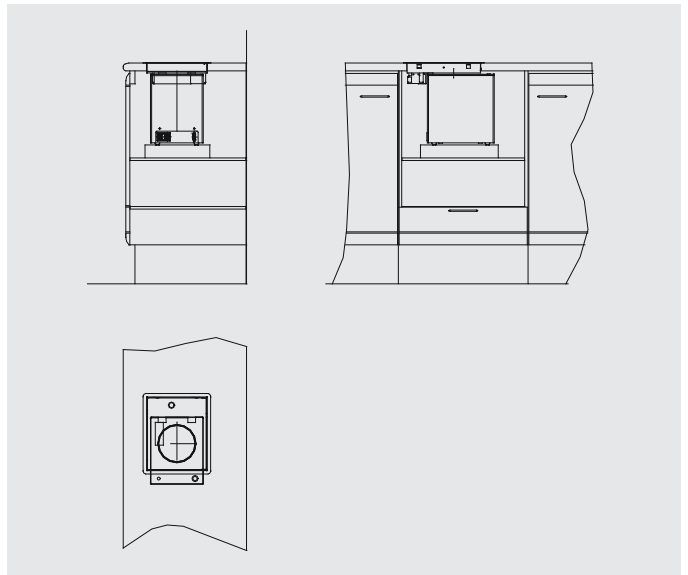
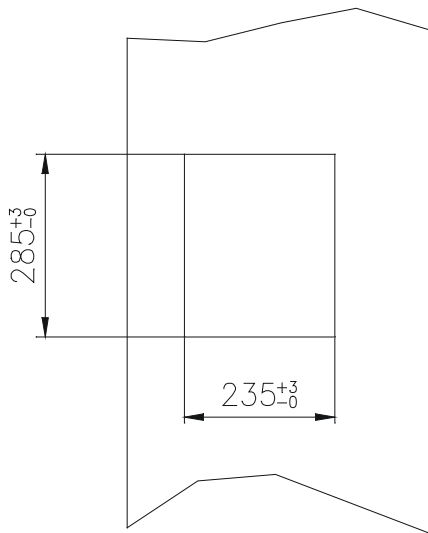
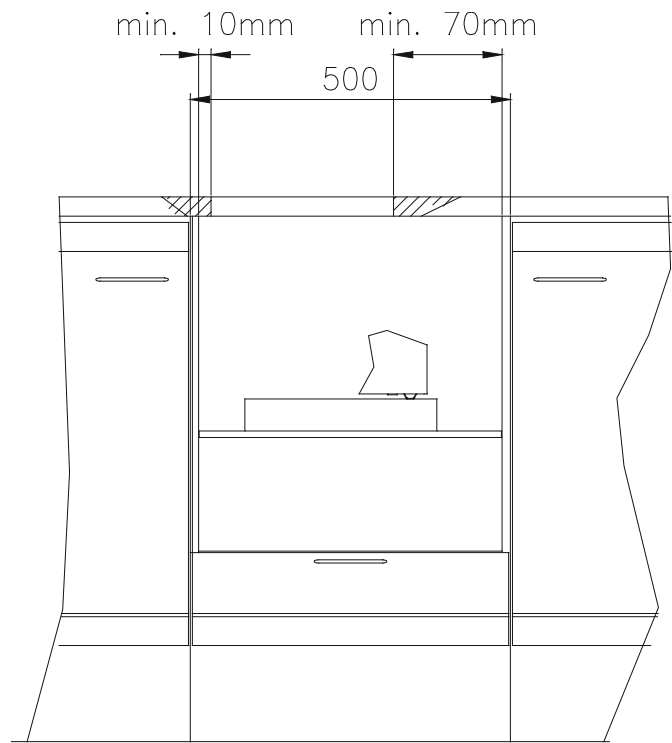
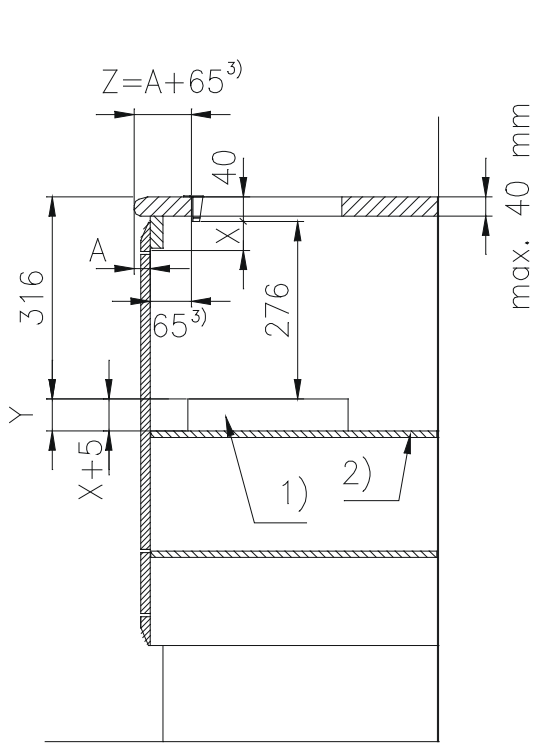
**⚠ For information on the device's hand-
ling and processing see MIGMA 200 in-
struction manual (430.8750.02)**

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Mounting measurements and device posi-
tioning for device width 500mm

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3) The Measurement can be reduced if the furniture width is 500mm.

Minimal distance: 40mm

2) The worktop and the board must have a minimal load capacity of 20kg.

1) Mixing device board for easy removal of the mixing device (see page 8).

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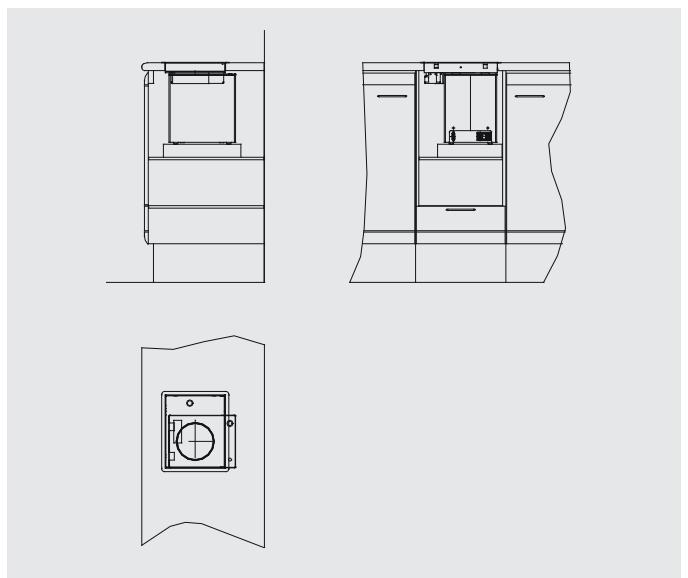
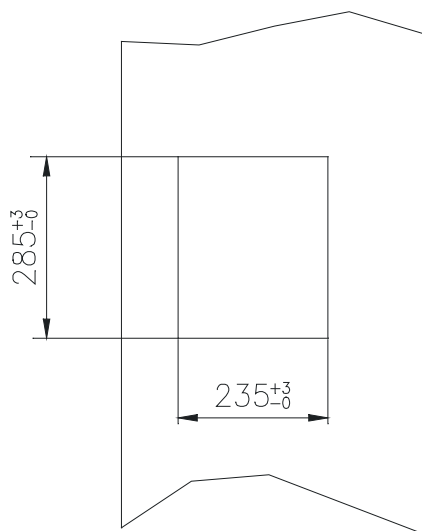
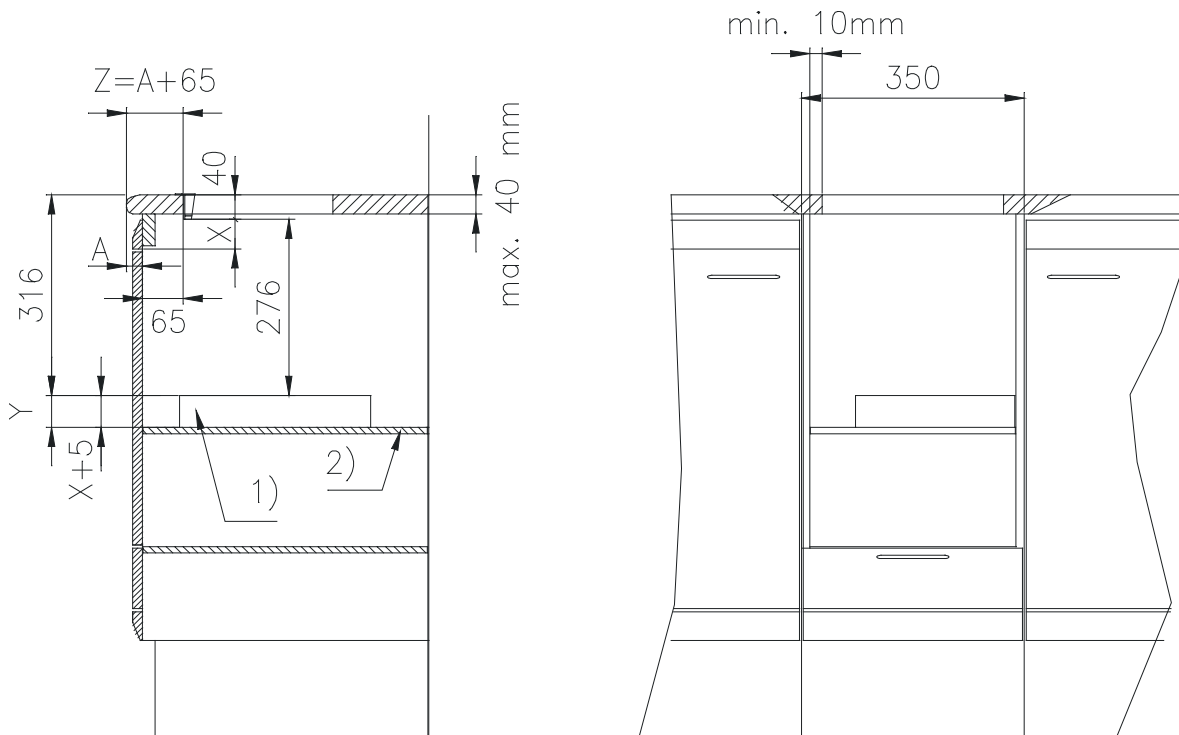
Mounting measurements and device posi-
tioning for device width 350mm

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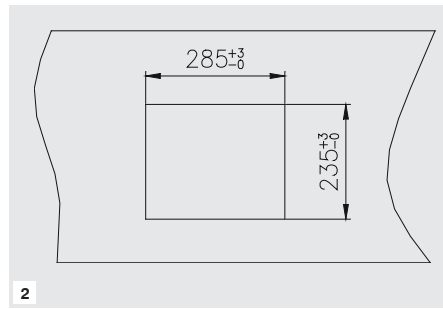
Mounting without countersinking the frame

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
2

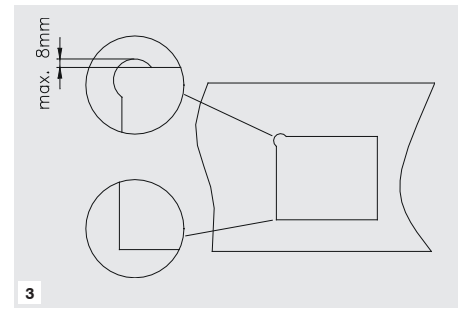- 1 Glue the table cover using silicone.
- 2 Opening of worktop 235 x 285mm
- 3 Cut it out with sharp edges or with positive rounding (max 8mm larger than basic cut).

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

Mounting with countersinking the frame

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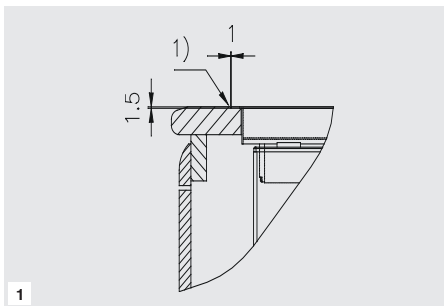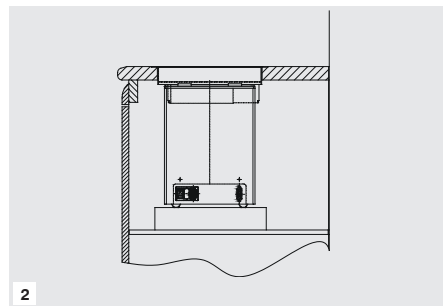
2

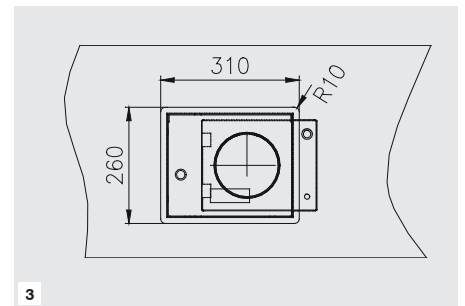- 1 Countersink the frame to the level of the worktop (frame thickness 1.5mm),
1) silicone joint of approx. 1mm
- 2 Table cover and worktop are at the same level.
- 3 Frame size 310 x 260 mm.
Depending on the mounting technique, add approx. 1mm for the silicone joint.

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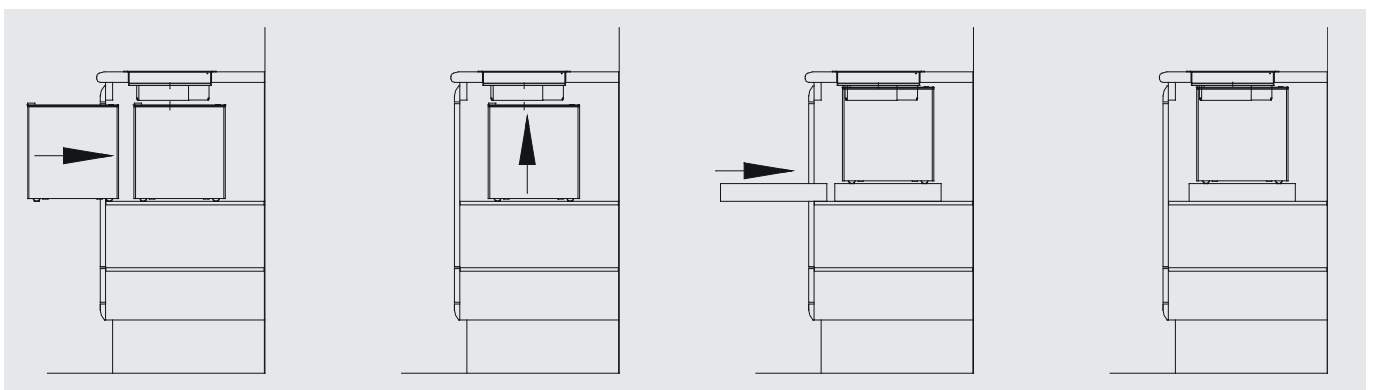
This board serves to remove the mixing device more easily. Depending on the construction of the furniture, the worktop would otherwise need to be adjustable or the device would have to be removed for maintenance work.

Minimal size of the mixing device board:
300 x 250 x Y mm.

The height Y must be laid out according to the furniture end and the position of the worktop.

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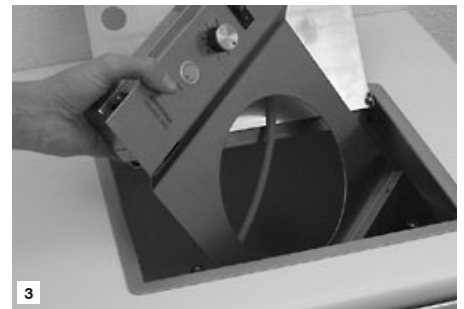
Dismounting the service panel

- 1 Release and remove the two attachment screws.
- 2 Push the service panel down on the right side and release it from the retainer on the left.
- 3 Turn the service panel and take it out through the opening.

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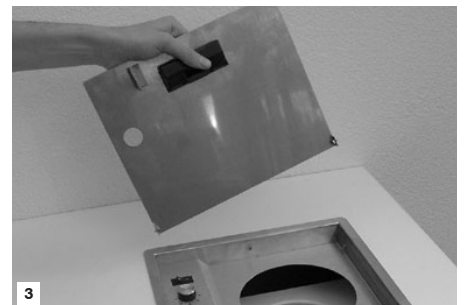
Dismounting / mounting the device without front opening

- 1 Release the counter nut at the hinge on the right and the left.
- 2 Remove the hinge axis on both sides.
- 3 Remove the cover.
- 4 Release and remove the two attachment screws.
- 5 Remove the service panel.
- 6 Take out the device through the opening.

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
**Austausch MIGMA 100-E
zu Neugerät MIGMA 200-E**

**Exchanging MIGMA 100-E
with new device MIGMA 200-E**

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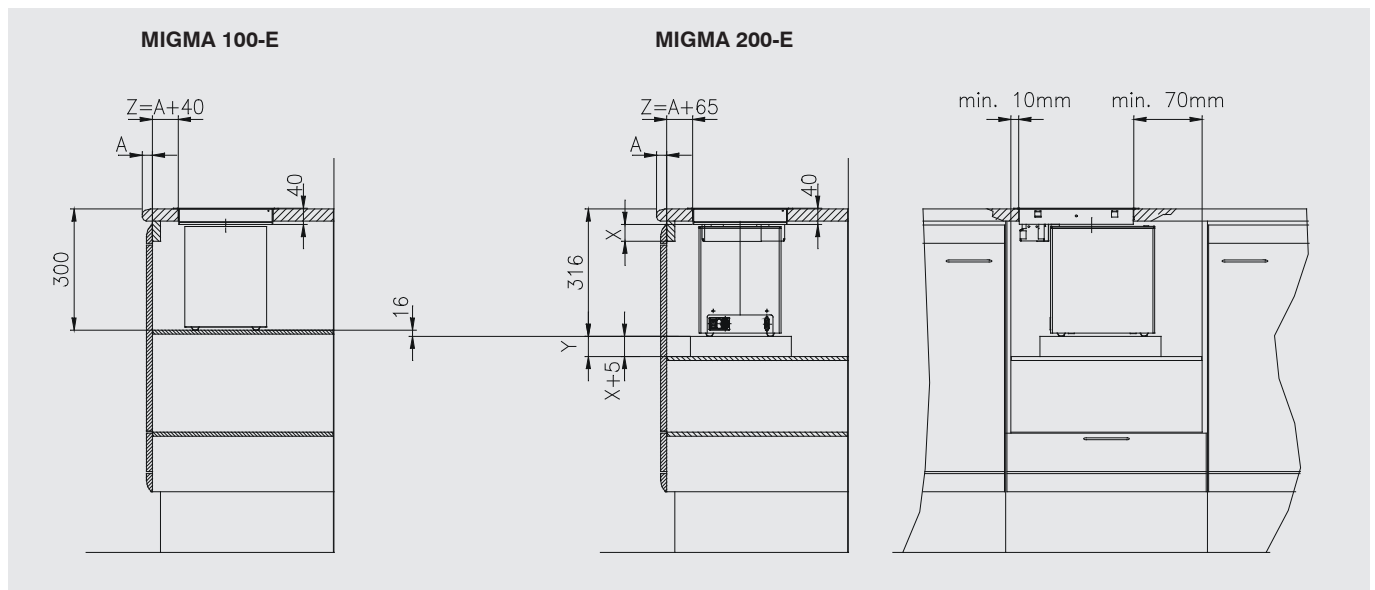- MIGMA 200-E is 16 mm higher than its predecessor MIGMA 100-E.
→ The shelf or the cupboard floor must be lowered by 16 mm.
- We recommend increasing the distance between table-edge and mounting hole by +25 mm compared to the former measures.
→ With a furniture width of 500 mm, the recommended mounting distance of 65 mm can be reduced to 45 mm if the minimal distance to the side wall can be kept.

Same mounting as predecessor!

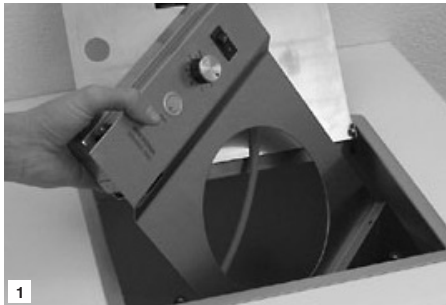
- The size of the frame that is inserted in the worktop remains the same.
→ Therefore, the size of the opening remains the same as well.
→ You can continue using the same frame in the worktop that has been used for a built-in mixing device when changing to MIGMA 200-E. You need, however, to replace the cover and the service panel.

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**Exchanging the service panel
MIGMA 100-E
with new device MIGMA 200-E**

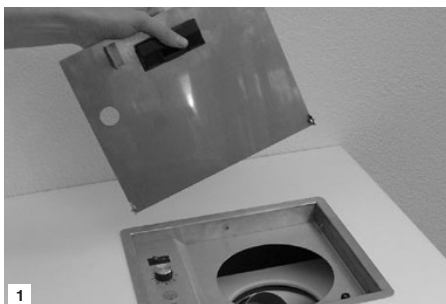
- 1 Remove the service panel MIGMA 100-E and replace it with the new service panel MIGMA 200-E.
- 2 Fit the panel into the retainer on the left side of the frame and pull it up to engage it.
- 3 Attach the panel with two screws on the right side.

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**Exchanging the cover
MIGMA 100-E
with new device MIGMA 200-E**

- 1 Remove the cover MIGMA 100-E and replace it with the new cover MIGMA 200-E.
- 2 Insert the hinge screw on the left and the right and adjust the position of the cover with the positioning and counter nut.
- 3 Tighten up the counter nut at the right and left hinge.

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Dimensions (cm) WxDxH 45 x 29 x 50
 Net weight (kg)19,9
 Gross weight (kg)23,1

Order number

430.6505.01 Universal mixing device
MIGMA 200-E
power plug type A (Schuko)

430.6505.02 Universal mixing device
MIGMA 200-E
power plug type L (Schweiz)

430.6505.03 Universal mixing device
MIGMA 200-E
power plug type K (US)

430.6505.04 Universal mixing device
MIGMA 200-E
power plug type D (GB)

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