



DESS clinical screw recommended torque

ENGLISH (INTERNATIONAL)

CONNECTION	PLATFORM SIZE	DESS REF. N°	DESS RECOMMENDED TORQUE [Ncm]	ORIGINAL EQUIVALENT REF. N°
EXT.HEX.UNIVERSAL (Brånemark® System)	NP	19.027	35	Nobel Biocare® 31168
	NP	19.227	35	Nobel Biocare® 29282
	NP	19.101	35	-
	RP	19.028	35	Nobel Biocare® 29290
	RP	19.228	35	Nobel Biocare® 29283
	WP	19.029	35	Nobel Biocare® 31169
	WP	19.229	35	Nobel Biocare® 29284
TRI – LOBE (Nobel Replace® Select™)	NP	19.004	35	Nobel Biocare® 31170
	NP	19.204	35	Nobel Biocare® 36818
	RP/WP	19.005	35	Nobel Biocare® 29293
	RP/WP	19.205	35	Nobel Biocare® 29475
ACTIVE HEX (Nobel Active®)	NP	19.027	35	Nobel Biocare® 31168
	NP	19.227	35	Nobel Biocare® 29282
	NP	19.101	35	-
	RP	19.030	35	Nobel Biocare® 28815
	RP	19.230	35	Nobel Biocare® 36918
OCTAGON (Straumann® Soft Tissue Level®)	RP (RN) on implant	19.026	35	STRAUMANN® AG 048.356
	RP (RN) on implant	19.102	35	STRAUMANN® AG 049.181
	RP (RN) on abutment	19.009	15	STRAUMANN® AG 048.350
	RP (RN) on abutment	19.103	15	-
CONICAL BL (Straumann Bone Level®)	NP (NC) on implant	19.032	35	STRAUMANN® AG 025.2908
	RP (RC) on implant	19.033	35	STRAUMANN® AG 025.4908
	NP/RP (NC/RC) on implant	19.037	35	STRAUMANN® AG 025.4900
	NP/RP (NC/RC) on implant	19.104	35	-
	NP/RP (NC/RC) on abutment	19.041	15	STRAUMANN® AG 025.0900-04
	NP/RP (NC/RC) on abutment	19.105	15	-
INTERNAL HEX CONIC (ASTRA TECH™)	RP implant level	19.016	20	DENTSPLY® 24449
	WP implant level	19.017	25	DENTSPLY® 24209
	UniAbutment®	19.018	15	DENTSPLY® 22435
INTERNAL HEX FD (XIVE®)	NP/RP/WP	19.024	25	DENTSPLY® 45-4305
INTERNAL HEX USA (Zimmer Screw Vent™ MIS® Implants)	NP/RP/WP	19.012	30	Zimmer® URS2
	NP/RP/WP	19.012L	30	-
	NP/RP/WP	19.042	30	Zimmer® MHLAS / MIS® MN-S0160
	RP on Multiunit	19.110	15	Zimmer® SCTS
EXTERNAL HEX USA (3i® Osseotite®)	NP/RP/WP	19.034	20	BIOMET 3i® UNIHT
	RP on Low Profile® Abutment	19.106	10	BIOMET 3i® LPCTSH
INTERNAL HEX CLICK (3i® Certain®)	NP/RP/WP	19.011	20	BIOMET 3i® IUNIHT
	RP on Low Profile® Abutment	19.106	10	BIOMET 3i® LPCTSH
EXTERNAL HEX BH (BioHorizons®)	NP	19.013	30	BioHorizons® 130-050
	RP/WP	19.014	30	BioHorizons® 140-050
MULTIUNIT®	RP	19.006	15	Nobel Biocare® 29287
	RP	19.206	15	Nobel Biocare® 29285
	WP	19.031	15	Nobel Biocare® 31163
	WP	19.231	15	Nobel Biocare® 29286
	WP	19.231	15	Nobel Biocare® 29286
ANKYLOS® C/X	All platforms	19.048	20	-
	All platforms	19.050	20	-
	All platforms	19.050	20	-
SPECIAL DESIGN (Dyna® System)	Universal 3.6	19.038	32	Dyna Dental 31A51
	Octa	19.039	30	Dyna Dental 81A50
	Extension Level Octa	19.040	30	Dyna Dental 82E50

All DESS® products are manufactured according to ISO 9001:2008 and ISO 13485:2012 and bear the CE marking.

3i® and 3i® Certain®, Miniplant®, MicroMiniplant™ and Osseotite® are registered trademarks of 3i Implant Innovations, USA
 ASTRA TECH™, OsseoSpeed™, ANKYLOS® C/X, XIVE® and FRIADENT® and UniAbutment® are registered trademarks of DENTSPLY Implants
 BioHorizons® registered trademark of BioHorizons, INC.
 Brånemark System®, NobelReplace® and Replace Select™ and NobelActive™ and NobelReplace® CC and NobelSpeedy™ and Multi-unit® registered trademarks of Nobel Biocare AB, Sweden
 ITI®, Straumann® and synOcta® registered trademarks of Straumann Holding AG, Switzerland
 MIS® registered trademark of MIS Implant Technologies Ltd
 Zimmer Screw-Vent® and SwissPlus® registered trademarks of Zimmer Dental INC, USA
 Dyna® registered trademark of Dyna Dental Engineering BV
 Locator® registered trademark of Zest Anchors INC.

The photographs of the products are for information purposes only. The type, reference and connection of each element should be checked. Some DESS® items may not be available in your country. These instructions replace all previous editions. For any detailed information about DESS® products please contact your local distributor.